

CHARGING

1. Plug the USB connector of the AC adapter into the USB port on the PAW unit.
2. Plug the AC adapter into an electrical outlet.
3. A red light on the PAW unit will indicate that the device is charging.
4. A full charge takes about 1.5 hours.
5. When the device is fully charged, the red light will turn off.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of one year. This warranty does not cover device failure due to normal wear and tear or improper use.

TECHNICAL SPECIFICATIONS

OUTPUT VOLTAGE

19.5v

OUTPUT SIGNAL

27.12Mhz

2Hz Pulse

2ms Burst

6.5 mWs/cm³

POWER

3.7v DC lithium ion rechargeable battery


COMPLIANCE

IEC 60601-1

IEC 60601-2



EMbrace RELIEF SYSTEM

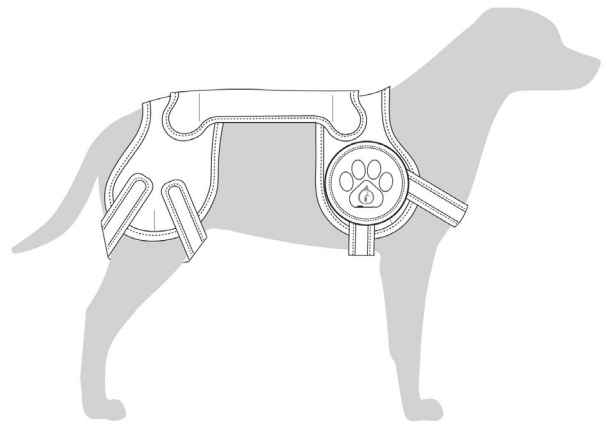

INSTRUCTIONS FOR USE

BEFORE USING THIS DEVICE, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS DEVICE.

INTENDED USE

The EMpower is used for the treatment of chronic and acute pain and inflammation associated with joints and soft tissue trauma, arthritis, and Invertebral Disc Disease (IVDD).

WARNINGS AND PRECAUTIONS

1. **DO NOT** leave dog unattended while wearing the device.
2. **DO NOT** allow the EMpower to get wet.
3. **DO NOT** allow your dog to chew on the EMpower. The EMpower can be fatal if swallowed. Seek emergency medical attention if this occurs.

ADDRESS:

(Headquarters and Returns)
Caerus Corp
1251 Red Fox Road
Arden Hills, MN 55112, USA

Animal OrthoCare
43760 Trade Center Place
Unit 135
Sterling, VA 20166

EMAIL:

aoc@caeruscop.com

WEBSITE:

www.caeruscop.com

PHONE:

651-440-9321


FAX:

651-207-8896


PACKAGE COMPONENTS

QTY	ID	DESCRIPTION
1	A	Garment
1	B	Shoulder Wrap
1	C	Hip Wrap
1	D	Spine Strap
4	E	Hook and Loop Straps
	F	EMpower
	G	Power Button
	H	Indicator Light
	I	Charging Port
1	J	AC Charger (not shown)


A

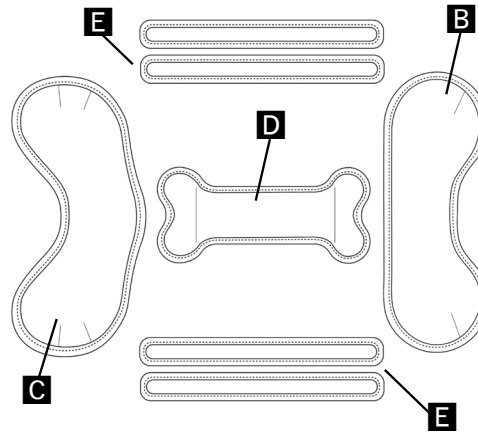


F

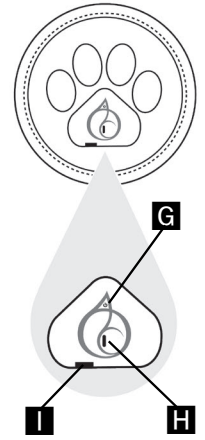


OPERATING INSTRUCTIONS

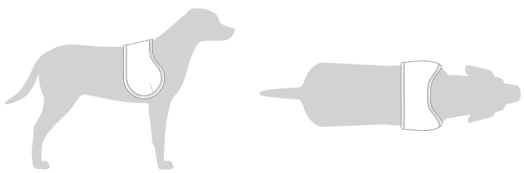


1. Place the Shoulder Wrap across the dog's shoulders.

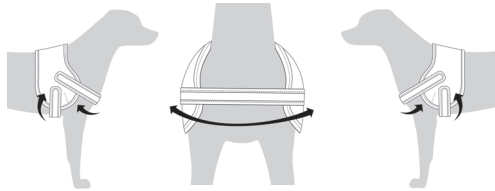



2. Attach the end of one Hook and Loop strap to the right shoulder, extend it across the chest, and attach the opposite end to the left shoulder. Using a second Hook and Loop strap, attach one end to the right shoulder. Wrap under the chest, behind the front legs, and attach the opposite end to the left shoulder.

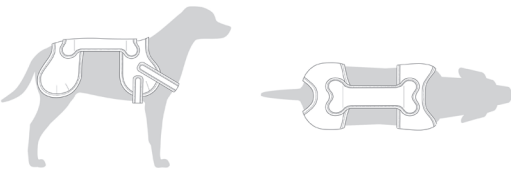

3. Attach one end of the Spinal Strap to the Shoulder Wrap. The Spinal strap should run along the center of the spine. Center the Hip Wrap over the dog's rear legs. Attach the end of the Spinal Strap to the center of the Hip Wrap.

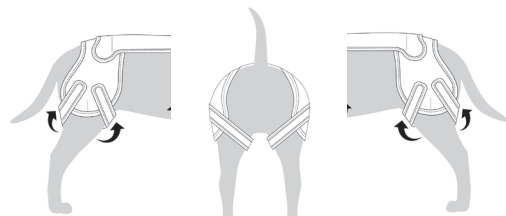

4. Using a Hook and Loop Strap, attach one end to the right rear portion of the Hip Wrap. Loop the strap between the rear legs and attach the opposite end to the right front portion of the Hip Wrap. Repeat on the left side.

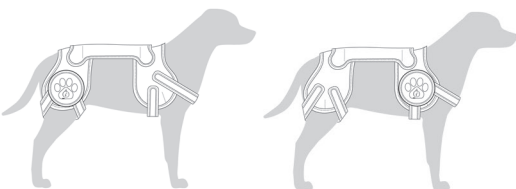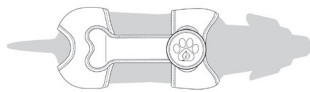

5. Attach the PAW directly over the area where treatment is needed. Press the Power Button to begin treatment. The LED will turn on, indicating treatment has started. The treatment will complete after **two hours**.

NOTE: Your dog may need a few minutes to adjust to wearing the garment. Check the fit to ensure there are no areas of discomfort. Monitor the dog during the treatment period. Remove the Garment and PAW if there are signs of discomfort or attempts to bite or scratch at the PAW unit.

PEMF is subsensory, meaning the frequency cannot be felt against the skin. The best evidence of the effective treatment is increased activity, ease of movement and decreased pain. You should begin seeing results after three to five consecutive days of treatments.