



Visualize Quality®

WARNING!

SENSOR HANDLING PRECAUTIONS

POSITIONING SYSTEMS

Some positioning systems have been found to damage the Bio-Ray Medallion beyond warranty coverage. Avoid applying pressure to the sensor face, and *never* clamp the sensor or the cable with a hemostat or poorly fitting holder.

STATIC ELECTRICITY/ELECTROSTATIC DISCHARGE

The Bio-Ray Medallion contains sophisticated electronic components which are sensitive to static electricity. The sensor should be connected to the control box at all times.

WARRANTY COVERAGE

The Bio-Ray Medallion includes a 2 year warranty which covers manufacturing defects. Visible signs of damage which suggest mishandling *will void warranty coverage* (flattened corners, dented or cracked case, bite punctures, nicked cable, etc). Handle your sensor with care.

BARRIER SHEATHS

We require the use of protective barrier sensor sheaths. If a barrier sheath is not used the sensor warranty may be void. A sample pack of sheaths is included with your system. Additional barrier sheaths are available.

CLEANING

The sensor must never be autoclaved or immersed in liquid. If required, the sensor may be wiped with an EPA approved surface disinfectant towelette. The connector plug should *never* come in contact with moisture. Do *not* use phenolic solutions which may harm plastic.

BIO-RAY MEDALLION EXTENSION CABLE

The Bio-Ray Medallion sensor cable is 9 ft (3 meters), which in most cases is adequate. Power for the sensor is supplied by your computer. If a longer cable is required, a **special** 16 ft (4 meter) USB extension cable is available which will allow the sensor to function normally without a power decrease. Please contact our office for pricing.