












7.4 Calibrating the X-ray head

This operation requires the execution of a certain number of shots, at a pre-set time, during which X-ray emission occurs. It is therefore necessary to pay close attention.

Switch on the handheld unit and go to menu T05. To access this menu press  +  keys, scroll the menu using the button  until P08 is displayed, press  +  to access the technical menu and press the  key several times until T05 is displayed, to enter the menu and press again and hold the key to activate the calibration. press the key  to enter the menu and press again and hold the  key to activate the calibration. Perform the shots until the counter reaches 0. Press the  key to quit the menu. Be careful never to release the shot button in advance as, in case of error, it could be necessary to repeat the operation.

Press simultaneously + keys from T05 to set the use of the default calibration values. Press simultaneously  +  keys from T05 to set the use of the default calibration values.