



**iM3 iPush L7000 Black - L7050 Yellow**  
**Registered Design iM3 Pty Ltd, Made in Australia**  
**[www.im3vet.com](http://www.im3vet.com)**

1. Place iPush onto the LED Advantage Push button handpiece.
2. Depress the lever with your thumb and insert the bur all the way until it stops.
3. Remove the iPush before using the Handpiece.
4. To change the Bur repeat step 2. and remove the Bur.

To increase the life of the Handpiece, Oil between each use and replace the Bur between each patient. See manual