





### Syringe Head Replacement Instructions


1. Turn off air supply to control top.
2. De-pressurize coolant bottles.
3. Unscrew the handle from the syringe head.

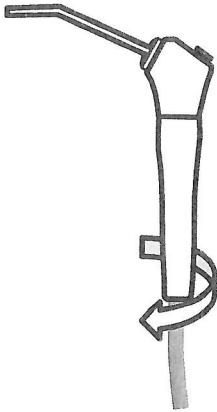


4. Mark the tubes to make sure they go onto the new head in the same positions.
5. Cut each tube adjacent to the white plastic sleeves on the bottom of the syringe head.

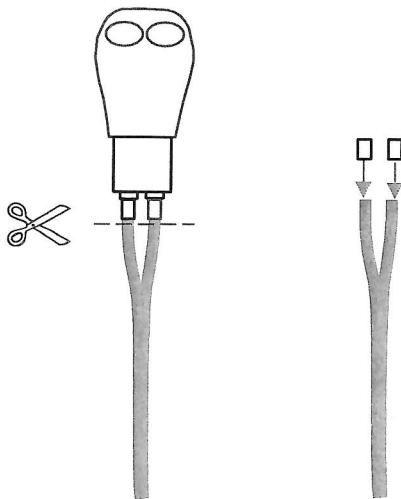

6. Slide the new white sleeves onto the tubing. The sleeves should be pushed on at least 1/2" to provide clearance for installation.
7. Push each tube onto its barb on the new head. Make sure to install them in the same position as before.

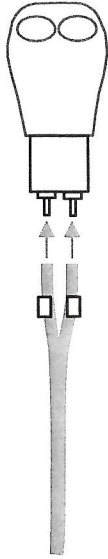

8. Slide the white plastic sleeves to the end of the tubes to lock them onto the barbs.

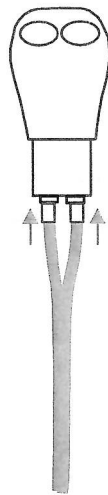

9. Screw the handle back onto the head.
10. Re-pressurize coolant bottles and turn on air supply to dental unit.