

AbbeyShield™ SPRAY ON BANDAGE



ACTIVE CONSTITUENT: 40mg/g ALUMINIUM POWDER
Propellant: HYDROCARBON

Spray it on and it stays! The Water Resistant Aerosol Bandage

What is AbbeyShield™?

AbbeyShield™ is a convenient, water-resistant aerosol bandage that creates a protective barrier against external irritants in simple wounds of small and large animals. AbbeyShield™ is ideal for application to minor cuts and abrasions.

Water Resistant


AbbeyShield™ contains a proprietary carrier that causes the active wound-protective formula to adhere to the skin without washing off in hot or cold water. AbbeyShield™ can be easily removed with soapy water to inspect the wound. The usual reapplication of AbbeyShield™ is daily to every three days to ensure that an optimal bacterial barrier is maintained.

Essential to Wound Recovery

AbbeyShield™ allows the wound to breathe, aiding wound healing. AbbeyShield™ may also be applied over ointment or creams.

Safe

AbbeyShield™ may be used safely on superficial wounds for protection against bacterial contamination. AbbeyShield™ does not contain sulfa or other antibiotics, making it permissible to use on food producing animals.

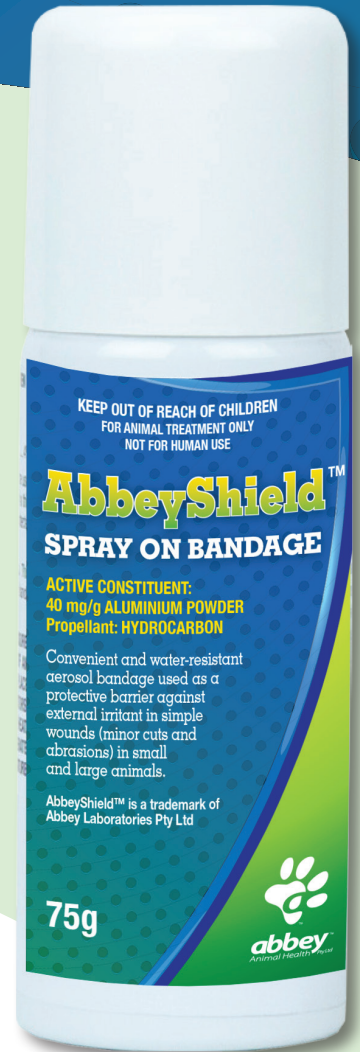

AbbeyShield™ is great for

- Dehorning / disbudding
- Castrating
- Summer sores
- Minor cuts and scrapes



Before and after application of AbbeyShield™

AbbeyShield™ SPRAY ON BANDAGE

ACTIVE CONSTITUENT: 40mg/g ALUMINIUM POWDER
Propellant: HYDROCARBON



Features

- No antibiotic residues
- Fast and easy spray application
- Allows the wound to breathe
- Reduces the risk of infection
- Protection without pain
- Exceptional adhesion
- Removable with soap
- Essential to all first aid kits
- Useful in both small and large animals

PRECAUTIONS

Avoid spraying in animal's eyes. This product stains. Avoid contact with hands and clothes.

Contents under pressure. Do not use in presence of an open flame or spark.

Do not place in hot water or near radiators, stoves or other sources of heat. Do not puncture or incinerate container.

Not intended for use in deep, infected, open/drainage or complex wounds. If in doubt, consult a veterinarian.

DOSAGE AND ADMINISTRATION

SHAKE WELL BEFORE USE.

Clean the area to be sprayed. Apply a thin layer of AbbeyShield™ SPRAY ON to the area to be protected.

Repeat daily or as directed by Veterinarian.

FIRST AID

If poisoning occurs, contact a doctor or Poisons Information Centre. Phone Australia 131126.

Additional information is listed in Safety Data Sheet.

DISPOSAL

Aerosols should be completely emptied before disposal. Place empty can in household rubbish. Do not puncture.

STORAGE

Store below 30°C (Room temperature).

AbbeyShield™ is a trademark of Abbey Laboratories Pty Ltd