

# Instructions for Inject-Ease® using 1.0ml Tuberculin (TB) Syringe

The Inject-Ease® is a single-user device that assists with self-injections by automatically introducing the syringe needle into the skin. This Inject-Ease is for use with most 1.0ml Tuberculin (TB) syringes for subcutaneous injections. The Inject-Ease is not intended to be used for intramuscular injections.


## Cautions

- The Inject-Ease is intended for the single user. To avoid exposure to contaminants, do not use the Inject-Ease on multiple users.
- Consult your health care professional for guidelines on self-injection.
- Prior to use, read all directions and practice self-injections with a needleless syringe.

## Instructions for Use:

The Inject-Ease comes assembled with a 5/8" spacer and 1ml adapter tip (see Figure A).


Before you begin, wash your hands and the injection site and remove the plunger cap from the Inject-Ease. Follow the steps below. Each step corresponds to the illustration on the left.

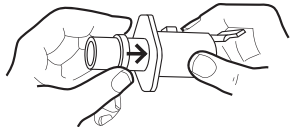

**1. Spacers Rings (Optional):**  
Since there are many 1ml tuberculin syringes available and their dimensions vary, it may be necessary to add a smaller spacer ring so that the needle is not visible when the syringe is loaded in the Inject-Ease.

Included in this package are two additional spacer rings that measure approximately 1/8" and 1/4" (3.17 mm and 5.34 mm) in width.

To check if you need to add a smaller spacer, put the syringe into the device (without medication) by following steps 3-8. Remove the needle cap of the syringe and check that the needle is completely hidden from view. If the needle is completely hidden from view, there is no need to add a spacer. Proceed to step 3.

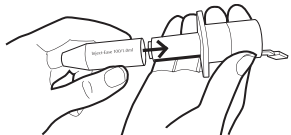

**2. Adding a Spacer Ring if Needed:** Remove the 100/1.0ml tip and 5/8" spacer ring. Add the smallest spacer and re-add the 5/8" spacer ring and 100/1.0ml tip. Recheck if the needle is

completely hidden from view by following the steps listed above. If the needle is completely hidden from view, proceed to step 3.



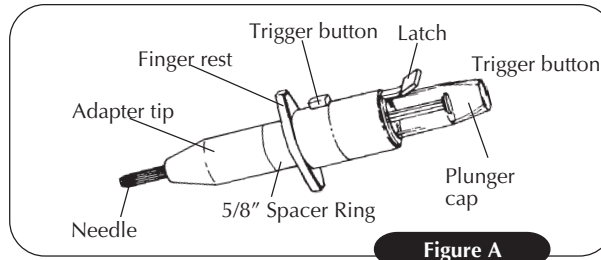



Figure A

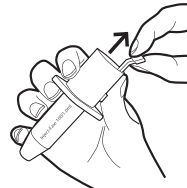


**3.** Pull the latch (T-bar) upward to cock the device.

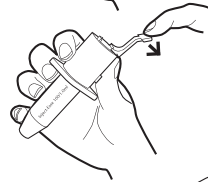

**4.** Push the latch (T-bar) backward to open the locking mechanism.

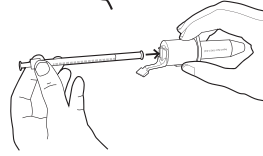


**5.** Insert the syringe containing your pre-measured dose with the **needle cap on** to prevent needle contamination.

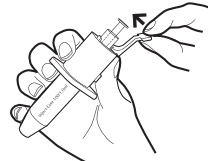


**6.** Push the latch (T-bar) forward to lock the syringe in place.




**7.** Press the trigger button to expose the needle cap. Remove the cap and set it aside.

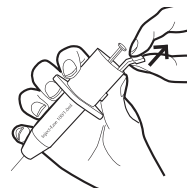


**8.** Pull the latch (T-bar) upward to re-cock the device.

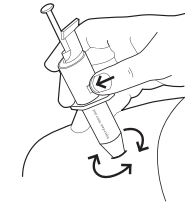

**9.** Hold the Inject-Ease so your thumb is on the trigger. Place the device next to your skin at the injection site and rotate it a few times to desensitize the skin. Now press the trigger button.

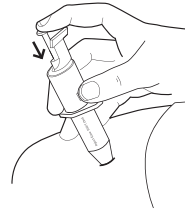

**10.** Push in the plunger of the syringe with your index finger. Injection is now complete. Remove the device from your skin.

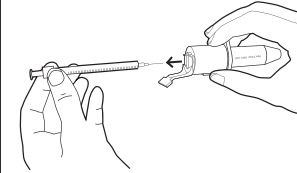

**11.** Pull the latch (T-bar) upwards to cock the device and then pull it backward to unlock the syringe. Lift the syringe from the device and discard in an appropriate sharps or biohazard container.

## Pre-Loading

Do not pre-fill syringes or pre-load them into the Inject-Ease for future use.

## Cleaning

The Inject-Ease can be wiped off with a clean damp cloth or washed with warm soapy water and air dried. To avoid rusting the internal spring, do not submerge the device in water or wash the device in the dishwasher. Adapter tips can be submerged in water after removing them from the device.

## Warranty

This AMBIMEDINC product carries a one (1) year warranty against manufacturer's defects in materials and workmanship and against breakage or failure under normal use. This warranty does not cover misuse or abuse. AMBIMEDINC will, as its option, repair or replace this product at no charge. This warranty gives you specific legal rights, and you may also have other rights under law.