**AANVULLENDE
GEBRUIKERSINFORMATIE**
**INFORMATIONS SUPPLÉMENTAIRES
SUR L'UTILISATEUR**
ADDITIONAL USER INFORMATION
**ZUSÄTZLICHE
BENUTZERINFORMATIONEN**

FOLDING ARMRESTS

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EN FOLDING ARMRESTS

INTENDED USAGE:

The SecuCare folding armrests are a complementary accessory for the SecuCare foldable shower seat and the SecuCare foldable shower chair (with feet). They work as a sturdy support on which the user can further rely while using the wall seats, completing their functionality and making the shower easier and safer.

INDICATIONS AND CONTRAINDICATIONS:

The SecuCare folding armrests are aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip, and to all those who may need further help and support while using the wall seats. The user is required to have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The wall armrests must not be used if the user has problems related to balance or perception disorders.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the wall armrests are correctly assembled.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not modify the wall armrests and their components in any way.
- Do not use the wall armrests if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the wall armrests seat should be folded in their non-use configuration.
- Do not use the wall armrests for a different purpose from the intended one.
- The wall armrests must not be stored outdoors.

MAINTENANCE INFO:

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

The user has to make certain of the structural integrity of the device and its components and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@seccare.nl and the responsible authority in your country.

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