BIOSliving



BIOSliving


Thank you for purchasing the BIOS Living wheelchair. Please review the manual before using. **DO NOT assemble or use** this equipment without first reading and understanding this instruction manual.

Note: This owner's manual was compiled from the most recent specifications and product information available at the time of publication. Thermor Ltd. reserves the right to make changes, which may cause slight variations between the illustration and explanations in this manual and the product purchased.

FEATURES

- Seat width: 18"/46 cm
- Folds easily
- 8"/20 cm front wheels
- 24"/61 cm rear MAG wheels
- Dual locking rear wheels for safe and secure transfers
- Adjustable, detachable footrests
- Comfortable handle grips offer increased comfort and control
- Convenient storage pocket
- Fully padded seat and backrest with all-weather easy to clean nylon covering



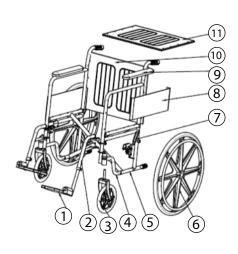

- 1. Footrest
- 2. Metal foot tube
- 8"/20 cm front wheels
- 4. Cross support
- 5. Side support bars
- 6. 24"/61 cm MAG rear wheels
 - 7. Brake
- 8. Metal side plate
- 9. Armrest
- 10. Backrest
- 11. Padded seat

Assembly Instructions

1. Unfold the wheelchair

Push both sides of the seat down to unfold the wheelchair to normal operating condition.


2. Assemble the footrests

Assemble both footrests to the front end of the wheelchair as shown in the diagram. Move and turn both left and right footrests to the normal operating position (facing forward).

3. Fold the wheelchair

Turn the footrests to the vertical position and pull-up on the middle of the seat to fold the wheelchair.



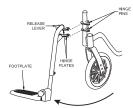



Safety Precaution

- The maximum load for the wheelchair is 300 lbs/136 kg, this includes the rider and any articles he/she may be carrying.
- Drive the wheelchair on smooth ground, and avoid obstacles. Failure to do so may cause the wheelchair to overturn.
- **Do not** use the wheelchair on stairs or escalators.
- Do not use the wheelchair as a seat in a motor vehicle, unless the motor vehicle is equipped with certified equipment to hold and support the wheelchair.
- **Do not** stand on the footrests, standing on the footrest could cause the wheelchair to overturn.

Maintenance

- Check if there are loose bolts before operation and/or within the first month of operation. Tighten loose bolts immediately.
- Check every three months during normal operation, to ensure that all parts are in good condition.
- Check all nuts of the wheelchair (especially the fixing nuts on the rear wheel axle). Adjust promptly, if loose. The connection bolts on the wheelchair cross bars are designed to be loose - no adjustments are necessary.
- Regularly check the movable parts and turning mechanism of the wheelchair, and apply lubricant.
- Dry the wheelchair promptly if it gets wet.
- Regularly wipe the wheelchair with a soft cloth during normal operation and apply antirust wax.

One Year Warranty

The BIOS Living Wheelchair has a 1 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 1 year of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

Thermor Ltd.

Attn: Repair Department 16975 Leslie Street Newmarket, ON L3Y 9A1

www.biosmedical.com

Email: support@biosmedical.com