1. Open the box and follow the steps below


- Remove packaging and all contents from box.
- Lift up front end of backrest and move it forward.
- Hold the seat handle to lift up the chair.
- Make sure the red buckle is set to the locked position when moving.

2. Install the anti-skid suction cups to the base


- The suction cups must be clean before use.
- The tongue of the suction cups must face backwards.
- Make sure to place the suction cups at the edge of the bathtub.

3. Assembling the seat to backrest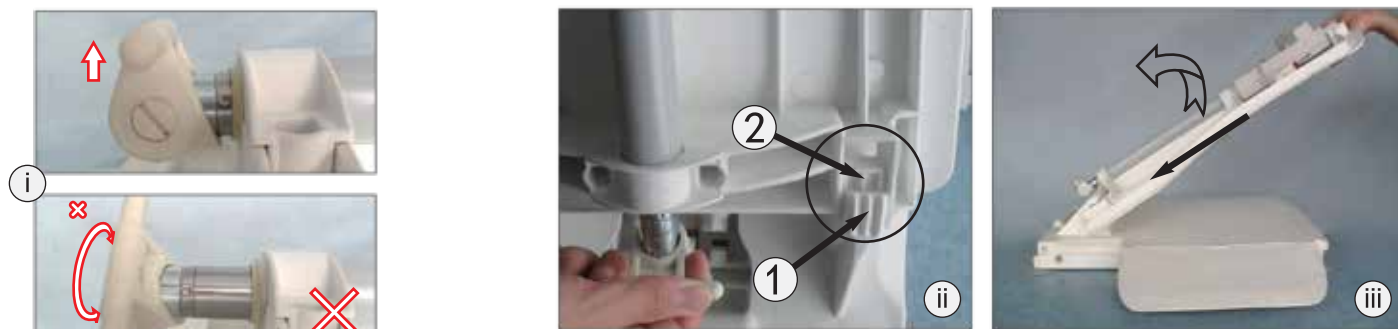



- Warning:** Make sure the push rod is fully retracted and in the correct direction.
- Backrest slides into the axle base which is at the heel end of the seat section.
- Lift up the backrest 45 degrees until it clicks into the axle base. Push the backrest backward until it reaches the lowest position.
- Warning:** Make sure the actuator's heel end is correctly fitted with the bath lift base. Otherwise there is risk of damage.

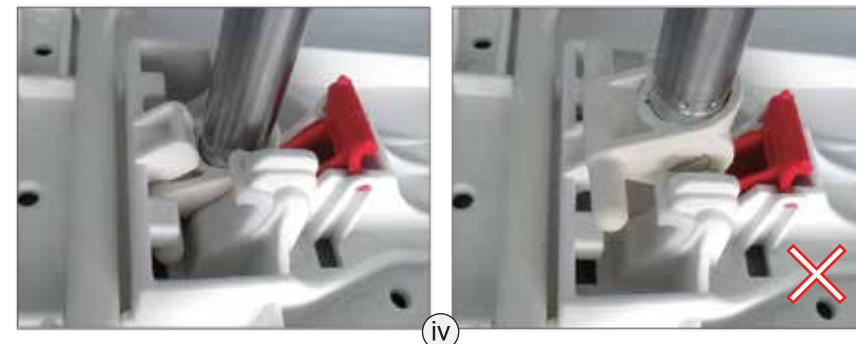4. Hand control with rechargeable battery

Connect the charger plug ① with the hand control connector ②.

- It will take 8 hours to fully charge the battery.
- Warning:** Make sure the charger and hand control are clean and dry when charging.
- The hand control can be put into the water when using the bath lift.
- For storage, please wipe dry and store in a cool dry place.

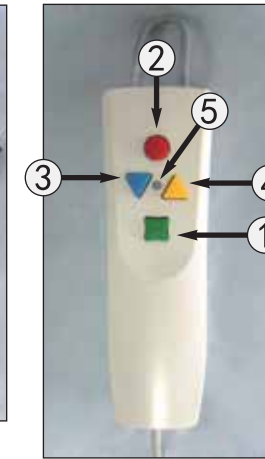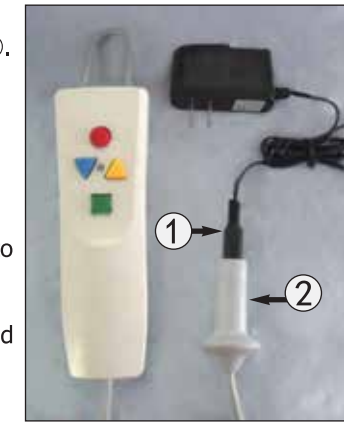- ON button
- Emergency stop
- Lower down
- Raise up
- Power indicator

Plug the hand control connector to the chair back. Make sure the wire is fitted on the pin correctly. Connector should be placed on the red marked position.

5. Placing or removing bath lift from the bathtub

- Place the base of the bath lift in the tub.
- Warning:** The backrest should not crash into the bathtub. If there is any collision, the seat and base should be readjusted.
- The seat must be raised to a proper height before usage.
- To take out the bath lift, retract the actuator to the lowest position and push the backrest 45 degrees forward and pull out.
- Draw each suction cup individually at the heel end to release.
- Note:** wipe the bath lift dry after usage and store in a cool dry place.

6. Using the sliding seat

- Sit on the soft seat.
- Rotate the soft seat and put your legs into the bathtub.
- Slide the soft seat to the seat centre.

Specifications: Weight (including hand control): 27.55 lbs (12.5 kg), Max Capacity: 287 lbs (130 kg)
Working Voltage: 12V, Waterproof Class: IPX7, Power Adapter Input: AC100-240V

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