
### INTENDED USE

Raised toilet seats increase the independence of the user by adding height to most toilet seats. By easing the process of sitting and rising from the toilet, raised toilet seats provide comfort as well as support.

**NOTE:** This product has been designed to fit most toilet seats. It may not fit all toilet seats. Ensure that the unit is correctly installed, fits properly and is stable before each use.

### CONTENTS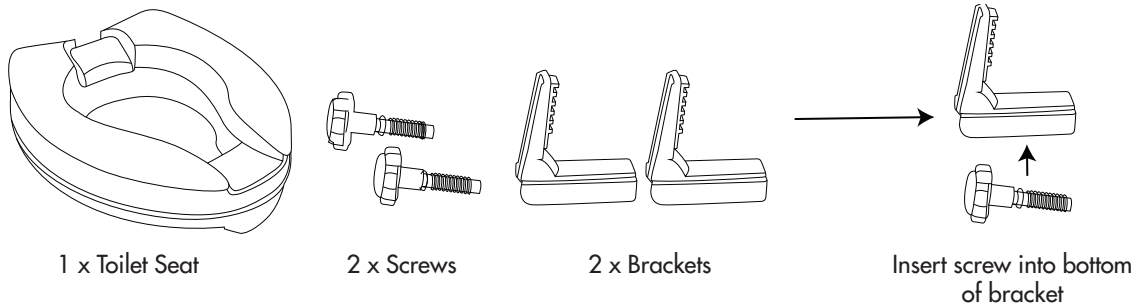

### IMPORTANT INFORMATION

- Always read instructions before starting assembly of the product.
- If any components of the toilet seat are damaged or missing do not use.
- Only use the raised toilet seat if it fits securely on toilet bowl.
- Before each use ensure that the raised toilet seat is correctly and securely locked in place. Check the adjustment knobs on sides of seat before each use to ensure that they are tight.
- During use the users weight should be centred over the raised toilet seat.
- When getting on and off the seat do not hold onto the seat as a support.
- This product has a maximum user weight of **300 lbs/136 kg**.

### INSTALLATION INSTRUCTIONS

1. Remove the existing toilet seat from the toilet.
2. Place the raised toilet seat upside down on a flat surface. **(Figure 1)**
3. Insert one of the brackets with the screw inserted into the slot on the side of the seat. **(Figure 2)**
4. Pushing lightly on the bracket turn the screw clockwise until it catches on the thread. **(Figure 3)**  
Repeat steps #3 and #4 for second bracket.
5. Once both brackets are attached to the toilet seat ensure that the screws are further apart than the width of the toilet bowl. Turn the seat over and place it on the toilet bowl. Ensure that the narrow cut out is at the front of the toilet and that the brackets are near the back of the toilet. The seat should be positioned so that the front lip of the seat is against the outside front of the toilet bowl.
6. Turn both screws on the brackets clockwise until the brackets are tight to the bowl of the toilet. Do not use tools to tighten the screws. Hand tighten only. The raised toilet seat should sit level on the rim of the bowl and fit tightly to the bowl.
7. Before each use check the brackets to ensure that the raised toilet seat is tightly secure.

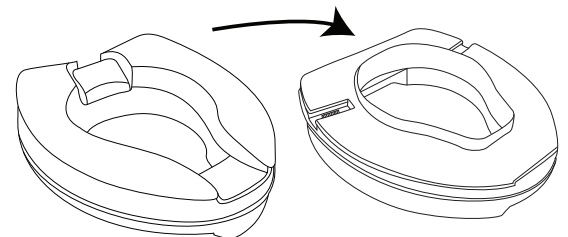
Figure 1

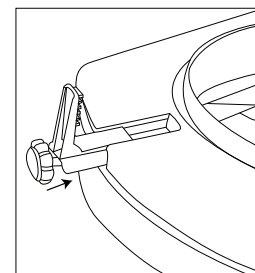
Figure 2

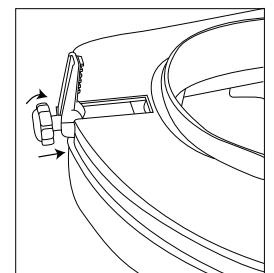
Figure 3

### REMOVAL INSTRUCTIONS

1. Turn screws on the brackets counter-clockwise until the seat is loose from toilet bowl.
2. Lift the front of the raised toilet seat and pull forward to remove.

### CARE AND CLEANING

- Use a non-abrasive detergent or household cleaner with warm water.
- Rinse with warm water and wipe dry.
- Screws can be removed from brackets for a more thorough cleaning.
- Do not use abrasive cleaners on this product or damage may occur.

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