



1. Open Eye Ease. Loosen cap from bottle, but do not remove completely.
2. Place the keyhole slot around the base of the bottle neck. Remove cap from bottle.
3. Close the Eye Ease. Do not force the bottle nose into the Eye Ease, it should close easily.
4. Hold down lower eyelid. Place Eye Ease over the eye with the lip of the Eye Ease against the cheek.
5. Tilt head backwards, look through pin-hole towards the light.
6. Squeeze bottle gently to allow prescribed drops to fall into the eye.
7. Open Eye Ease and replace top on eye drop bottle.
8. To clean, remove eye drop bottle and wash in warm soapy water. Rinse and dry thoroughly.

We recommend that Eye Ease is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded, remove Eye Ease and clean thoroughly before fitting to the new bottle. If you suffer from more than one eye complaint we recommend that you use a separate Eye Ease for each type of drop.

For clinics and hospitals: Eye Ease may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. Eye Ease can also be cleaned in sterilizing fluid. Rinse and dry thoroughly.

One Year Limited Warranty