EZee Life

Banc de douche portatif

Item # Ref
CRP0196 CH2036

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EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

EZee Life

Portable Shower Bench

Item # MFG #
CRP0196 CH2036

Limited Warranty

EZee Life brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

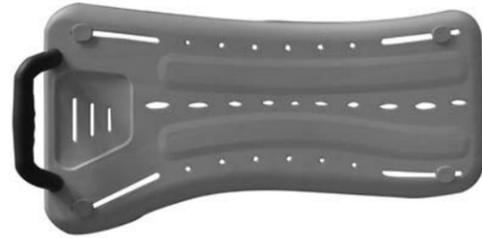
If you have a question about your EZee Life device or this warranty, please contact an authorized EZee Life dealer.

EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

EZee Life

Features

- Portable for easy storage and travel
 - Handle provides additional safety and stability for the user
 - Rubber stops help protect tub from scratches
 - Drain holes prevent pooling
 - Width adjusts from 17.7" to 28"
-

Assembly Instructions

1. Remove contents from box.
 2. Attach handle by threading plastic screws through the bottom of the bench and tightening.
 3. Attach rubber grips by threading the plastic screws through the top of the bench and into the rubber grips, making sure the grey rubber side faces outward. This will be the side that makes contact with the bath tub. **Do not tighten.**
 4. Place the shower bench in the tub with the handle on the outside. This handle provides support into and out of the tub.
 5. Move rubber grips flush against the side of tub and tighten screws.
 6. Make sure bath bench is secure before each use.
-

Care and maintenance

- To maintain the beauty of the original finish, clean with mild soap and rinse with warm water.
- Wipe shower bench dry after each use.

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