

# EZee Life™

## Folding Bath Chair

Model: CH 1063 Pink/Blue



EZee Life™ Products

220 Wildcat Road, Toronto, ON M3J 2N5



## Specifications:

Seat height adjustable base, 14.75" - 18.5" (37.5cm - 47cm)

Removable Ezee cushion back and seat for easy cleaning

Arm height 6" (15.2cm) from seat

Folded footprint 20" x 11" (50.1cm x 27.9cm)

External footprint when open 20" x 21" (50.1cm x 53.3cm)

Seat: 15.75" x 13" (40cm x 33cm)

Padded back height 5" (12.7cm) bottom to 12.5" (31.8cm) top

User weight capacity 176lbs (80 kgs)

## Features:

Folding Frame for easy storage and transportation

Choice of colours (Pink or Blue)

Padded seat, back and armrests

Adjustable height seat

Maintenance free aluminum frame

Carry handle moulded into backrest

Light weight

176 lb (80 kgs) weight capacity

## Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any part that has become defective.


The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

# Operating Instructions

## Folding the Chair

The CH1063 can easily be folded by grasping the top of the backrest (1) with one hand and the front of the seat pan (2) with the other and pull the two together.

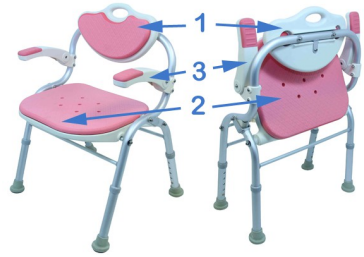


Fold the arms (3) to the up position for a more compact package.

Reverse this procedure to unfold the bath chair.

## Adjusting the Seat Height

1. Depress the pin lock button (1).
2. Slide the lower leg tube to the desired height and line up the adjustment holes (2) with the pin lock and allow the pin to pop through.
3. Repeat this procedure for all four legs.

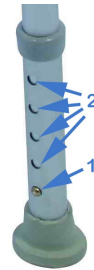

**Note: Ensure all four legs are set to the same height. Failure to set all the legs the same height will make the bath chair unstable and could lead to an accident.**

## Safety Information

Ensure there are no loose or damaged parts before using this bath chair.

Ensure the rubber leg tips are not loose or damaged before using the bath chair. If loose or worn, replace or repair.

To avoid tipping, DO NOT attempt to reach objects that are at a distance that require you to lean towards what you are reaching for.

To avoid tipping, DO NOT lean over the back of the bath chair.

DO NOT sit or transfer into the bath chair unless it is fully open.

Use caution on slippery surfaces.

DO NOT stand on seat.

DO NOT overload this bath chair. The weight capacity is 176 pounds (80 kgs).