

BIOS living

Raised Toilet Seat 59013

Thank you for purchasing the BIOS Living Raised Toilet Seat, please read all instructions carefully before installing this product.

Safety Information

- Weight Capacity: 300 lbs/136 kg

⚠ CAUTION

- Ensure that the seat is correctly and securely locked in place before using.
- The adjustment knob, located at the front of the seat, must be tight at all times to secure the seat to your toilet.
- This product is designed for use with most common types of toilets. It will not fit all toilet types. If you have a style of toilet that this product will not fit properly or cannot be tightened securely - discontinue use.
- The padded armrests are for the purpose of providing assistance while getting ON or OFF the raised toilet seat – **DO NOT** use them to support full body weight during transfer, as this may result in injury.

Installation

- Before each use adjust the padded armrests to your specific needs. When the desired position is reached, lock the armrests in place by aligning buttons, until they 'click' into their holes.
- Check that the arms are locked in place by gently pulling on them.
- Lift your toilet lid/seat assembly and place the rear section of the raised toilet seat inside the inner edge of the toilet bowl (some toilets may require that the seat assembly be removed to accommodate a proper fit).
- Ensure that the front adjustment bracket is sitting inside the toilet bowl clearing the toilet rim.
- With the raised toilet seat resting correctly on the toilet rim, straighten and centre the product over the toilet bowl.
- Turn the adjusting knob clockwise to tighten the raised toilet seat securely to the toilet bowl. Hand tighten only – **DO NOT** use tools and be careful not to over-tighten to prevent stripping of the adjustment bracket.
- Your weight should be centred over the toilet seat, not too far forward or too far to one side, as the seat may tip.
- Ensure that the raised toilet seat is correctly installed, fits properly and is stable before using.

Limited Lifetime Warranty

The BIOS Living Raised Toilet Seat has a limited lifetime warranty to be free of manufacturing defects in materials and workmanship under normal applications. If this product becomes inoperable due to defect, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owner's option) to:

Thermor Ltd.

Attn: Returns Department
16975 Leslie Street
Newmarket, ON L3Y 9A1

www.biosmedical.com

Email: support@biosmedical.com

BIOS | Medical
MANUFACTURED BY/FABRIQUÉ PAR :

THERMOR LTD.

16975 LESLIE STREET
NEWMARKET, ON L3Y 9A1

MADE IN CHINA/FABRIQUÉ EN CHINE

WWW.BIOSMEDICAL.COM

