


BIOS living Commode 59024



Instructions

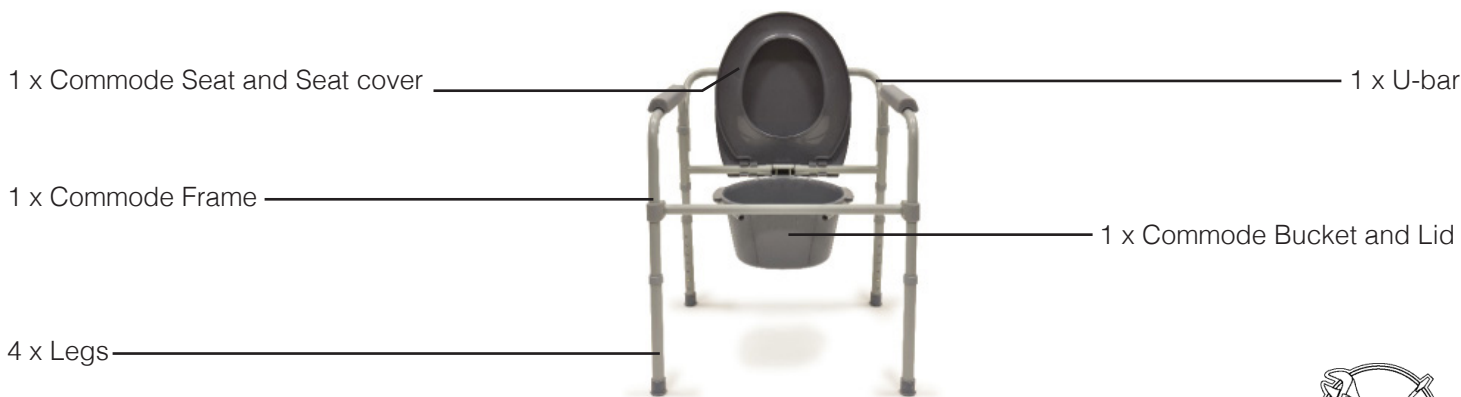

Unfold the commode frame



Insert the front cross bar into the front brackets (located on the front legs of the commode frame). Ensure both metal pins "CLICK" completely into holes




Adjust all 4 legs by moving the metal pins UP or DOWN over frame. Line up the metal pins with the corresponding holes; ensure the metal pins "CLICK" completely into holes. Make sure the height of each leg is the same.



"Snap" on the toilet seat and cover to the back support bar



Attach the U-bar by inserting both ends into the back of the commode frame. Ensure the metal pins "CLICK" completely into holes

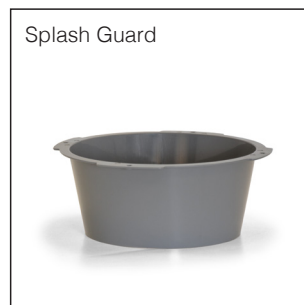

Splash Guard

Use the commode with the bucket for bedside and the splash guard with the commode for over the toilet use

Folding the commode

- Adjust legs to the lowest setting or remove to optimize space (optional)
- Push-in the metal pins and remove the U-bar
- Remove bucket or splash guard
- Push-in the metal pins to release the front cross bar and fold it down
- Fold down the toilet seat and lid
- Fold the commode as shown



Warranty

If this product proves to be defective in material or workmanship within one year of purchase, please return it to the address below. It will be repaired or replaced without charge upon receipt of the unit prepaid with \$20.00 to cover handling, packaging and return postage. Please include proof of purchase, your full name, address, daytime phone number or email address.

Toll Free: 1-800-387-8520

TH3987_59024_Commode_IB_100313

BIOS | Medical
 16975 Leslie Street
 Newmarket, ON L3Y 9A1
www.biosmedical.com