




<b>Project Type</b> <b>Instructions</b>	<b>Date:</b> January 28 2021 <b>Customer:</b> Thermor Ltd <b>Job Number:</b> TH3987 <b>Item Number:</b> 59024 <b>Designer:</b> Alex Vranjesevic	<b>Revision:</b> A	 <b>Contact:</b> Graphic's Department graphics@thermor-ins.com
	<b>UPC (For Reference Only)</b> 	<b>Reason For Project:</b> New Item: New Branding: Design Update: X N/A.	

# BIOS living

## Commode

### 59024

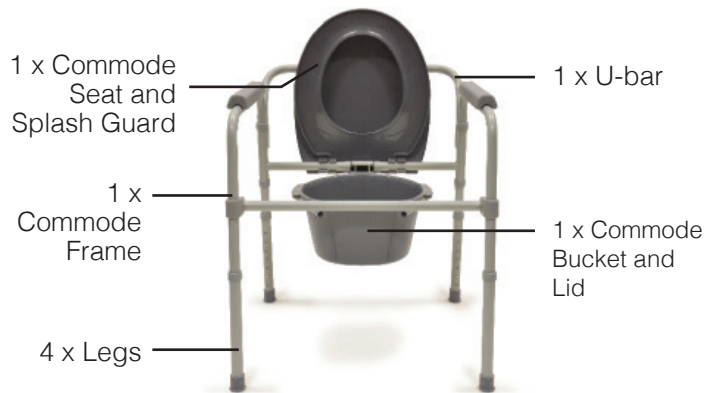

#### Caution

- **Do not** assemble or use this equipment without first reading and understanding this instruction manual. Failure to apply the cautions below could result in injury.
- **Do not** exceed the maximum weight capacity of 300 lbs / 136 kg. Doing so may result in injury.
- Consult with your healthcare provider in determining the suitability of the device for the particular user.

**IMPORTANT:** Always make sure all 4 legs are the same height before using.

#### Instructions



Unfold the commode frame



Insert the front cross bar into the front brackets (located on the front legs of the commode frame). Ensure both metal pins "CLICK" completely into holes



Adjust all 4 legs by moving the metal legs UP or DOWN over frame. Line up the metal pins with the corresponding holes; ensure the metal pins "CLICK" completely into holes. Make sure the height of each leg is the same



"Snap" on the toilet seat and cover to the back support bar



Attach the U-bar by inserting both ends into the back of the commode frame. Ensure the metal pins "CLICK" completely into holes



Splash Guard

Use the commode with the bucket for bedside and the splash guard with the commode for over the toilet use

#### Folding the Commode

- Adjust legs to the lowest setting or remove to optimize space (optional)
- Push-in the metal pins and remove the U-bar
- Remove bucket or splash guard
- Push-in the metal pins to release the front cross bar and fold it down
- Fold down the toilet seat and lid
- Fold the commode as shown



#### One Year Warranty

The Commode has a 1 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 1 year of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

**Thermor Ltd.**  
Attn: Repair Department  
16975 Leslie Street  
Newmarket, ON L3Y 9A1

www.biosmedical.com  
Email: support@biosmedical.com

BIOS | Medical  
MANUFACTURED BY / FABRIQUÉ PAR  
**THERMOR LTD.**  
16975 LESLIE STREET  
NEWMARKET, ON L3Y 9A1  
MADE IN CHINA / FABRIQUÉ EN CHINE  
WWW.BIOSMEDICAL.COM