

<b>Project Type</b> <b>Instructions</b>	<b>Date:</b> March 22 2021 <b>Customer:</b> Thermor Ltd <b>Job Number:</b> TH5114 <b>Item Number:</b> LF759 <b>Designer:</b> Alex Wranjesevic	<b>Revision:</b> A	 <b>Contact:</b> Graphic's Department graphics@thermor-ins.com
	UPC (For Reference Only)  0 57475 26757 9	<b>Reason For Project:</b> New Item: New Branding: Design Update: N/A:X	

# BIOS living

## LF759 TOILET SAFETY FRAME INSTRUCTION MANUAL

- Lightweight aluminum frame provides sturdy support
- Contoured armrests provide comfort and security
- Adjustable height: 28" - 32" / 71 - 81 cm
- Adjustable width: 18" - 25" / 46 - 64 cm
- Weight capacity: 220 lbs / 100 kg

### ASSEMBLY INSTRUCTIONS:

- Step 1. Remove the toilet seat.  
Begin by first removing toilet seat bolts.
- Step 2. Align the mounting bracket with the toilet seat bolt holes. (Figure 1)
- Step 3. Place the toilet seat on top of the mounting bracket and align holes. (Figure 2)
- Step 4. Install bolts and tighten securely.
- Step 5. Slide armrest tubing into the mounting bracket and secure with push buttons. (Figure 2)
- Step 6. Set safety frame to desired size using push buttons. (Figure 4)  
Make sure all buttons are secured before using the product.

### ONE YEAR LIMITED WARRANTY

**CUSTOMER SERVICE**  
**Toll Free: 1-800-387-8520**

BIOS | Medical  
 MANUFACTURED BY / FABRIQUÉ PAR :  
**THERMOR LTD.**  
 16975 LESLIE STREET  
 NEWMARKET, ON L3Y 9A1  
 MADE IN CHINA / FABRIQUÉ EN CHINE  
 WWW.BIOSMEDICAL.COM

Figure 1

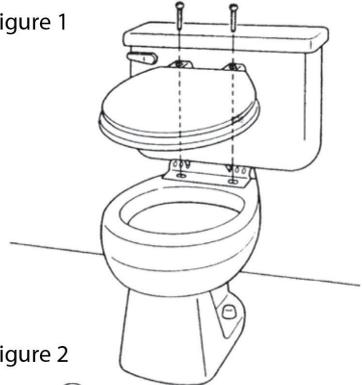


Figure 2

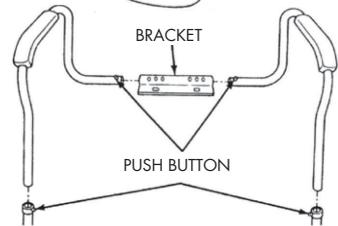


Figure 3

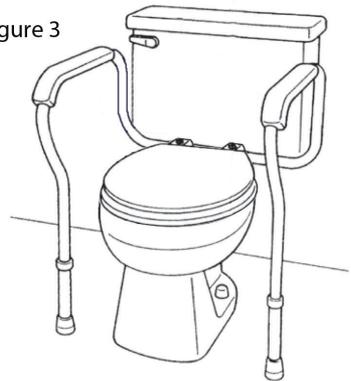


Figure 4

