


Padded Transfer Bench LF740

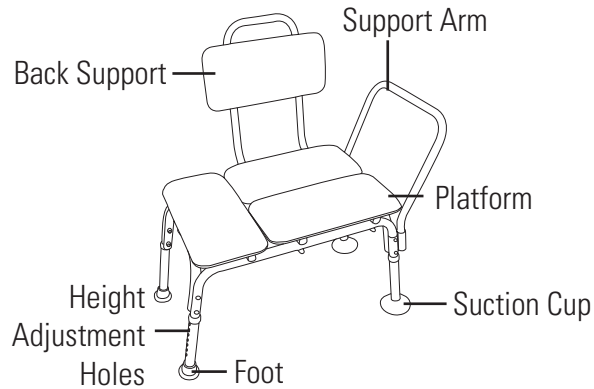

ASSEMBLY INSTRUCTIONS

1. Remove contents from box.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the larger suction cups by pressing the push pin on the attached portion of the leg while sliding the unattached leg portion over top. Repeat for the second leg. **(Figure 1)**.

Note: These legs are inserted into the same side that the arm support will be.

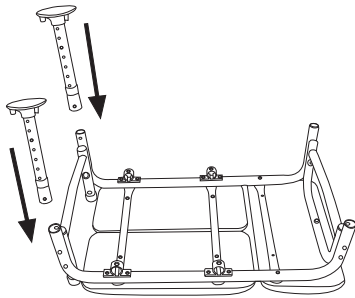


Figure 1

4. Attach leg with the smaller suction cups by pressing the push pin on the attached portion of the leg while sliding the unattached leg portion over top. **(Figure 2)**.

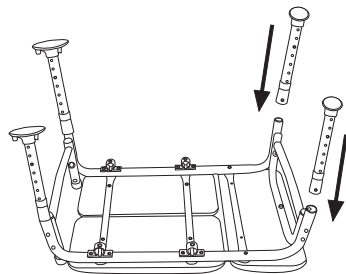


Figure 2

5. Turn transfer bench right-side-up. Attach armrest by pressing push pins and sliding into the arm receptacles on the side of the bench. Make sure the arm arcs outward, not inward. **(Figure 3)**.

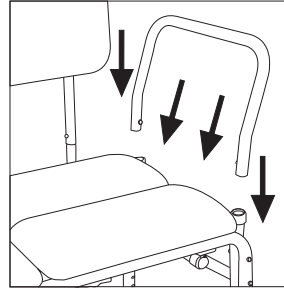


Figure 3

6. Insert backrest into base by pressing push pins and sliding supports through both receptacles. Note: backrest must go through both sets of receptacles to become secure. **(Figure 4)**.

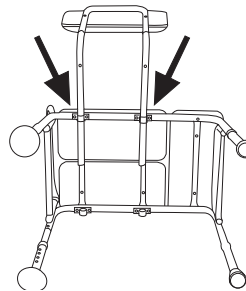


Figure 4

Note: Back support is reversible to accommodate left or right tub entry.

7. Place legs with larger suction cups into bath tubs.

The transfer bench is best used when the bench is level.

HEIGHT ADJUSTMENT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three legs.

Note: Legs inside the tub may be adjusted to a different height than the legs outside the tub; however, the legs inside the tub should be adjusted to the same height, and the legs outside the tub should be adjusted to the same height.

Note: Transfer benches should not be adjusted to a height above the tub wall.

Note: When removing a transfer bench from the tub, always release the suction cups by pinching the two tabs together.

MAINTENANCE

1. Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that the bench is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of the bench.
3. Check fit and tightness of all nuts, bolts and knobs weekly to ensure that the bench is safe to use.

SAFETY PRECAUTIONS

DO NOT ATTEMPT TO ADJUST OR OPERATE TRANSFER BENCH WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Make certain that the height adjustment snap buttons fully protrude through the same respective holes of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Ensure **ALL** screws, nuts and/or bolts are tightened.
- Check all the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- All floor leg extensions with rubber tips **MUST** touch the floor simultaneously at all times.
- **DO NOT** stand on transfer bench.
- BIOS Medical assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

ONE YEAR WARRANTY

The BIOS Living Transfer Bench has a one year warranty to be free of manufacturing defects for the first year of the original owner.

Should repair be necessary, return the unit with all component pieces. Enclose proof of purchase and \$5.00 for return shipping and insurance. Ship the unit prepaid and insured (at owners option) to:

BIOS | Medical

Repair Department
16975 Leslie Street
Newmarket, ON L3Y 9A1
www.biosmedical.com
Email: support@biosmedical.com

Please include your name, return address, phone number, and email address. BIOS Medical will repair or replace (at BIOS Medical's option) free of charge any parts necessary to correct the defect in material or workmanship. Please allow 10 days for repair and return shipping.

TH5664

Manufactured by:
BIOS | Medical
16975 Leslie Street
Newmarket, ON L3Y 9A1
www.biosmedical.com