




<b>Project Type</b> Instructions	Date: May 4 2021 Customer: Thermor Ltd Job Number: TH6849 Item Number: LH514 Designer: Alex Vranjesevic	Revision: A	 Contact: Graphic's Department graphics@thermor-ins.com
UPC (For Reference Only)  0 57475 28514 2	Reason For Project: New Item: X New Branding: Design Update: N/A	Artwork is not to be amended or scaled. If any changes are required please send through Thermor office with instructions, and we will amend here and resend artwork through. <b>PLEASE RETURN WITH SIGNATURE</b>	
		Colour Breakdown 	

# BIOS living

LH514

## Bed Rope Ladder Instructions

Thank you for purchasing the BIOS Living Bed Rope Ladder. This ladder is intended to help aid in rising to a sitting position when lying down.

Please keep these instructions for future use.

### ⚠ WARNINGS:

- Inspect the rope ladder prior to each use for sign of wear. **Do not** use the rope ladder if it appears weakened.
- Be sure rope ladder is securely attached to the bed before each use.

### INSTRUCTIONS FOR USE:

The rope ladder can be used on a single and double bed that have legs.

1. Attach the rope around the legs at the foot of the bed. **(Figure A)**.

Use one of the following methods to attach the rope:

- a. Form turns in the rope and slip them under the legs of the bed **(Figure A-1)**.  
**OR**
- b. Wrap the rope around the leg and twist its around itself **(Figure A-2)**.

2. The user pulls them self-up in the bed by pulling on the runs of the ladder with a hand-over-hand movement.
3. A string (not included) can be attached to the top rung of the ladder and the headboard to help keep the ladder ready to use **(Figure A-3)**.

### CARE AND CLEANING:

Wipe with a damp cloth and a mild detergent and allow to fully dry before use.

### ONE YEAR WARRANTY

The Bed Rope Ladder has a 1 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 1 year of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owner's option) to:

Thermor Ltd.  
Attn: Repair Department  
16975 Leslie Street  
Newmarket, ON L3Y 9A1

www.biosmedical.com  
Email: support@biosmedical.com

