# BIOS living

## Transfer Bench

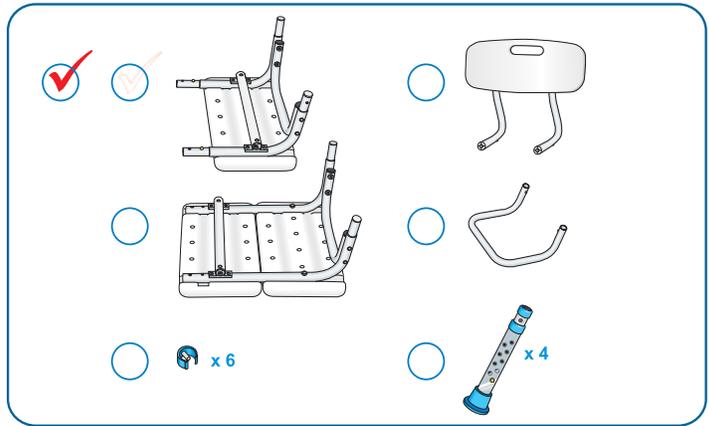
### User's Manual

Weight Capacity 300 lbs/136kg





# 1

## Check all parts

# 2
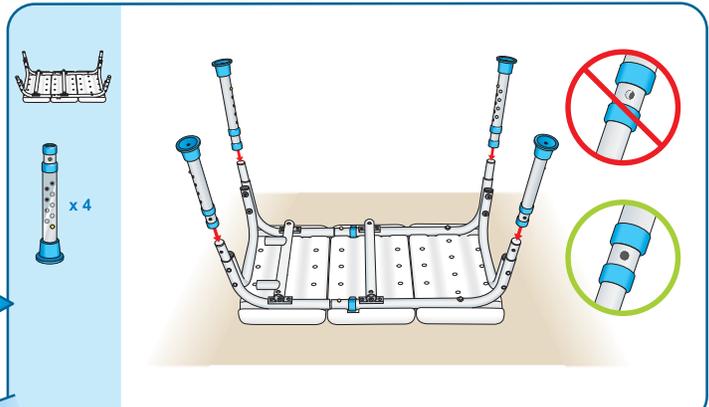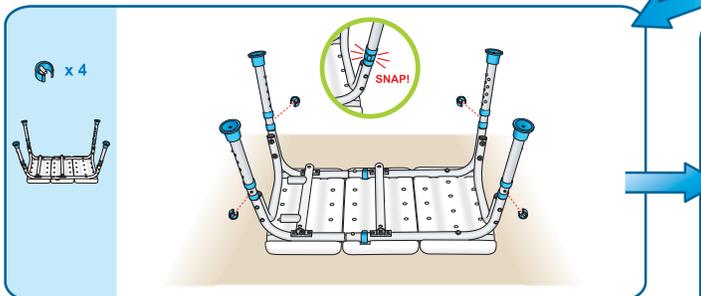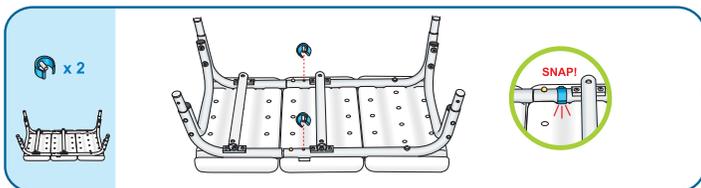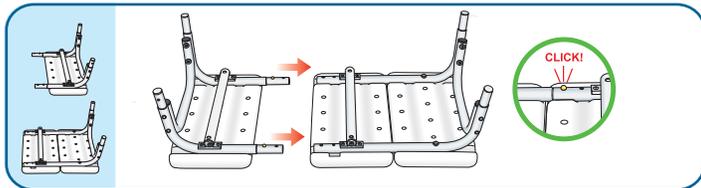
## Tool-free setup

**MAINTENANCE**

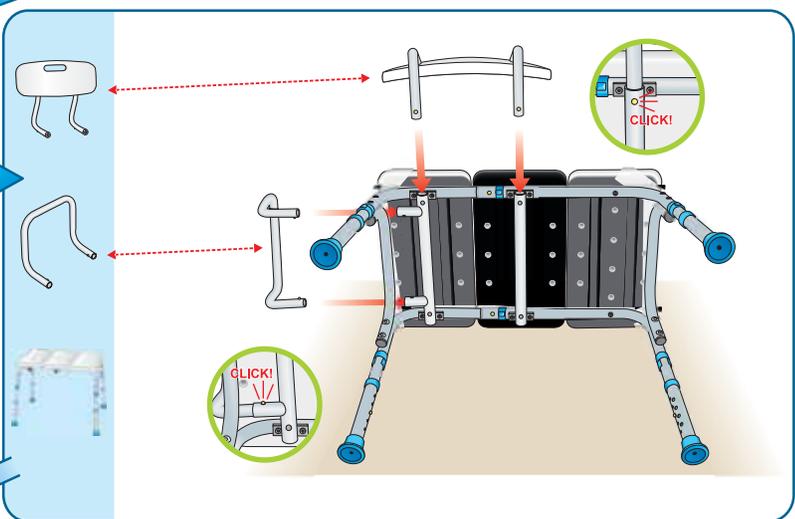
1. Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that the bench is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of the bench.
3. Check fit and tightness of all nuts, bolts and knobs weekly to ensure that the bench is safe to use.

### SAFETY PRECAUTIONS

**DO NOT ATTEMPT TO ADJUST OR OPERATE TRANSFER BENCH WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- Make certain that the height adjustment snap buttons fully protrude through the same respective holes of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- **DO NOT** stand on transfer bench.
- BIOS Medical assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

### Warranty

#### One Year Warranty

This Transfer Bench has a one year warranty to be free of manufacturing defects in materials and workmanship under normal applications for lifetime of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owner's option) to:

**Thermor Ltd.**  
Attn: Repair Department  
16975 Leslie Street  
Newmarket, ON L3Y 9A1

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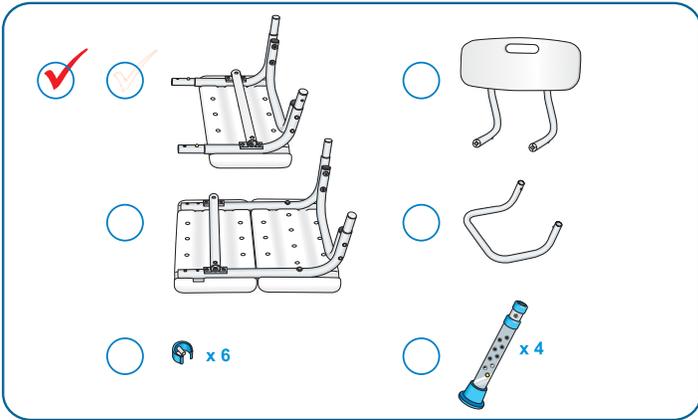
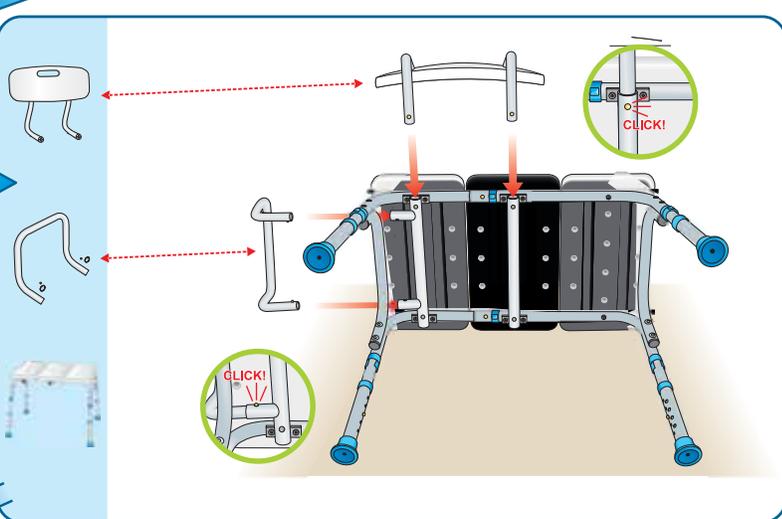
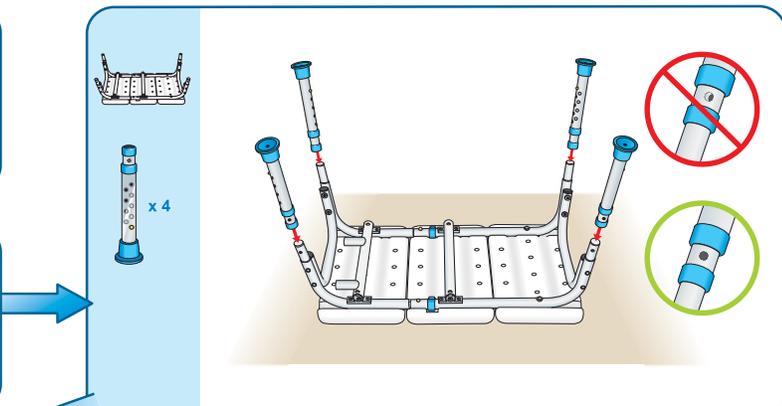
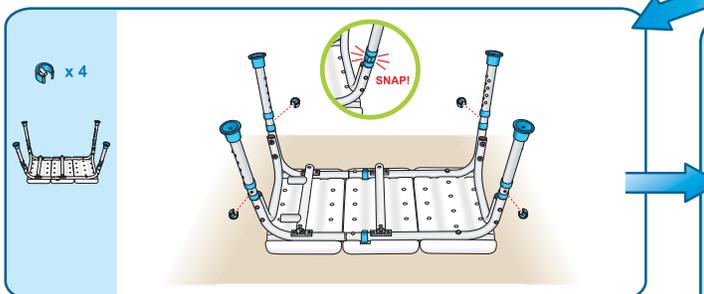
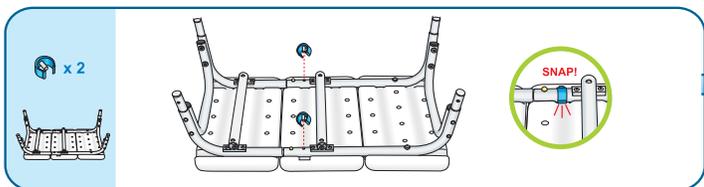
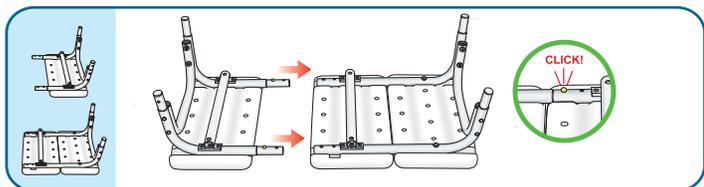
Thermor Ltd.  
Attn: Repair Department  
16975 Leslie Street  
Newmarket, ON L3Y 9A1

www.biosmedical.com  
Courriel: support@biosmedical.com