

Importing Images

To ANY Computer with ANY Practice Management Software

For Windows Office Computers

1. When importing images from SD Card, you will need to have Practice Management System running.
2. Open the Patient file where images and x-rays are retained.
3. Insert the SD Card (or use adapter) with images from the DrQuickLook™ unit into computer. NEVER force.
4. A window will come up with options. Ignore or shrink this window.
5. Go to the patient x-ray/image section of your Practice Management software.
6. *Some Practice Management Systems have an Import button. Click Import.*
7. If Import button not present, Click "**File**" in upper left of screen.
8. Click "**Import**" from dropdown menu.
9. Look for DQL-SD under the word "**Computer**" on the left hand side.
10. If you don't see DQL-SD, click "**Computer**", a dropdown menu will appear.
11. Click DQL-SD to open.
12. Double click DCIM folder and Open
13. Highlight images to be transferred.
14. Save in the Patient's file.

For MAC Users

Simply insert the SD Card in your computer and the images will instantly open and be ready for viewing, then simply transfer to patient's electronic chart.



1-888-346-6153

www.DrQuickLook.com