

SUPERTOUC[®]

NEW



PRODUCT INFORMATION

FFP3 Valved Moulded Mask

Description:

Our FFP3 Valved Cupped Mask offers great level of respiratory protection against airborne particulates and fine dust particles whilst giving you a comfortable fit throughout the day.

- ✓ CE Certified to EN 149:2001+A1:2009
- ✓ Filter efficiency level greater than 99% (FFP3)
- ✓ Category III
- ✓ Latex free elasticated head strap
- ✓ Adjustable nose bridge and soft nose seal for improved fit
- ✓ Valved to avoid heat build-up for greater comfort
- ✓ Cupped shape for comfortable fit

Packaging:

10 Pieces per box - 20 boxes per case

Product Codes:

STP3V2 | Standard | FFP3 Valved Moulded Mask

Standards:

CE 2797 CAT III



Single use only
Do not reuse


PPE Regulation (EU) 2016/425

EN 149:2001+A1:2009 - Respiratory protective devices:
Filtering half mask to protect against particles.

SUPERTOUC[®] How To: Don & Doff Respiratory Protective Equipment

Part 1 Correctly fitting (donning) Respiratory Protective Equipment

1.

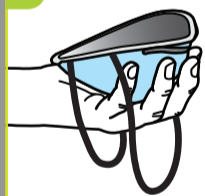

Cup the respirator in one hand with the straps hanging downwards. Check the respirator for any damage. If it is defective, **do not use it** and dispose of it immediately.

2.



Hold the respirator in place on your face. Make sure the nose piece is facing up.

3.



Pull the bottom strap over your head and position it at the bottom of your neck, below the ears. Do the same with the top strap, positioning it at the crown of the head above the ears.

4.



Tighten the straps as desired, and mould the metal nose strip with your fingertips to ensure a secure seal around your face.

5.



To ensure that the mask has no leakage, cup your hands over the filter material and breathe in. If the respirator sucks down hard onto the face, the seal is secure.

6.



Continue to breathe in and out sharply. If the respirator is fitted correctly, you should not feel any leaks around its edges.

Part 2 Correctly removing (doffing) Respiratory Protective Equipment

1.



Do not under any circumstances touch the front of the respirator as it may be contaminated.

2.



To remove, pull the bottom strap over your head, followed by the top strap. Hold onto the top strap, using this to pull the respirator away from your face.

3.



Carrying the respirator by the top strap, discard it into a waste container as soon as possible.

4.



Make sure to wash your hands thoroughly before touching any other surfaces or objects.

www.supertouch.com
sales@supertouch.com
0345 130 9922

Stay Clean. Stay Safe. Stay Alert.

The information shown on this specification sheet is intended as a guide and is advisory only.

All details were correct at the time of issue. As conditions of use are ultimately beyond our control, we advise that the product is tested for the particular application before use.

Issue No.: 1

Issue Date: 09/03/2021