

Hold your baby safely, comfortably, and longer with the Kangaroo Zak

The One and Only Device Ergonomically Designed for Kangaroo Care

Privacy, Effectiveness, and Simplicity:

Before arriving to the bedside, wear the KZ with nothing underneath and with the zippers on either side. You may wear a gown, open blouse, or robe open on the front.

You may transfer the baby from/to the bed to/from Kangaroo Care alone, but ONLY after the training and approval of the Kangaroo Care professional.

While kangarooing, place The Zaky behind the neck to scent it and warm it naturally. Leave it with the baby.

Kangaroo Position
is strictly vertical with legs
and arms flexed and head in a
lateral position between the
mother's breast, to allow maximum
exposure. Diaper no higher than the abdomen.
Preemies must wear a hat to prevent heat loss.

Instructions: Lower top of the KZ (or partially open the zipper), place and position your baby, then cover the baby up to the ears to provide support to the body, head, and neck.

Open the KZ all the way to place leads through the bottom

(optional).

Kangaroo Zak holds the weight and posture of your baby maximizing the area of contact. Provides effective containment, boundaries, thermal-regulation; all with natural fibers, without restraining movement, pressure points, excess fabric, or creases.

2 adjustable unisex sizes available:

Size 1 fits S/M/L
Size 2 fits XL/2XL/3XL
The nurse will help you with proper fit to provide boundaries without compromising respiratory effort

Hands-free

facilitates prolonged, comfortable, and safe sessions. Concentrate on your baby, read, sleep, work on crafts, tc.

interventions (from the top or the bottom and/or in case of emergency.)
Partially open zipper to pump or breastfeed while holding.

Best Practice is Kangaroo Care as soon as possible, for as long as possible, and as uninterrupted as possible (Nyqvist et al., 2010, May Acta Paediatrica)

kangaroo



the parents' comfort



standing and sitting





features for NICU



wash/dry it in cold, delicate cycle



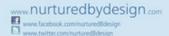

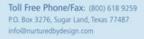



















They help children of any age





kangaroozak。zenbyzak



HANDS Right / Left Gray Cream COLORS AVAILABLE FOR ZAKY Lilac