


Safety & Protection Products



SpeedySwab Rapid COVID-19 Antigen Self-Test

SpeedySwab is an FDA EUA authorized Rapid Antigen Self-Test designed for nasal application and is highly effective in detecting all COVID-19 variants and sub-variants. The test uses the long-proven Lateral Flow Assay (LFA) technology.

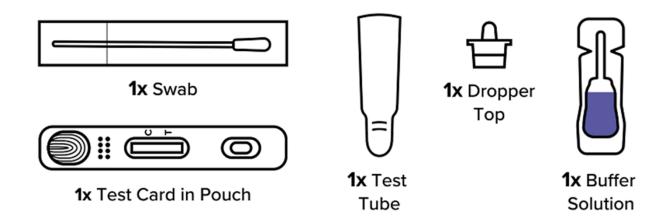

In three easy steps, simply swirl the soft tip of the nasal swab $\frac{1}{2}$ " to $\frac{3}{4}$ " into each nostril for 15 seconds, insert the swab into a test tube with a buffer solution, and apply 3 drops onto the test card. Results will be available in 15 minutes.

SpeedySwab has a reliable design supporting highly sensitive detection of variants and sub-variants. Its simple format is designed for self-administration on adults, or by adults on children for ease of use. And the single test box comes in a convenient size, 5 ½ inches, making it perfect for on-the-go testing.



Safety & Protection Products

Features:

- Highly Sensitive detects proteins from SARS-CoV-2 (COVID-19) variants and subvariants
- Rapid Results provides highly reliable results in 15 minutes
- Reliable designed for self-administration ease of use
- FDA EUA Authorized



Sizes:

SpeedySwab is conveniently sized for all your testing needs:

1 Test/Box: 5.5 x 2.5 x 0.75 inches

2 Tests/Box: 5.5 x 2.75 x 0.875 inches

4 Tests/Box: 5.875 x 3 x 1.5 inches

25 Tests/Box: 9 x 5.7 x 2.75 inches



Components:

- 1 x Test Card
- 1 x Buffer Solution
- 1 x Sterile Swab
- 1 x Test Tube
- 1 x Dropper Top
- 1 x Tube Holder
- 1 x Instructions for Use

