# CareStart™ COVID-19 Antigen Home Test

# USER INSTRUCTIONS

You must follow the test directions carefully to get an accurate result. To obtain the most accurate results, use the instructions **FOR USE UNDER EMERGENCY USE AUTHORIZATION (EUA) ONLY**.

**IMPORTANT: Swabbing the nostril is critical for obtaining an accurate result. If you do not swab your nose, the device will produce a false negative result.**

**1** Wash your hands thoroughly for at least 20 seconds before the test.

**2** Unpack the test components from the tray.

**3** Remove the test cassette and place it on a flat, clean surface.

**4** Locate the extraction vial and gently peel off the aluminum cap. Be careful not to touch the swab tip.

**5** Locate a nasal swab and remove from the pouch. Be careful not to touch the swab tip.

**6** Gently insert the swab no more than 3/4 inch into the **LEFT** nostril. Rotate the swab at least **5 times** in a circular path for a total of **15 seconds**. If you have questions, see the CDC Guidelines.

**7** Gently remove the swab from the **LEFT** nostril and place it into the circular path for a total of **15 seconds**. Remove the swab from the **RIGHT** nostril.

**8** Place the swab into the extraction vial while squeezing the sides of the vial to rotate the swab. Discard the swab in trash.

**9** Remove the vial by rotating against the extraction vial while squeezing the sides of the vial to rotate the swab. Discard the swab in trash.

**10** Close the vial by pushing the cap firmly onto the vial.

**11** With your finger, mix thoroughly by flicking the bottom of the vial.

**12** Invert the extraction vial and hold the sample vertically above the sample well. Squeeze the vial gently. Allow **THREE (3)** drops of sample to fall into the sample well.

**13 Start a timer.** Read the result at **10 minutes.** The test result should not be read after 15 minutes.

**IMPORTANT:** Do not use the test cassette after 10 minutes during this time. Do not use the test cassette after 15 minutes if you are using the mobile app during this process.

**Disposal:** Dispose of all used test kit components and swab samples in household trash.

**Using Mobile Application:**

- Ensure you have an internet connection and the App prior to starting the test.
- Ensure you are using a smartphone or tablet. (For a list of compatible smartphone OS systems, visit [www.accessbio.net/app](http://www.accessbio.net/app))
- Only open the foil packaging of the test cassette after the App instructed to do so.

Please abort the test and follow the In-App instructions if you see any of the following:

- Download and open the App. On (Go™) Mobile Application Store or Google Play Store. Ensure you are connected to the internet during your test.
- Answer a few questions in the App.
- Follow step-by-step instructions for your test.
- Watch the instructional video.
- The App will assist in your visual result interpretation. Please follow the instructions provided in the App. You will be required to take a few questions about the result interpretation.

**COVID-19 Detected (Positive)**

One purple-colored line next to "C" and one blue-colored line next to "T" indicates COVID-19 positive result.

**IMPORTANT:** Look very closely! The color intensity in the test region should be considered as positive.

**Invalid**

Invalid barcode or absence of a purple-colored line next to "C".

Re-test with a COVID-19 test kit if your test has indicated that you are not infected. Interpret the result of the test. You will need to re-test with a new test or consult a healthcare professional if you still have symptoms. You should self-isolate at home and avoid contact with others as per CDC recommendations to avoid spreading the virus to others.

**COVID-19 Not Detected (Negative)**

One purple-colored line only next to "C" indicates a negative result.

**IMPORTANT:** Look very closely! The color intensity in the test region should be considered as positive.

Re-test in 24-48 hours if your first test result is negative. A negative test result indicates that antigens from SARS-CoV-2 were not detected and the patient is unlikely to be infected with the virus and presumed to be contagious. Test results should always be considered in the context of clinical presentation and epidemiologic data. Test results should be used in conjunction with management decisions. You should self-isolate at home and avoid contact with others as per CDC recommendations to avoid spreading the virus to others.

**Results Interpretation**

**Make sure you wait the full 10 minutes.**

Your test results by following the in-app interpretation instructions are more precise.

**NOTE:** The test results should be read by visual and interpreted at 10 minutes after the sample application and interpretation instructions are followed. Exceeding 15 minutes may yield inaccurate results.

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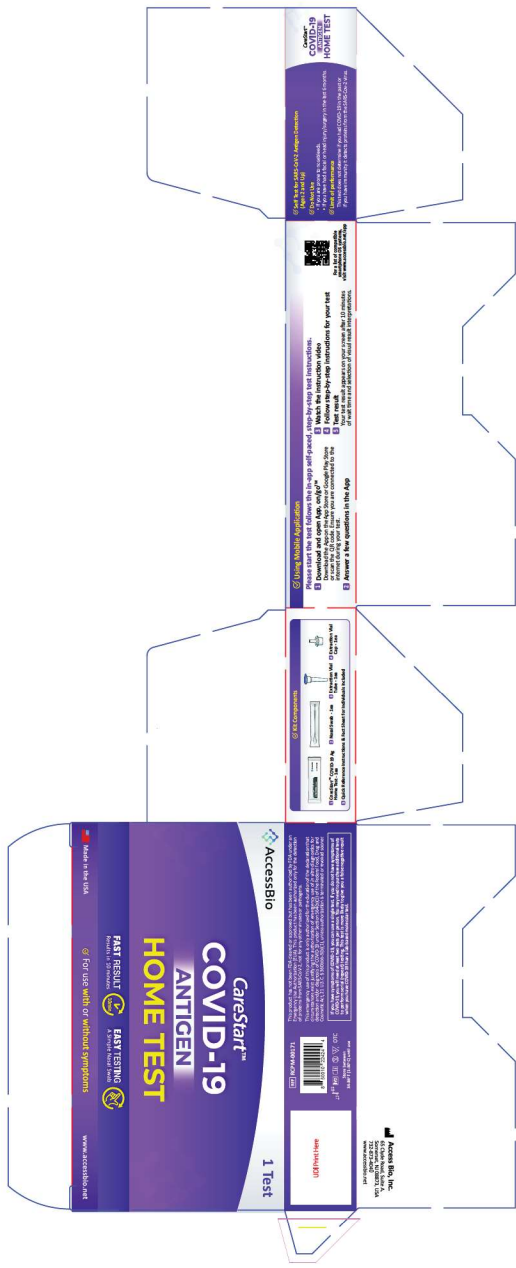
**CareStart COVID-19 ANTIGEN HOME TEST**

Use this test to check for COVID-19 antigen in your nasal mucus. This test is for use only by you or someone you know. It is not for use in a laboratory or clinical setting. This test is not for use in a laboratory or clinical setting. This test is not for use in a laboratory or clinical setting.

**AccessBio**

FAST RESULT FAST TESTING

For use with or without symptoms

www.accessbio.net

2 Tests

**Instructions for Use**

1. Wash your hands with soap and water for at least 20 seconds before and after using the test.

2. Remove the test from the box and open the cap.

3. Insert the swab into the test and follow the instructions on the test.

4. Read the result within 15 minutes.

**Download the CareStart app**

Download the app on the App Store or Google Play. Scan the QR code to download the app. The app will help you follow the instructions for the test and provide you with a digital copy of your results.

**Follow the instructions for your test**

1. Wash your hands with soap and water for at least 20 seconds before and after using the test.

2. Remove the test from the box and open the cap.

3. Insert the swab into the test and follow the instructions on the test.

4. Read the result within 15 minutes.

For use with or without symptoms  
www.accessbio.net

# CareStart<sup>™</sup> COVID-19 ANTIGEN HOME TEST

AccessBio

AccessBio

# CareStart<sup>™</sup> COVID-19 ANTIGEN HOME TEST

**FAST RESULT**  
Results in 10 minutes

**EASY TESTING**  
A Simple Nasal Swab

For use with or without symptoms  
**4 Tests**

**Using Mobile Application**  
You will need to download the mobile App and follow the test instructions for testing.

**1 Download and open App on App Store**  
Download the App on the App Store or Google Play Store or scan the QR code to have your phone connect to the internet during your test.

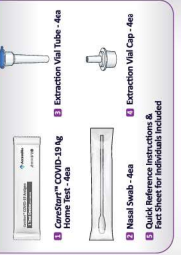
**2 Answer a few questions in the App**  
**3 Watch the instruction video**  
**4 Follow step-by-step instructions for your test**

**5 Test result**  
The App will assist in the visual result interpretation. Please use the right information provided in the App and then look at the device and answer some questions to the result interpretation.

## CareStart<sup>™</sup> COVID-19 ANTIGEN HOME TEST

**KIT Components**

- CareStart<sup>™</sup> COVID-19 Ag Home Test - 4ea
- Nasal Swab - 4ea
- Extraction Vial Tube - 4ea
- Extraction Vial Cap - 4ea
- Quick Reference Instructions & Fact Sheet for Individuals Included

UDI Print Here

Store between 34-86°F (1-30°C) until use  
Made in the USA www.accessbio.net

3411020202020  
V911120202020  
2011010101010

## CareStart<sup>™</sup> COVID-19 ANTIGEN HOME TEST

**Self Test for SARS-CoV-2 Antigen Detection (Ages 2 and Up)**

- Do Not Use**
  - If you are prone to nosebleeds.
  - If you have a facial or head injury/surgery in the last 6 months.
- Limit of performance**  
This test is only for use if you have recently had symptoms of SARS-CoV-2 virus.

This product is FDA approved for use as a self-test for SARS-CoV-2 virus. It is not intended for use as a diagnostic tool for any other virus or for pathogens. This emergency use of this product is not authorized for the detection of emergency use of in vitro diagnostics for infectious and/or emerging viruses, including SARS-CoV-2, U.S.C. § 360bbb-3(b)(1). Unless authorization is granted by the FDA, the use of this product for the detection of SARS-CoV-2 virus is not authorized.