


Parts Reference - O2 Tank Carrier for Rollator (Item #53000)



53000

For D & E O2 Tanks

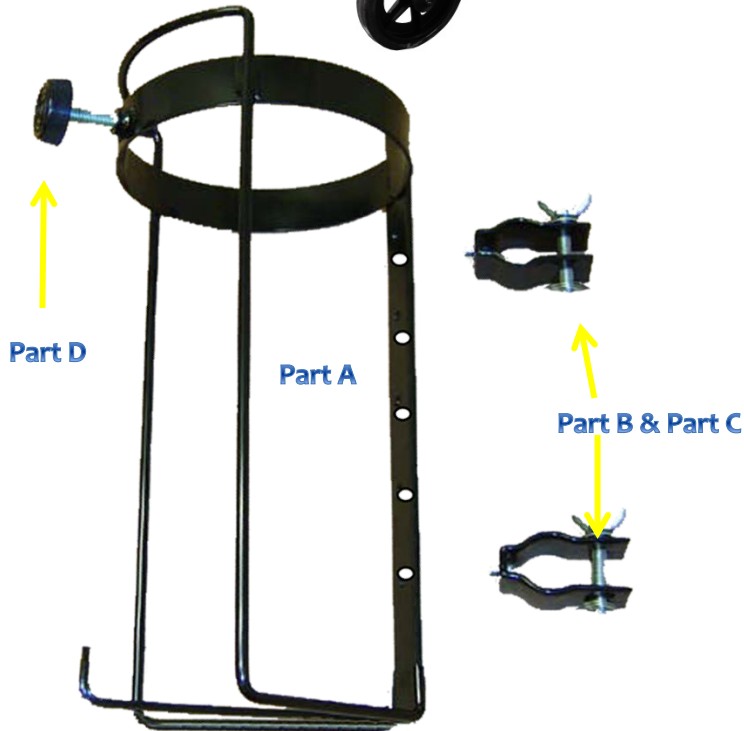

Tools Required: You will need a Phillips screwdriver to install this item.

Assembly Instructions

1. Take Mounting Brackets (Part B and Part C), and remove the wingnut/screw assembly (including the 2 washers).




2. Take the Tank Holder, and determine the location on the Rollator/Walker where you desire to install the Tank Holder. The Tank Holder Mounting Brackets will need to be installed on a vertical tube of Rollator/Walker.

Make sure to find a location where the O2 Tank will not tip over.

3. Based on the desired location, determine where the Mounting Brackets should be installed on the Rollator/Walker.

4. Based on where the Mounting Brackets should be installed on the Rollator/Walker, identify which two hole positions on the Tank Holder should be used to attach the two Mounting Brackets to Tank Holder.

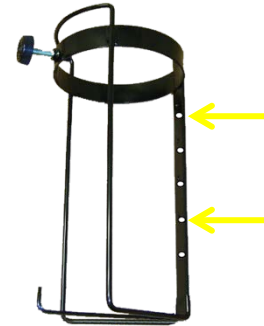

5. Attach Mounting Brackets to Tank Holder by tightening the short screws into desired hole positions.



Make sure to align the Mounting Brackets so they will position the Tank Holder vertically along vertical tube of Rollator/Walker.

6. Clamp the Mounting Brackets onto the vertical tube of the Rollator/Walker. Thread the long wingnut screw through one washer, then insert the long screw through both holes of a Mounting Bracket. Place the other washer onto the screw before tightening the wingnut. Repeat for the second Mounting Bracket.

7. Insert O2 Tank into Tank Holder, and tighten Screw Knob (Part D) to secure O2 Tank.





EAGLE HEALTH SUPPLIES INC.
www.eaglehealth.com

Instruction Pamphlet

For O2 Tank Carrier for Walkers/Rollators

53000

Have Questions? Please contact your authorized dealer for assistance.

Thank you for purchasing an Eagle Health Supplies product!

*If you would like to register your product for the 1-year warranty,
please fill out the included Warranty Card, and return to Eagle Health Supplies.*

Email: sales@eaglehealth.com

Phone: (714) 532 - 1777

Fax: (714) 532 - 9777