

Items 62231, 62221, 62321, 62331
Shower Chair



Fig. 1



Fig. 2



Fig. 3



Fig. 4

To assemble

1. Carefully remove the parts from the box. Do not use if any of the following parts are missing: (4) legs, (2) side frames, (1) backrest, and (1) seat. Models 62321 and 62331 will have a seat with a replaceable cut out instead of a regular seat like the one in the pictures.
2. Under the seat there should be two rails about 9" long. Slide them out of the plastic clamps.
3. Insert the rails into the horizontal tubing on one of the side frames until the push buttons pop into the holes. (Fig. 1)
4. Take the other side frame and insert it into the other side of the rails. (Fig. 2)
5. Take the backrest and insert it into the backrest tubes making sure that the push buttons pop into the holes
6. Take the seat and face the shower head holders in the corners of the seat to the front. (Fig. 3)
7. Put one side in first (Fig. 3) Then slowly push the other side down. Then push the whole seat down until it snaps into place. Make sure that the plastic clamps underneath the seat are firmly clamped around the crossbars. (Fig. 4)
8. Insert the legs making sure that the push buttons pop into the holes. Ensure that all legs are at the same level before use. Adjust the suction cup tips by rotating them so that they are in full contact with the ground.