

Parts Reference - Bariatric Shower Chair (Item #62621)



62621



Part A

Part B & Part C



Part D



Part E

Assembly Instructions

1. Take one leg frame (Part B or Part C, it does not matter which), and lay it flat on the ground.



2. Take the seat with the two pre-assembled tubes located under the seat (Part A), and insert the two tubes into the leg frame (Part B or C). Make sure the brass buttons snap firmly into the leg frame.




3. Take the other leg frame (Part B or Part C), and insert the seat tubes on the opposite side of the seat (Part A) into the leg frame. Make sure the brass buttons snap firmly into the leg frame.



4. Take the 4 legs (Part D), and insert them into the bottom of the leg frames. Set the chair height by using the same button-hole location for each leg. Make sure the brass buttons snap firmly into the leg frame. For your safety, adjust the rubber tips on the legs by turning them so they make maximum floor contact.



5. Take the backrest (Part E), and insert it into the two oblong backrest holes located on top of the seat towards the back side of the seat. Push the backrest down into the backrest holes, snapping it firmly into place.

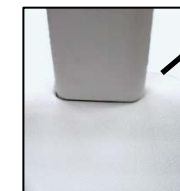

Tips for Care and Use

Rubber Tips

Be sure the rubber tips are turned outward for maximum floor contact and stability.

Cleaning

Rinse off the seat area with water after each use to prevent dirt, soap scum, and mineral deposits from forming which could block the push button areas.

If any push button does not completely snap into place, or if you feel the chair is not secure, please do not use the chair, and notify your authorized dealer immediately.

This seat has a 500lbs weight limitation.

Have Questions? Please contact your authorized dealer for assistance.

Thank you for purchasing an Eagle Health Supplies product!

If you would like to register your product for the 1-year warranty, please fill out the included Warranty Card, and return to Eagle Health Supplies.



EAGLE HEALTH SUPPLIES INC.
www.eaglehealth.com

Instruction Pamphlet

For Bariatric Shower Chair

62621

Email: sales@eaglehealth.com

Phone: (714) 532 - 1777

Fax: (714) 532 - 9777