
SNAP N SAVE

The XX Long Toilet to Tub Sliding Transfer Bench Item No. 67993 consists of a seat with attached sliding rings, left and right leg frame, a backrest, two sliding rails, an H-shaped leg support, and six legs. Weight capacity is 350 lbs.

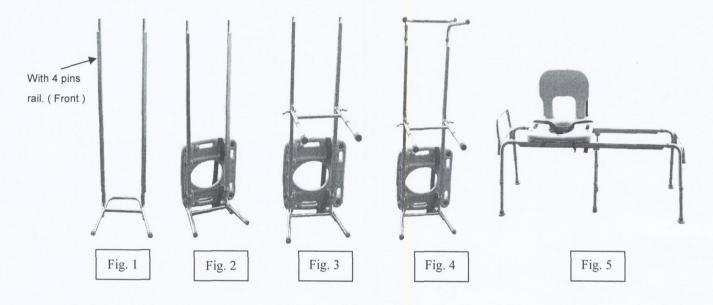

- 1. Take the side leg frame with the handlebar on it and put it on the ground (Fig. 1)
- 2. Determine which side of the transfer bench you want to be the front. The side with the handlebar will be the one inside your tub.
- 3. There are two sliding rails. One of them has 2 brass push buttons and one has 4. Insert the sliding rail with the 4 push buttons on the side you want be the front, making sure that the button snaps firmly into place. (Fig. 1)
- 4. Put the other sliding rail into the other side, making sure that the button snaps firmly into place.
- 5. Making sure that the seat side (front) with the red tabs on it is on the same rail as the 4 brass push buttons, insert both sliding rails into the sliding rings under the seat on each side. (Fig. 2)
- 6. Place the H-shaped leg support underneath the sliding rails at the midpoint, and loosen the mouting brackets by holding the locking handle in one hand and loosening the nut on the other side. (Fig. 3)
- 7. When the brackets are loose enough, slide them over the flat part of the sliding rails. Make sure that the brackets on the top part of the H-shaped support are over the flat part of the sliding rails.
- 8. Take the other side leg frame and fit it onto the sliding rails as you did with the other side in Step1. (Fig. 4)

- 9. Put the bench on the ground horizontally. Press the backrest into the two rectangular holes on the seat until they are securely into place. (Fig. 5)
- 10. Take it to location you would like to use it in. Make sure the handlebar side goes into the tub.
- 11. Insert each leg into the leg frame and set the hole location on all four legs according to the tub and toilet seat height. Adjust the rubber tips on each leg to fully contact the floor, by turning them.

 The bench should be level at this point.
- 12. Slide the H-shaped leg support on the rails so that it is in the middle between the toilet and tub. Then tighten the mounting brackets by holding the locking handle in one hand tightening the nut on the other side. Do not overtighten yet.
- 13. Adjust the leg heights on the H-shaped leg support so that the legs are barely off the ground, about 1/8th $-\frac{1}{4}$ of an inch. Make sure that the push buttons are all the way out of the hole and the legs are not touching the ground.
- 14. After you are finished, push down the locking handle on H- shaped leg support to further tighten the mounting brackets and the sliding rails.
- 15. Make sure that all push buttons are all the way out and the bench is secure before use.

To Use (This item requires someone to push the chair):

- 1. Slide the seat all the way to the end outside of the tub (the end opposite to the handle bar). You will hear a click which signifies that the lock has snapped into place. Then transfer the patient onto the seat. For added security, use the safety belt around the patient's lap. To release the locking mechanism, press down on the red button closest to that side and slide the patient with the seat all the way into the tub. When you slide the seat all the way to the other side, you will hear a click which signifies that the lock has snapped into place.
- 2. After completing the shower, unlock the locking mechanism, by pushing down on the red tab closest to that side and slide the patient with the seat out of the tub all the way to the opposite end. Sliding the patient all the way to the end will lock the seat into place again. Release the safety belt, then transfer the patient off the bench.

Maintenance:

To keep your sliding bench in optimal condition, we recommend to use WD-40, spray a little bit onto a damp cloth and wipe on the sliding rails. This will help keep the sliding smooth. Clean off any soap scum at least once a week and check to make sure all buttons are pop out and secure.

WARNING:

If any button does not pop out completely, or you feel the bench is not secure, do not use. Immediately contact the place of purchase.