

## Parts Reference - Shower Chairs




73131



72131

73231 (not shown)

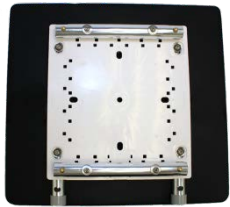

Part A



Part B & Part C



Part D



Part E

## Assembly Instructions

1. Take one leg frame (Part B or Part C, it does not matter which), and lay it flat on the ground.



2. Take the seat with the two pre-assembled tubes located under the seat (Part A), and insert the two tubes into the leg frame (Part B or C). Make sure the brass buttons snap firmly into the leg frame.



3. Take the other leg frame (Part B or Part C), and insert the seat tubes (Part A) into the leg frame. Make sure the brass buttons snap firmly into the leg frame.



4. Take the 4 legs (Part D), and insert them into the bottom of the leg frames. Set the chair height by using the same button-hole location for each leg. Make sure the brass buttons snap firmly into the leg frame. For your safety, adjust the rubber tips on the legs by turning them so they make maximum floor contact.



5. Take the backrest (Part E), and press the two brass push buttons on the frame of the backrest. Insert the backrest into the tubes underneath the seat, and push it in until the brass push buttons snap firmly into place.



## Tips for Care and Use

### Rubber Tips

Be sure the rubber tips are turned outward for maximum floor contact and stability.

### Cleaning

Rinse off the seat area with water after each use to prevent dirt, soap scum, and mineral deposits from forming which could block the push button areas.

If any push button does not completely snap into place, or if you feel the chair is not secure, please do not use the chair, and notify your authorized dealer immediately.

*Have Questions? Please contact your authorized dealer for assistance.*

Thank you for purchasing an Eagle Health Supplies product!

*If you would like to register your product for the 1-year warranty, please fill out the included Warranty Card, and return to Eagle Health Supplies.*



**EAGLE HEALTH SUPPLIES INC.**  
[www.eaglehealth.com](http://www.eaglehealth.com)

## Instruction Pamphlet

**For Padded Shower Chair - 72131**

**For Padded Hip Chair – 73131**

**For Hip Chair - 73231**

Email: [sales@eaglehealth.com](mailto:sales@eaglehealth.com)

Phone: (714) 532 – 1777

Fax: (714) 532 – 9777