

Swivel Shower Chairs - Parts Reference

75232



75233



Part A



Part B & Part C



Part D



Part E

Assembly Instructions


1. Take seat with swivel plate (Part A), flip it over with the bottom side facing up, and lay it flat on the ground.



2. Take one leg frame (Part B or Part C, it does not matter which), and insert the leg frame into the two parallel tubes underneath the swivel plate. Make sure the brass buttons snap firmly into the leg frame.



3. Take the other leg frame (Part B or Part C), and insert it into the opposite side of the parallel tubes underneath the swivel plate. Make sure the brass buttons snap firmly into the leg frame.

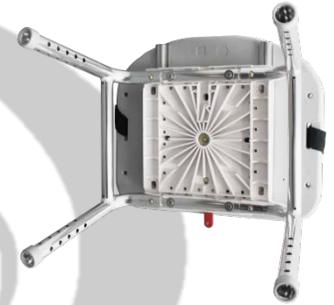

4. Take the 4 legs (Part D), and insert them into the leg frames. Set the chair height by using the same button-hole location for each leg. Make sure the buttons snap firmly into place. For your safety, adjust the rubber tips on the legs by turning them outward for maximum floor contact.



5. Sit the chair up on the legs. Take the backrest (Part E), and press the two brass push buttons on the frame of the backrest. Insert the backrest into the tubes underneath the seat, and push it in until the brass push buttons snap firmly into place.



Tips for Care and Use

Safety Belt

For added security and stability, adjust the safety belt around the user's lap.

Swivel Seat

Press down on the red tab located under the center of the seat's front edge to turn the swivel seat, and rotate the seat slightly. Once the seat starts to swivel, release the red tab. The swivel seat locks every 90 degrees.

Rubber Tips

Be sure the rubber tips are turned outward for maximum floor contact and stability.

Cleaning

Rinse off the seat area with water after each use to prevent dirt, soap scum, and mineral deposits from forming which could block the push button areas.

If any push button does not completely snap into place, or if you feel the chair is not secure, please do not use the chair, and notify your authorized dealer immediately.

Optional Armrests

Please see your authorized dealer to purchase Armrests – Item #75003 (sold individually).

Have Questions? Please contact your authorized dealer for assistance.

Thank you for purchasing an Eagle Health Supplies product!

If you would like to register your product for the 1-year warranty, please fill out the included Warranty Card, and return to Eagle Health Supplies.



EAGLE HEALTH SUPPLIES INC.
www.eaglehealth.com

Instruction Pamphlet

For Swivel Shower Chairs

**75232
75233**

Email: sales@eaglehealth.com
Phone: (714) 532 – 1777
Fax: (714) 532 – 9777