

BIOMODEX[®] NEUROVASCULAR



UPLOAD INSTRUCTIONS FOR BIOMODEX MEDICAL IMAGE CLOUD


Please follow this quick guide to upload images to Biomodex's medical image management platform for neurovascular patient-specific or custom cartridges.

Upload your case

- Gather the required images using the document TRN-008_EVIAS_Image_Requirements
- Navigate to the following upload address:

biomodex.ambrahealth.com/share/biomodex

This link should lead you to a web page that looks like the one shown in the image below. If it looks different, please retype the address or copy and paste the URL above into the address bar.

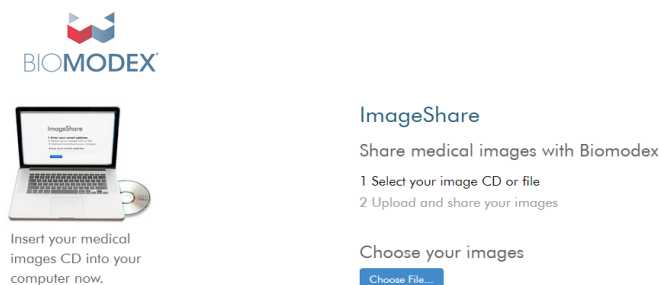

Select your case

- Click the blue button that says "Choose File..."
- Select the folder which contains the DICOM series of interest (1 upload per case/patient)
- The series will display some information similar to the image shown here:

	Description	Images	Modality	Study Date
<input checked="" type="checkbox"/>	Vascular^02_CTA_HEAD_JUN2020 (Adult)	284	CT	10-28-2020

Provide case information

- Complete the following fields

* Uploader's Email

Process Express?

Do you need the model in the next 5 days?

* Product Family

R&D is for Biomodex Internal only.

Upload Purpose

- Uploader's Email: professional email address
 - Product Family: Neurovascular
 - Upload Purpose: New Product (online order placed)
 - Process Express: Select «yes» if model is needed in 5 business days or less, this may incur extra charges.
- The image upload must be done after placing the order on Biomodex online order platform.

BIOMODEX[®] NEUROVASCULAR

Order #

Type

- AVM
 Aneurysm
 Carotid
 Ischemic
 MMA

Any Aneurysms?

- No
 Yes

How Many?

Location(s):

Injected from which side?

Cartridge Name Identifier

Please enter the patient's initials.

Hospital Name

Your Company Name

Special Instructions or Comments?

Message:

- Order#: This number is the one provided to you after the order was completed on the Biomodex online platform
- Please complete as many fields as possible
- Message: include any specific requests associated with the order.

Anonymize your series and upload your case

- ✖ When the file contains non-DICOM compliant images, e.g. radiation dose report, select the checkbox «Pixel De-Identifier»
- ✖ Click on the upload button to send your case
- ✖ Anonymization will occur automatically (any PHI will be removed)
- ✖ Uploaded case will be stored on Ambra's secure cloud servers for approval

* I understand this study will be shared with Biomodex, I have authority to share this study.

Use DICOM pixel de-identifier before upload

Upload Selected Studies

Preview Selected Studies

Cancel