

INSIDE 247LABKIT

CHECK YOUR KIT FOR THE FOLLOWING



URINE CUP



TRANSPORT TUBE



PIPETTE



LAB
PAK/RETURN
LABEL



REQUISITION FORM



ALERT STICKER



SPECIMEN BAG

RESULTS

Results will be sent to your personal email within 48 - 72 hours after your sample has been tested.

If you receive positive results contact 247Labkit by email at private@247labox.com or by phone at 855.895.4716. We will assist in directing you to a nearby clinic which can prescribe appropriate treatment. Also, consult with your primary care physician for additional information.

For all questions or concerns including understanding your results contact 247Labkit by email at private@247labox.com or by phone at 855.895.4716.

INSTRUCTIONS

1. Fill the cup with your urine sample.




2. Use pipette to transfer urine from cup to transport tube. Fill the transport tube between the two black lines. Complete the alert sticker and place on the transport tube.



3. Ensure transport tube with urine sample is secure. Place the transport tube in the specimen bag.



4. Complete the requisition form. Ensure all information is correct. Place the requisition form inside the specimen bag.

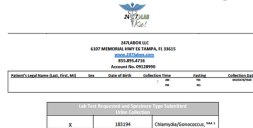

5. Place your specimen bag with your sample & requisition form inside the 247Labkit. Place your 247Labkit inside the UPS Laboratory Pak.



A free return shipping label comes with your 247labkit. Place your return shipping label on the UPS Laboratory Pak. Return the UPS Laboratory Pak to your nearest UPS location or drop box.

TRACK YOUR SAMPLE

Tracking your sample will help you understand when to expect your results. The time frame for receiving your results depends largely on your return shipping label (Return Sample Section).

Your kit comes complete with a free economy return shipping label. Shipping can sometimes be unpredictable which may delay when our lab receives your sample and when you receive your results.

To track your sample, locate your tracking # on your return shipping label. Keep a copy of your tracking # for your records.



Visit <https://www.ups.com/track>
Enter your tracking # and select track.



Tracking

