



VISTA | APEX

PATENTED


CHX-Plus™

2% Chlorhexidine Gluconate Solution

enhanced with surface modifiers and wetting agents



REF 503900

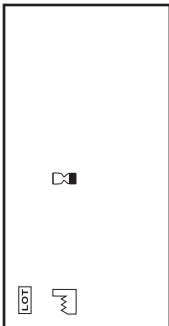

Description:

CHX-Plus™ is an enhanced 2% Chlorhexidine Gluconate solution with powerful wetting agents and proprietary surface modifiers to improve depth of penetration. Used for the final irrigation step for long lasting cleansing.

Instructions for Use:

1. Select desired syringe and irrigation tip.
2. Fill the syringe with the desired amount of CHX-Plus™.
3. Attach irrigation tip securely to the syringe.
4. Irrigate as desired.
5. Remove and dispose of syringe and irrigating tip.

Warnings:

**** NOT TO BE USED AS AN ORAL RINSE ****

Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com

EC REP **Emergo Europe**
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Fax: +31 (0)70 346 7299
Tel: +31 (0)70 345 8570

503900-V-ENG (11)
Made in USA

16 oz (480mL) VOLUME



Rx ONLY

WARNING

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.