## essence SLOW SPEED CONTRA ANGLES MAINTENANCE & CLEANING MANUAL

26-PBL214, 26-PBFG215, 26-SI216, 26-SO217, 26-PBL218

### SPECIAL PRECAUTIONS:

1. Always exercise extreme caution when operating your Attachment and Head
2. Always inspect your Attachment and Head thoroughly before each use.
3. Always use PPE (Proper Protective Equipment) when operating and cleaning your Slowspeed.
4. See your Occupational Health and Safety Act or other safety ordinances for specific instructions.
5. This is a precision instrument. Do not drop or apply excessive force when handling.
6. If product is dropped your dealer should be contacted, it is recommended that you do not use.
7. Contact Curionright away if any problem should occur with any of your products.
8. Sterilize before first time use and before every subsequent use thereafter.
9. This product is for use by a Dentist or trained professional in the dental field.

### HOW TO USE:

1. Attachment of E-Type Contra Angles. Slide Attachment onto shaft of Air Motor until locked in place.
2. To remove the Contra Angle, pull attachment, or push the release button located on the exterior of Air Motor.

### CYCLES:

The Contra Angle can be subject to 250 cycles = (Performed Sterilization Process) per EN ISO 14457 requirement 5.10 before showing signs of deterioration.

### FEATURES & SPECIFICATIONS:

1. Contra Angles conform to ISO 14457 specifications.
2. Connection with Attachments: meets ISO-9168 standard for E-Type Connections
3. Method of Sterilization: Autoclave

### CLEANING: Preparation and Cleaning before Sterilization:

*\*Clean and lubricate the handpiece as soon as possible after treatment. Failure to properly maintain the handpiece may cause infection, overheating leading to burn injuries.*

**⚠ DO NOT IMMERSE INSTRUMENTS IN DISINFECTANT SOLUTION OR CLEAN IN ULTRASONIC DEVICES. MANUAL CLEANING ONLY.**

#### 1. PREPARATION:

- a) Always wear protective gloves, a mask, and protective goggles for safety purposes and to minimize the risk of infection.
- b) Remove the bur. (if applicable)
- c) Wipe the exterior of the handpiece using a cleaning wipe.
- d) Operate the handpiece at the chair-side for at least 20 seconds to purge fluids in the handpiece.
- e) Remove the handpiece, carry the handpiece to the decontamination area and proceed to step 2.

#### 2. CLEANING AND DRYING:

*\*Wash the handpiece and irrigation nozzle under running water following the procedures below. (Water condition: <38°C (100°F), >3.5L/min; water should be same quality as drinking water)*

- a) Clean the external surfaces of the handpiece/accessory with a soft bristled toothbrush for 15 seconds or more.
- b) Clean around the bur insertion hole for 15 seconds or more, using an interdental brush (wire thickness 0.7mm) that conforms to size 4 in ISO 16409. (if applicable)
- c) Under appropriate lighting (natural office lighting), inspect the handpiece/accessory for blood or other debris. If any visible debris remain, repeat the process until it is visually clean.
- d) Wipe the exterior of the handpiece/accessory clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (ie. Isopropyl)

### LUBRICATION:

USE ONLY AUTOCLAVABLE HANDPIECE LUBRICANT.

1. Remove contra angle from motor (leave motor connected to the airline).
2. Using spray lubrication can or lubrication station, apply lubricant into contra angle.
3. Reattach entire contra angle to air motor if not using a lubrication station.
4. Run entire tool at normal speed for 10 seconds.
5. Wipe off excess oil from entire tool surface.

### AUTOCLAVING PROCEDURE:

Sterilize in accordance with ISO17665-1:2006. Suitable autoclaving packaging is recommended to preserve item in sterile condition. Once the product has been sterilized it should be used as at earliest convenience to ensure sterilization is maintained.

1. Clean before sterilization (See Cleaning).
2. Lubricate before sterilization (see LUBRICATION above).
3. Set parameters for Moist Heat Sterilizer. PreVac cycle 134 degrees Celsius for 8 minutes and 20 minute dry time .
4. Use Sterilized head as soon as possible to ensure sterilization is maintained.

### WARRANTY INFORMATION:

essence Slow Speed Contra Angles are warranted to be free of manufacturing defects for a period of 12 months\* from date of purchase.

*\*Warranty is conditional on proper use and maintenance of the Contra Angles. Warranty is limited to repair or replacement at manufacturers' discretion. Under no circumstances will the manufacturer be held responsible for incremental or consequential damages. Please contact for warranty repair.*

### Disclaimer:

Curion recommends that the sterilization process outlined in the Directions for use (DFU) described in this document be followed precisely. Curion is not responsible for any products supplied in which the end user does not follow the DFU.

## Bisco Dental Products (Canada) Inc. Db a Curion Dental Products

2571 Smith Street Richmond, BC., V6X 2J1 Canada  
See Curion Website for other products and product information  
**curion.ca or Phone 1.800.667.8811**

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