

INSTRUMENT SELECTION DECISION TREE

LM SharpDiamond™
(Sharpen-Free)



*Created by Kathleen Bokrossy, RDH,
President of rdhu

CLIENT

Periodontitis

Periodontal Health & Gingivitis

Stage 1

Stage 2

Stage 3

Stage 4

Universal Curette

Universal Curette Plus

DIAGNOSIS

- Diagnostic Kit**
- Mirror
 - Flexplorer 11F-12F ***
 - North Carolina Probe ***

- Perio Diagnostic Kit**
- Mirror
 - Flexplorer 11F-12F ***
 - North Carolina Probe ***
 - Furcation Probe ***

- Diagnostic Kit**
- Mirror
 - Flexplorer 11F-12F ***
 - North Carolina Probe ***

TREATMENT

- Option A***
- Servo Cassette 5E (Blue)
 - H6/H7 Sickle Scaler
 - 204S Sickle Scaler
 - Columbia 4L/4R Curette
 - Anterior Dual Gracey (Syntette) Curette*
 - Dual Gracey (Syntette) Curette*

- Stage 2 -4***
- Servo Cassette 5E (Blue)
 - Stainless Steel Furcator KS ***
 - Gracey 15/16 Curette*
 - Gracey 17/18 Curette*

- Option A**
- Servo Cassette 5E (Blue)
 - SharpJack Sickle Scaler
 - 204S Sickle Scaler
 - Columbia 4L/4R Curette

- Option A***
- Servo Cassette 5E (Blue)
 - SharpJack Sickle Scaler
 - 204S Sickle Scaler
 - Columbia 4L/4R Curette
 - Anterior Dual Gracey (Syntette) Curette*
 - Dual Gracey (Syntette) Curette*

- Option B***
- Servo Cassette 8E (Blue)
 - H6/H7 Sickle Scaler
 - M23 Sickle Scaler
 - Columbia 4L/4R Curette
 - Gracey 1/2 Curette*
 - Gracey 11/12 Curette*
 - Gracey 13/14 Curette*

- Option B**
- Servo Cassette 5E (Blue)
 - H6/H7 Sickle Scaler
 - 204S Sickle Scaler
 - Columbia 4L/4R Curette

- Option B***
- Servo Cassette 8E (Blue)
 - H6/H7 Sickle Scaler
 - 204S Sickle Scaler
 - Columbia 4L/4R Curette
 - Gracey 1/2 Curette*
 - Gracey 11/12 Curette*
 - Gracey 13/14 Curette*

*Available in Standard or Mini

***Currently only available in DuraGradeMax stainless steel

*Available in Standard or Mini

DECISION TREE KITS

LM feel the difference

curion
dentistry, curated.

EN: 1.800.667.8811
FR: 1.800.211.1200
curion.ca

Sometimes, less is more! Effectively access all areas with the least number of instruments – for each stage of periodontal involvement.







Decision Tree Kits offer simplified tray set-ups that maximize procedural efficiency and effectiveness. Each instrument kit in this series contains LM's winning combination of advanced sharpen-free tips* and the industry's highest-rated ergonomic handles.

*Exceptions include Diagnostic Instrument Kits and Furcator K/S

FEATURES AND BENEFITS:

- 100% sharpen-free tips (where available)
- Unparalleled edge retention & superior sharpness
- Thorough calculus removal with less hand strain
- More comfortable scaling for both patient and clinician
- Improved productivity & clinical results
- Excellent tactile sensitivity
- Exceptional control even in wet environments
- Reduced waste of sterilization pouches and chemical indicators*

*IPAC: Clinicians may no longer sharpen chairside. Must sharpen sterile instruments and then re-sterilize prior to use.

SUGGESTED # OF KITS	KIT TYPE	KIT CONTENTS	SKU #	KIT CONTENTS	SKU #
8	 <p>LM-DIAGNOSTIC KIT 01-LMKIT-DIAG-SS</p>	<ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · Mirror Handle 28 - Grey · North Carolina Probe (Stainless Steel)*** · Flexplorer (Stainless Steel)*** 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM28GREYES · 01-LM54BES · 01-LM11F-12FES 		
6 - 8	 <p>Sharpen-Free DECISION TREE KIT - UNIVERSAL CURETTE 01-LMDTSD-UNIV-OPA-CASSETTE 01-LMDTSD-UNIV-OPB-CASSETTE</p>	<p>Option A</p> <ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · SharpJack Sickle Scaler · 204S Sickle Scaler · Columbia 4L/4R Curette 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM315-316SDES · 01-LM303-304SDES · 01-LM219-220SDES 	<p>Option B</p> <ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · H6/H7 Sickle Scaler · 204S Sickle Scaler · Columbia 4L/4R Curette 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM146-147SDES · 01-LM303-304SDES · 01-LM219-220SDES
	 <p>Sharpen-Free DECISION TREE KIT - UNIVERSAL CURETTE PLUS 01-LMDTSD-UNIVPLUS-OPA-STD-CASSETTE 01-LMDTSD-UNIVPLUS-OPB-STD-CASSETTE</p>	<p>Option A*</p> <ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · SharpJack Sickle Scaler · 204S Sickle Scaler · Columbia 4L/4R Curette · Anterior Dual Gracey (Syntette) Curette · Dual Gracey (Syntette) Curette 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM315-316SDES · 01-LM303-304SDES · 01-LM219-220SDES · 01-LM275-276SDES · 01-LM215-216SDES 	<p>Option B*</p> <ul style="list-style-type: none"> · Servo Cassette 8E (Blue) · H6/H7 Sickle Scaler · 204S Sickle Scaler · Columbia 4L/4R Curette · Gracey 1/2 Curette · Gracey 11/12 Curette · Gracey 13/14 Curette 	<ul style="list-style-type: none"> · 01-LM6582 · 01-LM146-147SDES · 01-LM303-304SDES · 01-LM219-220SDES · 01-LM201-202SDES · 01-LM211-212SDES · 01-LM213-214SDES
3	 <p>LM-PERIO DIAGNOSTIC KIT 01-LMKIT-DIAG-PERIO-SS</p>	<ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · Mirror Handle 28 - Grey · North Carolina Probe (Stainless Steel)*** · Flexplorer (Stainless Steel)*** · Furcation Probe (Stainless Steel)*** 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM28GREYES · 01-LM54BES · 01-LM11F-12FES · 01-LM20B-21BES 		
4	 <p>Sharpen-Free DECISION TREE KIT - STAGE 1 01-LMDTSD-ST1-OPA-STD-CASSETTE 01-LMDTSD-ST1-OPB-STD-CASSETTE</p>	<p>Option A*</p> <ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · H6/H7 Sickle Scaler · 204S Sickle Scaler · Columbia 4L/4R Curette · Anterior Dual Gracey (Syntette) Curette · Dual Gracey (Syntette) Curette 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM146-147SDES · 01-LM303-304SDES · 01-LM219-220SDES · 01-LM275-276SDES · 01-LM215-216SDES 	<p>Option B*</p> <ul style="list-style-type: none"> · Servo Cassette 8E (Blue) · H6/H7 Sickle Scaler · M23 Sickle Scaler · Columbia 4L/4R Curette · Gracey 1/2 Curette · Gracey 11/12 Curette · Gracey 13/14 Curette 	<ul style="list-style-type: none"> · 01-LM6582 · 01-LM146-147SDES · 01-LM313-314SDES · 01-LM219-220SDES · 01-LM201-202SDES · 01-LM211-212SDES · 01-LM213-214SDES
3	 <p>Sharpen-Free DECISION TREE KIT - STAGE 2-4* 01-LMDTSD-ST2-4-STD-CASSETTE</p>	<ul style="list-style-type: none"> · Servo Cassette 5E (Blue) · Furcator KS (Stainless Steel)*** · Gracey 15/16 Curette · Gracey 17/18 Curette 	<ul style="list-style-type: none"> · 01-LM6552 · 01-LM259-260ES · 01-LM245-246SDES · 01-LM247-248SDES 		

* Available in Standard or Mini ***Currently only available in DuraGradeMax stainless steel