

INSTRUMENT SELECTION DECISION TREE

LM DuraGradeMax™
(Stainless Steel)



*Created by Kathleen Bokrossy, RDH,
President of rdhu

CLIENT

Periodontitis

Periodontal Health & Gingivitis

Stage 1

Stage 2

Stage 3

Stage 4

Universal Curette

Universal Curette Plus

DIAGNOSIS

Diagnostic Kit
· Mirror
· Flexplorer 11F-12F
· North Carolina Probe

Perio Diagnostic Kit
· Mirror
· Flexplorer 11F-12F
· North Carolina Probe
· Furcation Probe

Diagnostic Kit
· Mirror
· Flexplorer 11F-12F
· North Carolina Probe

TREATMENT

Option A*
· Servo Cassette 5E (Blue)
· H6/H7 Sickle Scaler
· 204S Sickle Scaler
· Columbia 4L/4R Curette
· Gracey 1/2 Curette*
· Dual Gracey (Syntette) Curette*

Stage 2 -4*
· Servo Cassette 5E (Blue)
· Furcator KS
· Gracey 15/16 Curette*
· Gracey 17/18 Curette*

Option A
· Servo Cassette 5E (Blue)
· H6/H7 Sickle Scaler
· 204S Sickle Scaler
· Columbia 4L/4R Curette

Option A*
· Servo Cassette 5E (Blue)
· H6/H7 Sickle Scaler
· 204S Sickle Scaler
· Columbia 4L/4R Curette
· Gracey 1/2 Curette*
· Dual Gracey (Syntette) Curette*

Option B*
· Servo Cassette 8E (Blue)
· H6/H7 Sickle Scaler
· M23 Sickle Scaler
· Columbia 4L/4R Curette
· Gracey 1/2 Curette*
· Gracey 11/12 Curette*
· Gracey 13/14 Curette*

Option B*
· Servo Cassette 8E (Blue)
· H6/H7 Sickle Scaler
· 204S Sickle Scaler
· Columbia 4L/4R Curette
· Gracey 1/2 Curette*
· Gracey 11/12 Curette*
· Gracey 13/14 Curette*

*Available in Standard or Mini

*Available in Standard or Mini

*Available in Standard or Mini

DECISION TREE KITS

LM feel the difference

curion
dentistry, curated.







EN: 1.800.667.8811
FR: 1.800.211.1200
curion.ca

Sometimes, less is more! Effectively access all areas with the least number of instruments – for each stage of periodontal involvement.

Decision Tree Kits are focused tray set-ups designed to maximize procedural efficiency and effectiveness. Each instrument in this series combines LM's superior metallurgy and the industry's highest-rated ergonomic handles. Choose your best combination of carefully curated Decision Tree Kits to meet the specific needs of your patients.

FEATURES AND BENEFITS:

- Sharpen less frequently with DuraGradeMax stainless steel tip technology
- Superior edge retention & sharpness
- More comfortable scaling for both patient and clinician
- Improved productivity & clinical results
- Excellent tactile sensitivity
- Exceptional control even in wet environments

SUGGESTED # OF KITS	KIT TYPE	KIT CONTENTS	SKU #	KIT CONTENTS	SKU #
8	 <p>LM-DIAGNOSTIC KIT 01-LMKIT-DIAG-SS</p>	<ul style="list-style-type: none"> Servo Cassette 5E (Blue) Mirror Handle 28 - Grey North Carolina Probe (Stainless Steel)*** Flexplorer (Stainless Steel)*** 	<ul style="list-style-type: none"> 01-LM6552 01-LM28GREYES 01-LM54BES 01-LM11F-12FES 		
6 - 8	 <p>Stainless Steel DECISION TREE KIT - UNIVERSAL CURETTE 01-LMDTSS-UNIV-CASSETTE</p>	<p>Option A</p> <ul style="list-style-type: none"> Servo Cassette 5E (Blue) H6/H7 Sickle Scaler 204S Sickle Scaler Columbia 4L/4R Curette 	<ul style="list-style-type: none"> 01-LM6552 01-LM146-147ES 01-LM303-304ES 01-LM219-220ES 		
	 <p>Stainless Steel DECISION TREE KIT - UNIVERSAL CURETTE PLUS 01-LMDTSS-UNIVPLUS-OPA-STD-CASSETTE 01-LMDTSS-UNIVPLUS-OPB-STD-CASSETTE</p>	<p>Option A*</p> <ul style="list-style-type: none"> Servo Cassette 5E (Blue) H6/H7 Sickle Scaler 204S Sickle Scaler Columbia 4L/4R Curette Gracey 1/2 Curette Dual Gracey (Syntette) Curette 	<ul style="list-style-type: none"> 01-LM6552 01-LM146-147ES 01-LM303-304ES 01-LM219-220ES 01-LM201-202ES 01-LM215-216ES 	<p>Option B*</p> <ul style="list-style-type: none"> Servo Cassette 8E (Blue) H6/H7 Sickle Scaler 204S Sickle Scaler Columbia 4L/4R Curette Gracey 1/2 Curette Gracey 11/12 Curette Gracey 13/14 Curette 	<ul style="list-style-type: none"> 01-LM6582 01-LM146-147ES 01-LM303-304ES 01-LM219-220ES 01-LM201-202ES 01-LM211-212ES 01-LM213-214ES
3	 <p>LM-PERIO DIAGNOSTIC KIT 01-LMKIT-DIAG-PERIO-SS</p>	<ul style="list-style-type: none"> Servo Cassette 5E (Blue) Mirror Handle 28 - Grey North Carolina Probe (Stainless Steel)*** Flexplorer (Stainless Steel)*** Furcation Probe (Stainless Steel)*** 	<ul style="list-style-type: none"> 01-LM6552 01-LM28GREYES 01-LM54BES 01-LM11F-12FES 01-LM20B-21BES 		
4	 <p>Stainless Steel DECISION TREE KIT - STAGE 1 01-LMDTSS-ST1-OPA-STD-CASSETTE 01-LMDTSS-ST1-OPB-STD-CASSETTE</p>	<p>Option A*</p> <ul style="list-style-type: none"> Servo Cassette 5E (Blue) H6/H7 Sickle Scaler 204S Sickle Scaler Columbia 4L/4R Curette Gracey 1/2 Curette Dual Gracey (Syntette) Curette 	<ul style="list-style-type: none"> 01-LM6552 01-LM146-147ES 01-LM303-304ES 01-LM219-220ES 01-LM201-202ES 01-LM215-216ES 	<p>Option B*</p> <ul style="list-style-type: none"> Servo Cassette 8E (Blue) H6/H7 Sickle Scaler M23 Sickle Scaler Columbia 4L/4R Curette Gracey 1/2 Curette Gracey 11/12 Curette Gracey 13/14 Curette 	<ul style="list-style-type: none"> 01-LM6582 01-LM146-147ES 01-LM313-314ES 01-LM219-220ES 01-LM201-202ES 01-LM211-212ES 01-LM213-214ES
3	 <p>Stainless Steel DECISION TREE KIT - STAGE 2-4* 01-LMDTSS-ST2-4-STD-CASSETTE</p>	<ul style="list-style-type: none"> Servo Cassette 5E (Blue) Furcator KS (Stainless Steel)*** Gracey 15/16 Curette Gracey 17/18 Curette 	<ul style="list-style-type: none"> 01-LM6552 01-LM259-260ES 01-LM245-246ES 01-LM247-248ES 		

* Available in Standard or Mini

***Currently only available in DuraGradeMax stainless steel