

# EndoHandle

EndoHandle filing is becoming the standard way to begin root canal preparation.

The EndoHandle gives you easy access and visibility to find and file the natural canal. With a gentle pecking motion you can tease the file down the canal and start circumferential filing. With this up/down filing motion the irrigation solution is agitated, pulp debridement starts, rough canal walls become smooth and pre-flared, and the orifice is enlarged. The EndoHandle prepares the perfect Glide Path for safe rotary filing, reducing the stress and strain that cause file separation.

---

## How to file with The EndoHandle

Circumferential Linear Filing is a straight-line, up-down filing motion done sequentially around the periphery of the canal. The file is placed in the canal and withdrawn in a directional manner against each side of the canal wall. No rotation, just safe straight-line filing to clean and pre-flare the walls of the canal. The leverage of filing with the EndoHandle is at least five times more efficient than traditional two-finger filing.

---

## Fits any standard file

The Universal EndoHandle is designed to fit any standard endodontic file. This allows you to quickly change to any file type and size you may need. First place a bend in the shaft of the file. Next hold the small cap, and simply twist the handle to open the unique chamber, put your file in the chamber and twist to tighten.

---

Packaged in variable quantity amounts.

- Color Coded to ISO Specifications
- For intended use only by dental professionals.

Instruments must be cleaned and sterilized before each use.

**Autoclave Sterilization. Neutral solutions only.**

Venta Endo  
150 E 200 N Ste. G  
Logan UT, 84321