

Adjust and Polish Zirconia and Lithium Disilicate the eZr Way!

eZr Intra-oral Adjustment:



- **No chatter, chipping or microfracture** - Unlike diamonds bonded directly to a hard metal shaft, eZr's rubber substrate cushions the diamonds to prevent damage to the restoration.
- **No heat build-up** - The carefully engineered substrate does not transfer heat during use.
- **FAST!** - eZr's coarse grit blue grinders make your adjustments fast and convenient.

eZr Intra-oral Polishing:



- **Unparalleled smooth polish** - Don't take our word for it! - "The eZr system is the only one I've tried that polishes zirconia to smooth!" Todd Brower, DDS, Holland, Michigan.
- **Increased physical stability** - Highly polished zirconia and lithium disilicate is stable and non-abrasive to opposing dentition.
- **No paste required** - Forget the mess and expense of polishing pastes. eZr is all you need.




Making adjustments eZr™ for you!

eZr™ is a three-step diamond polishing system for the processing of zirconium oxide and lithium disilicate. The grinders and polishers are ideal for adjusting and correcting chairside restorations such as inlays, onlays, veneers and crowns.

FPZK01 - eZr™ Kit



1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.

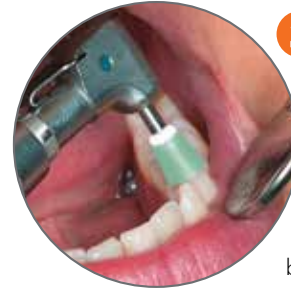

2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.

Order #	Description	Use	Price
FPZK01	Complete system kit, 2 grinders and 6 polishers	Gross removal, Smoothing, Pre-Polishing, & High Gloss Polishing	
FPZC030	Blue coarse grit disc gross removal, 5/pack	Gross removal	
FPZC010	Blue coarse grit flat end taper gross removal, 5/pack	Gross removal	
FPZM010	Green medium grit small point polisher, 5/pack	Smoothing and Pre-Polishing	
FPZM020	Green medium grit large point polisher, 5/pack	Smoothing and Pre-Polishing	
FPZM030	Green medium grit disc polisher, 5/pack	Smoothing and Pre-Polishing	

Order #	Description	Use	Price
FPZM040	Green medium grit cup polisher, 5/pack	Smoothing and Pre-Polishing	
FPZF010	Orange fine grit small point polisher, 5/pack	High Gloss Polishing	
FPZF020	Orange fine grit large point polisher, 5/pack	High Gloss Polishing	
FPZF030	Orange fine grit disc polisher, 5/pack	High Gloss Polishing	
FPZF040	Orange fine grit cup polisher, 5/pack	High Gloss Polishing	

*Canadian MSRP

Be sure to check out the website at www.garrisondental.com for info on other innovative Garrison products, demo videos and our 6-month money-back guarantee.

Distributed by Curion

curion.ca | EN: 1.800.667.8811 | FR: 1.800.211.1200

