



LMSharpDiamond

CARE RECOMMENDATIONS



Thank you for your recent LM Sharp Diamond purchase. To maximize the longevity of your Sharp Diamond instruments, please note the following:

CARE

- Use of LM ServoE cassettes throughout the maintenance cycle to protect tips as well as users from accidental pricking injuries is recommended.

CLEANING

- Safe for cleaning in washers and ultrasonic cleaners.
- ✗ **Do Not** use metal brushes to remove debris.

STERILIZATION

- Sterilize as you do your stainless steel scalers (should not be used with disinfectants containing amines, phenol or chlorine).
- Set autoclave sterilization temperature to 134 °C/273 °F.
If using hot air sterilization, the maximum temperature setting is 180 °C/355 °F.
- Can be sterilized alongside any other metal instruments.

USAGE

- Recommended for use after ultrasonic scaling on light to moderate deposits.
- Designed to reduce hand fatigue - users can maximize the ergonomic benefits of shaving off calculus with a lighter touch.
- Fracturing calculus is not necessary nor recommended.
- ✗ **Do Not** use to trim margins or remove overhangs.
- ✗ **Do Not Sharpen.** The sharpen-free capability is a result of combining a proprietary stronger base alloy and a diamond-like micromembrane coating using the most advanced deposition technology.
Sharpening will weaken the tips.

Please contact us if you have further questions.

BISCO DENTAL PRODUCTS CANADA INC.

2571 Smith Street, Richmond, BC, V6X 2J1 CANADA

1.800.667.8811 www.biscocanada.com info@biscocanada.com