

Quick Protection Where It's Needed.



DMG MiniDam

A Simple Way to Safeguard Adjacent Tissue and Dentin


Protecting the gingiva and maintaining a dry field can mean the difference between a successful procedure and a costly do-over. MiniDam from DMG America is a new, latex-free gingival protection device that goes on quickly and holds on securely. Once placed, it protects the proximal area during treatment, and provides comfortable gingival coverage for patients. The unique shape of the MiniDam allows it to self-

stabilize around adjacent teeth, where it securely maintains its position without the use of clamps. The clinician simply pre-stretches the silicone material, which already has holes integrated into it and then pulls it over the teeth to be treated. This provides a relatively dry working area, unlimited access and an unobstructed view of the working area. *Dental Milestones Guaranteed.*


Quick, easy and comfortable
for the patient.




Easy as 1-2-3!

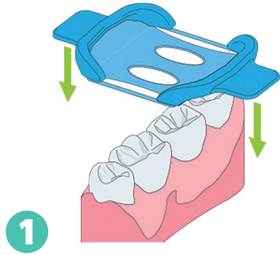

1 Stretch the DMG MiniDam and pull towards the gingiva.

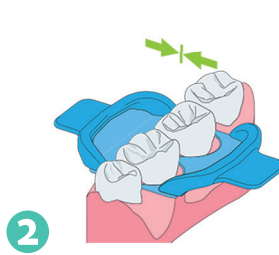

2 Release the DMG MiniDam and fold cervically.

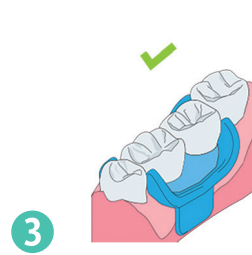

3 The DMG MiniDam stays in place securely without clamps.

The DMG MiniDam protects the proximal area during treatment.

Simplify Your Procedures

- Ideal for bleaching, acid etching or Icon resin infiltration
- Reliable local protection
- Quick and easy
- Comfortable for patients
- Secure hold without clamps
- Latex-free

Package Contents

Item #	Product
220381	MiniDam (20 pieces)

For more information, or to order MiniDam, contact your authorized dental supplier, call 800-662-6383 or visit dmg-america.com.



Works great with Icon

[Learn More](#)

www.dmg-america.com/minidam



DMG America

242 South Dean Street
Englewood, NJ 07631

P: 800-662-6383 201-894-5505

F: 201-894-0213 www.dmg-america.com

DMG
AMERICA