MiniDam

Gingivaschutz

DMG Chemisch-Pharmazeutische Fabrik GmbH
 Elbgaustraße 248 · 22547 Hamburg · Germany
 www.dmg-dental.com
 091956/#1-2015-01

DE	Gebrauchsinformation	DA	Brugsanvisning
EN	Instructions for use	SV	Användningsinformation
FR	Mode d'emploi	PL	Wskazówki dotyczące użycia
IT	Istruzioni per l'uso	RU	Инструкция по применению
ES	Instrucciones de empleo	ZH	使用说明书
PT	Instruções de uso	JP	取扱説明書
NL	Gebruiksaanwijzing	KO	사용 설명서

Figures

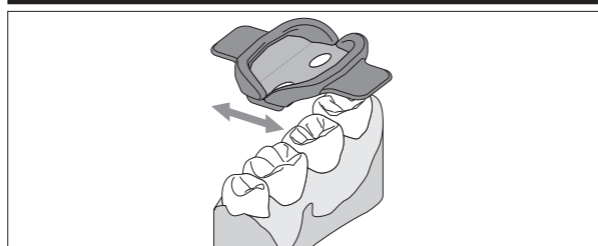
Fig. 1

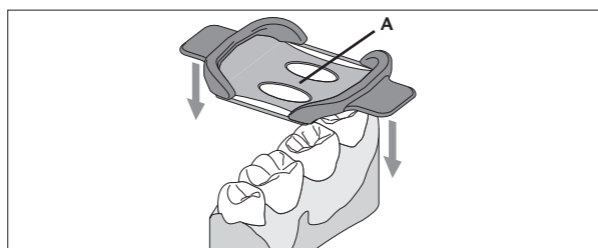
Fig. 2

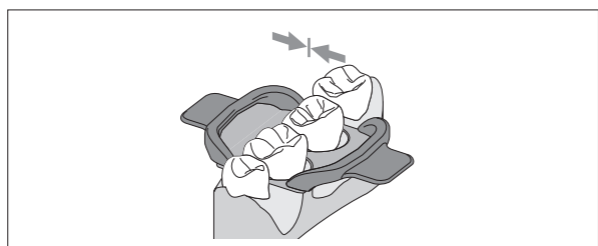
Fig. 3

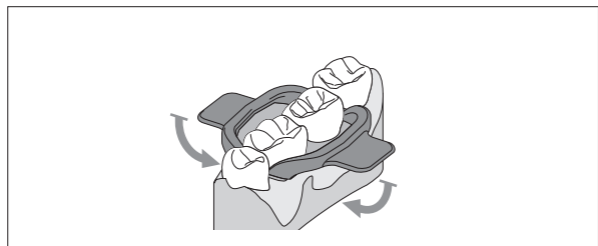
Fig. 4

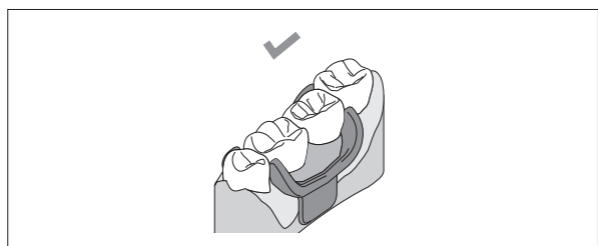
Fig. 5

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Deutsch

Produktbeschreibung

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English

Product description

The gingival protection device is comprised of silicone and is used to facilitate a relatively dry environment in the proximal area between two teeth and to protect the gingiva against dental materials.

Indications

Establishing a relatively dry environment in the case of complete dentition, for example when using Icon or Composite filling materials.

Contraindications

- Not suitable for establishing absolutely dry environments.
- Do not use in cases of allergies to one of the ingredients or in the event of contact allergies.

Basic safety information

- For dental use only!
- Keep out of the reach of children!

Side effects

No systemic side effects are known to date. Hypersensitivity to components of the material or contact allergies cannot be excluded in individual cases.

Instructions for use

Avoid damaging the gingival protection device with dental floss or other materials. Do not use damaged gingival protection devices.

Recommended use

1. Clean the working area.
2. If the teeth are very close together: separate the teeth using a wedge.
3. Pre-stretch the gingival protection device by gently pulling the tabs apart.
4. Place the gingival protection device over the affected teeth and pull the tabs apart (Fig. 1).
5. Place the interdental section of the gingival protection device in the proximal position.

Note: The middle interdental section is used to protect the working area (Fig. 2: A).

6. Pull the gingival protection device towards the gingiva until the interdental sections are positioned in the proximal area (Fig. 2).
7. Release the gingival protection device (Fig. 3).
8. Fold the gingival protection device cervically where the teeth are isolated.
9. Fold the tabs cervically (Fig. 4).
10. Remove wedges as appropriate.
11. Verify that the gingival protection device is properly seated and correct if required (Fig. 5).
12. Further measures to facilitate a dry environment (e.g. cotton rolls, aspiration) can be carried out as required.
13. Perform treatment.
14. Remove the gingival protection device and discard.

Disposal

After use, dispose of the product in household waste.

Storage and shelf life

- ▶ Store in a dry place (15-25°C/59-77°F) at room temperature.
- ▶ Do not use after the expiration date!

Composition

Silicone elastomers, pigments, additives

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