

# How to open Optishade images in Matisse Software

Step by step guide

26-7-2021 by Marat Awdaljan



Optishade not connected: press button on device for 3 seconds and release



MEASURE



GALLERY



COMPARE



CASES



CALIBRATION




SETUP


4.2



**1. Click on *setup***

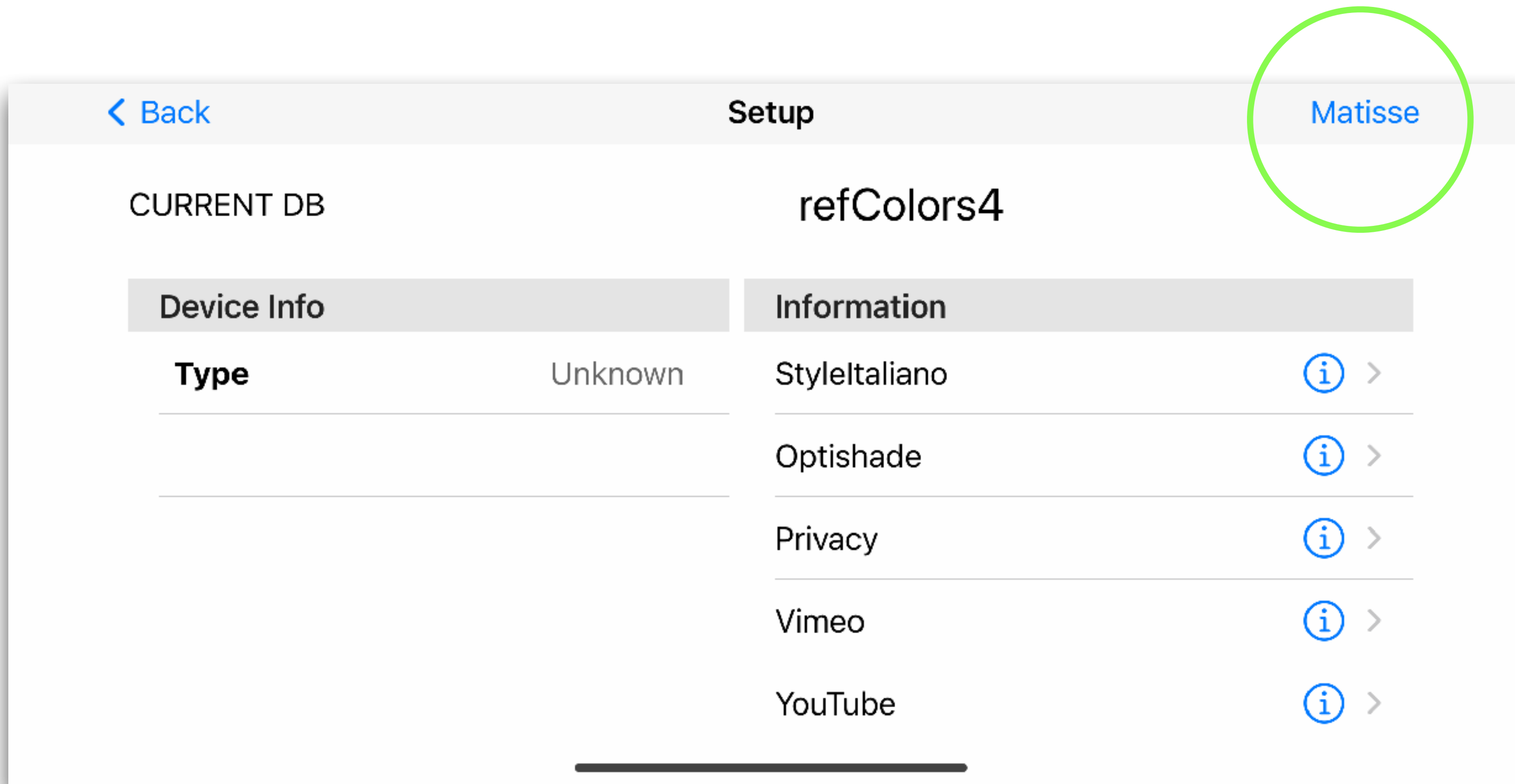

**2. Click on *Matisse***

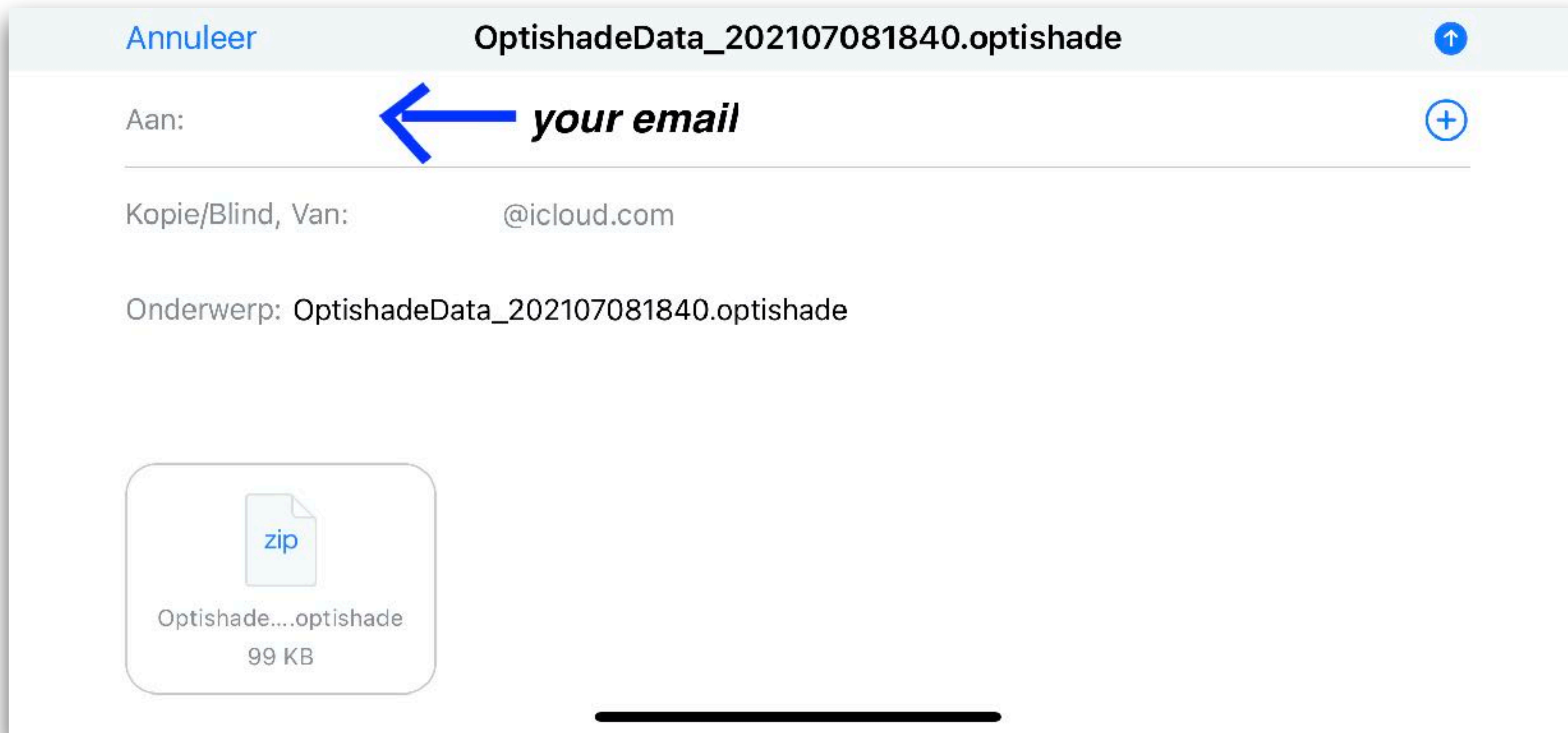

### 3. Fill in your email address

Optishade not connected: press button on device for 3 seconds and release



MEASURE



GALLERY



COMPARE



CASES



CALIBRATION




SETUP


4.2



### 3. Go to *Gallery*

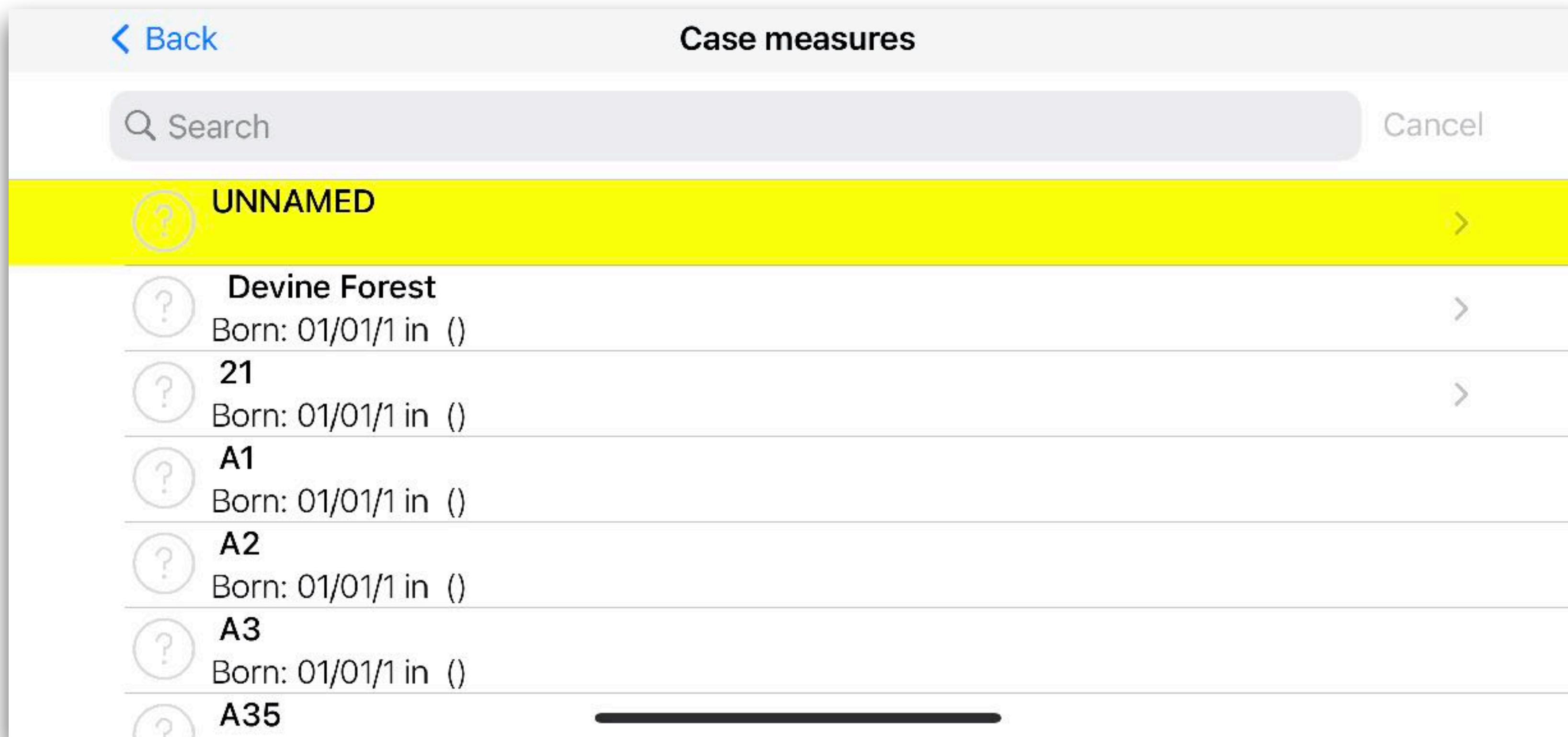


## 4. Select the folder

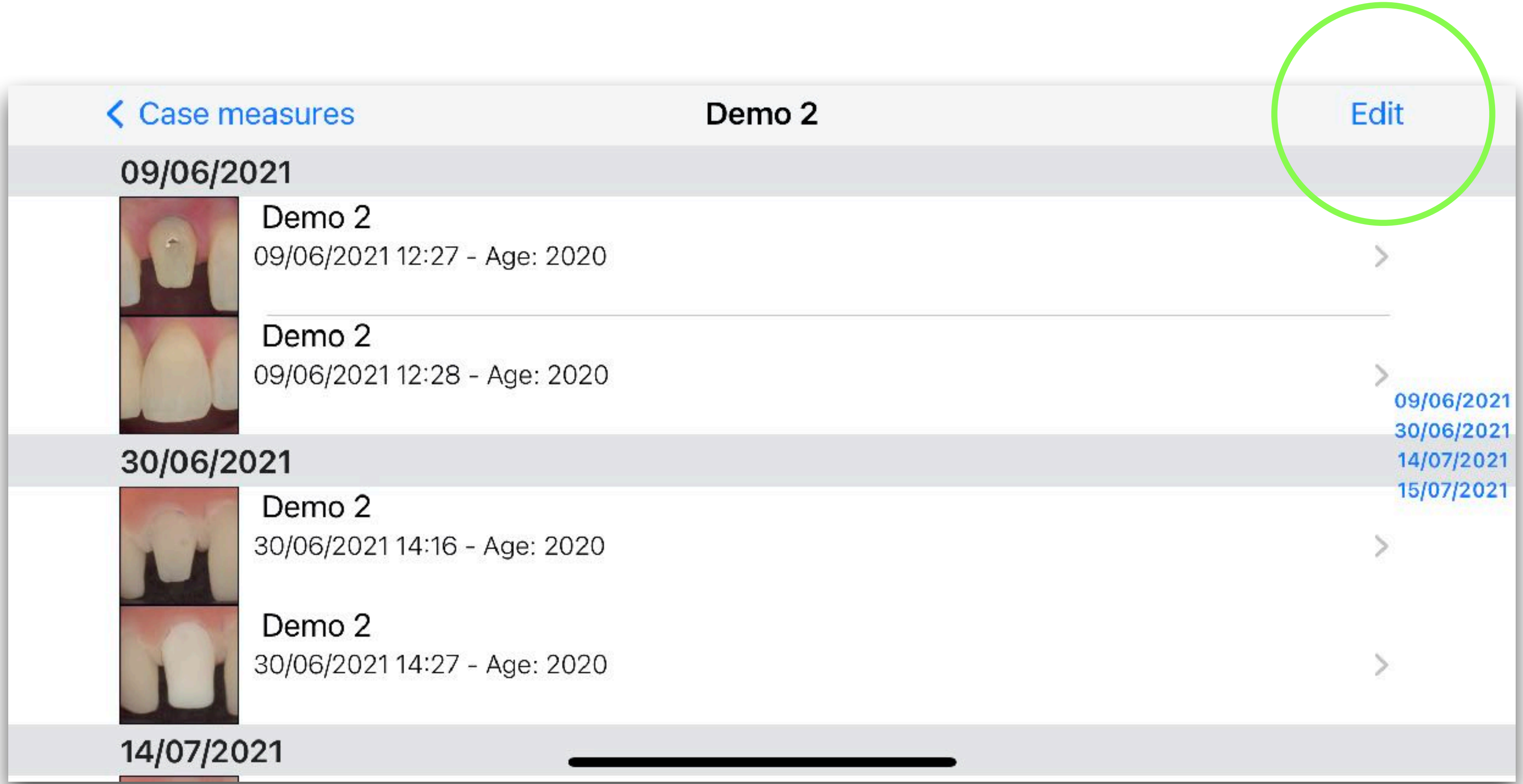


**5. Click on *Edit***



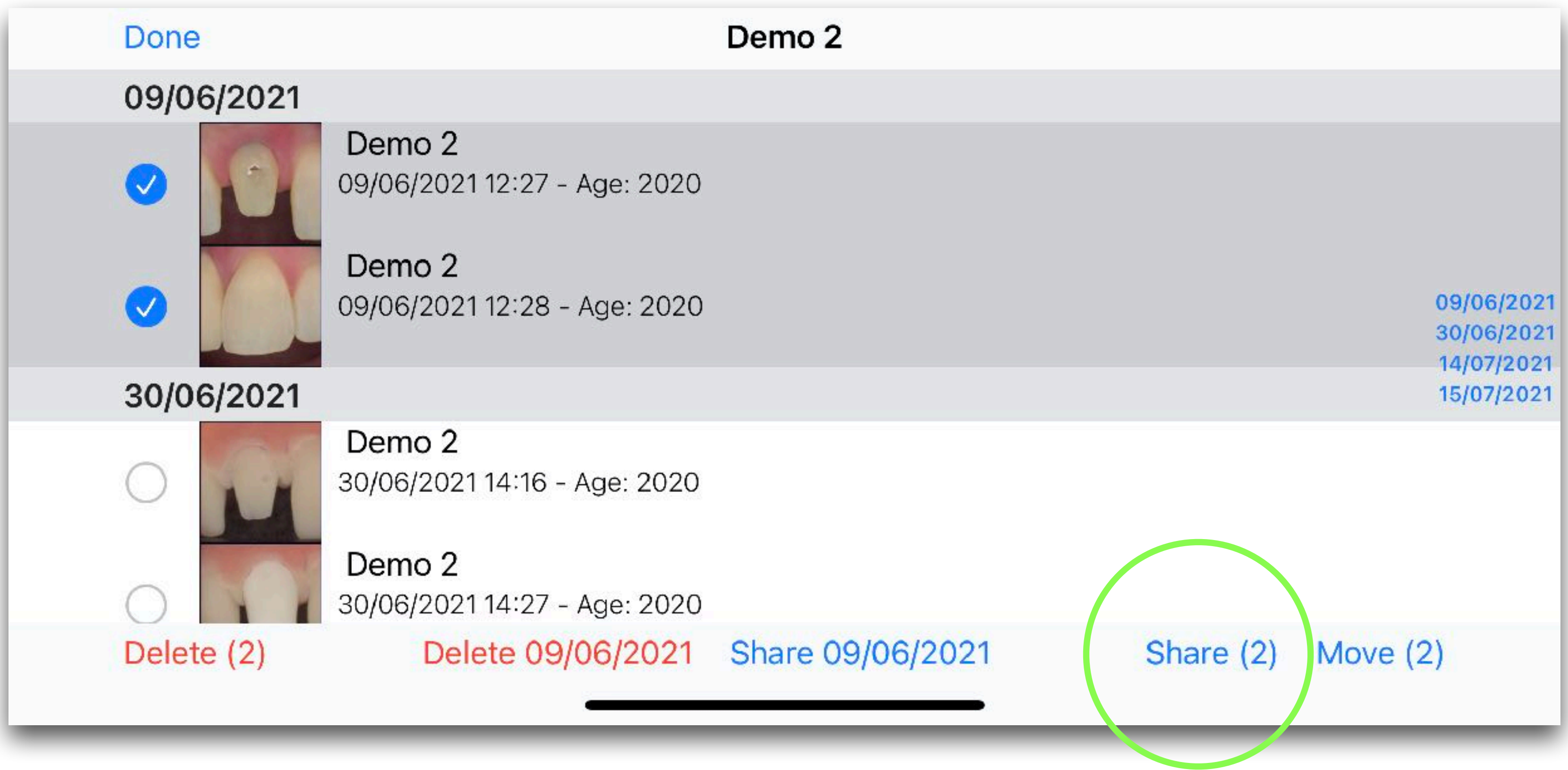

**6. Select the images and click on *Share***




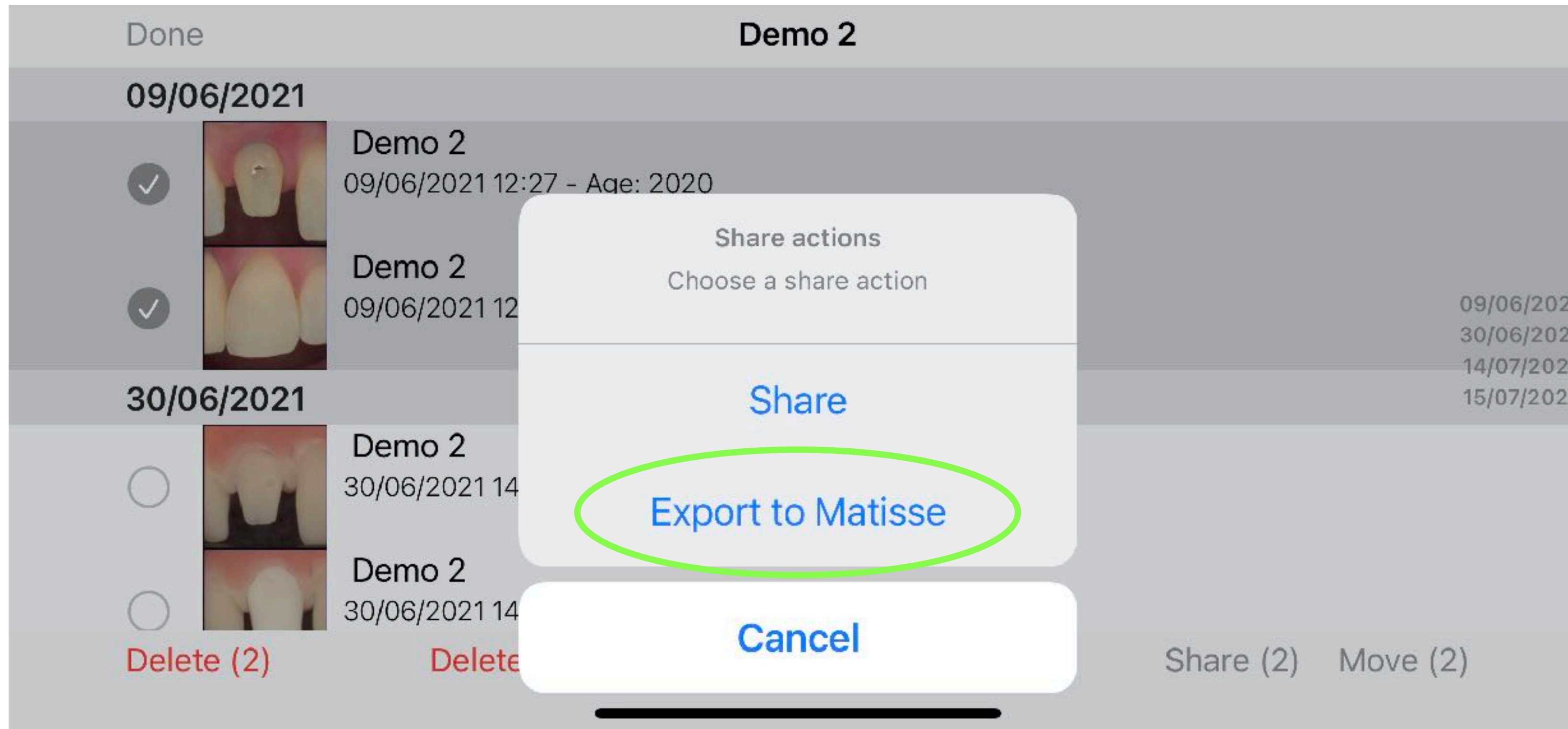


## 7. Select *Export to Matisse*



**8. This file will be send to your email.  
Please save it on your pc!**

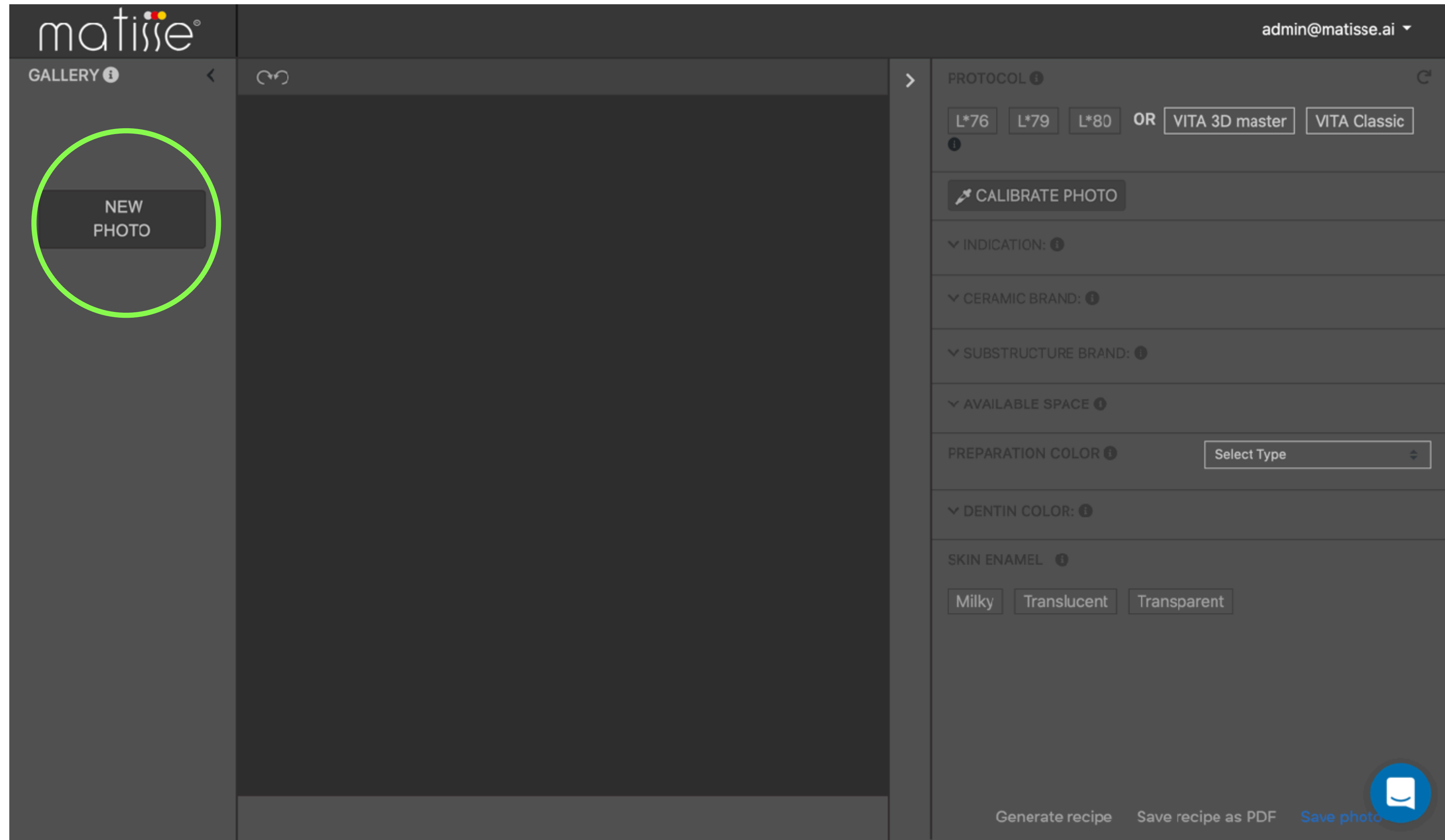


**9. Open Matisse and click on *New Photo***

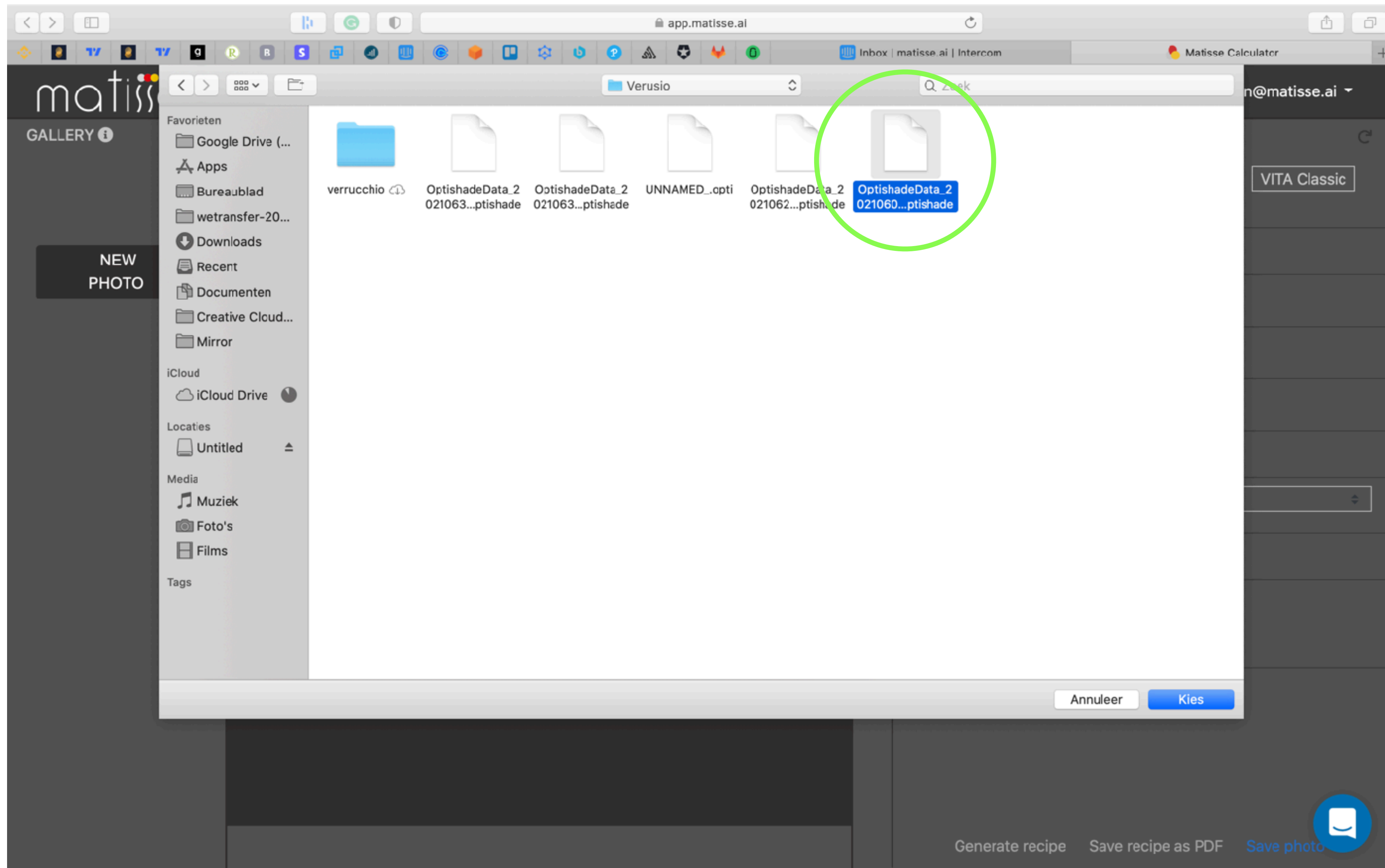

## 10. Select the optishade file



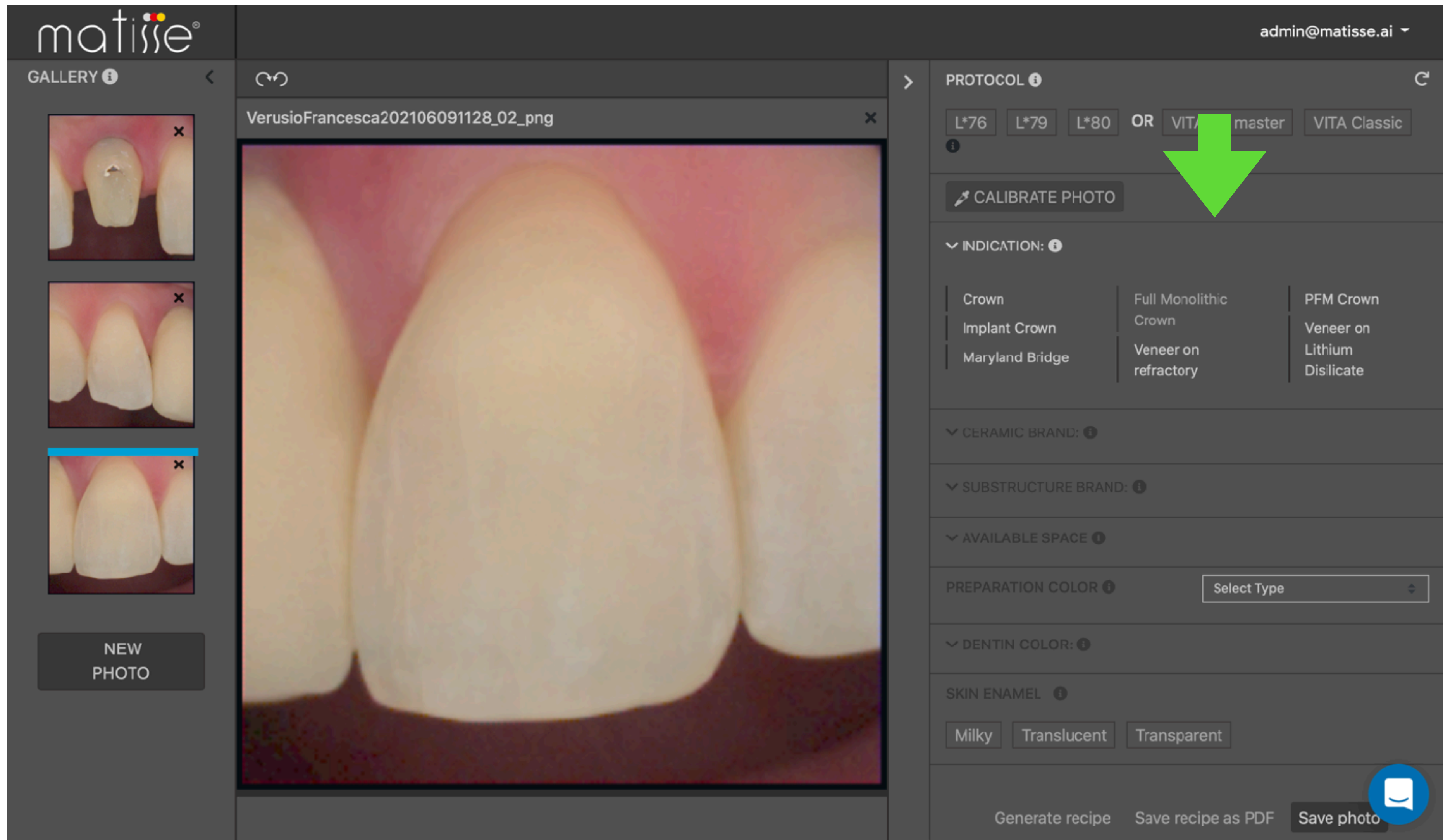

**11. Fill in the form on the right side to get a recipe**

# Thank you!

for further help please contact [info@matisse.ai](mailto:info@matisse.ai)

matisse®

