# Sanctuary™

DENTAL DAM SYSTEMS

**Sanctuary™**

**Sanctuary™**  
**Latex Dental**

**THE DENTAL ADVISOR**  
 ★★★★★

- Dental dam**
  - powder free, non-stick, single use only, for professional use only
- Digue dentaire**
  - sans poudre, non-stick, à usage unique, exclusivement réservé à un usage professionnel
- Kofferdam**
  - powder free, non-stick, for one-time/one-patient use only, for professional use only
- Diga dentale**
  - senza polvere, non-stick, monouso, esclusivamente per uso professionale
- Diques de goma**
  - sin polvo, no-stick, de un solo uso, recomendado solo para uso profesional

6" x 6" 36 sheets SCENT:  PLAIN  MINT

Recommended For Professional Use Only

COLOR:  BLUE  GREEN

THICKNESS:  THIN  MEDIUM

**Sanctuary™**  
**Non Latex Dental dam**

**THE DENTAL ADVISOR**  
 ★★★★★

- Dental dam**
  - powder free, non-stick, single use only, for professional use only
- Digue dentaire**
  - sans poudre, non-stick, à usage unique, exclusivement réservé à un usage professionnel
- Kofferdam**
  - powder free, non-stick, for one-time/one-patient use only, for professional use only
- Diga dentale**
  - senza polvere, non-stick, monouso, esclusivamente per uso professionale
- Diques de goma**
  - sin polvo, no-stick, de un solo uso, recomendado solo para uso profesional

6" x 6" 36 sheets SCENT:  MINT (PURPLE)

Recommended For Professional Use Only

**Sanctuary™**  
**Powder Free Latex Dental dam**

**THE DENTAL ADVISOR**  
 ★★★★★

- Dental dam**
  - powder free, non-stick, single use only, for professional use only
- Digue dentaire**
  - sans poudre, non-stick, à usage unique, exclusivement réservé à un usage professionnel
- Kofferdam**
  - powder free, non-stick, for one-time/one-patient use only, for professional use only
- Diga dentale**
  - senza polvere, non-stick, monouso, esclusivamente per uso professionale
- Diques de goma**
  - sin polvo, no-stick, de un solo uso, recomendado solo para uso profesional

Thin  Medium  Heavy

6" x 6" 36 sheets SCENT:  MINT

Recommended For Professional Use Only

**Sanctuary™**  
**Latex Dental dam**

**THE DENTAL ADVISOR**  
 ★★★★★

**Silk Blue**

- Dental dam**
  - powder free, non-stick, single use only, for professional use only
- Digue dentaire**
  - sans poudre, non-stick, à usage unique, exclusivement réservé à un usage professionnel
- Kofferdam**
  - powder free, non-stick, for one-time/one-patient use only, for professional use only
- Diga dentale**
  - senza polvere, non-stick, monouso, esclusivamente per uso professionale
- Diques de goma**
  - sin polvo, no-stick, de un solo uso, recomendado solo para uso profesional

6" x 6" 36 sheets SCENT:  MINT

Recommended For Professional Use Only

THICKNESS:  THIN  MEDIUM  HEAVY

# Dental Dams

# Powder Free **Latex** Dental Dam


Sanctuary ... the world's first powder-free natural rubber latex dental dam.

## Cutting-edge Advantage

Sanctuary Dental Dam is specially designed to be resilient, with a high tear tolerance to allow for the easy fixing of clamps.

It is the world's first latex dental dam which is powder free and low protein, making it the excellent choice for professional use.

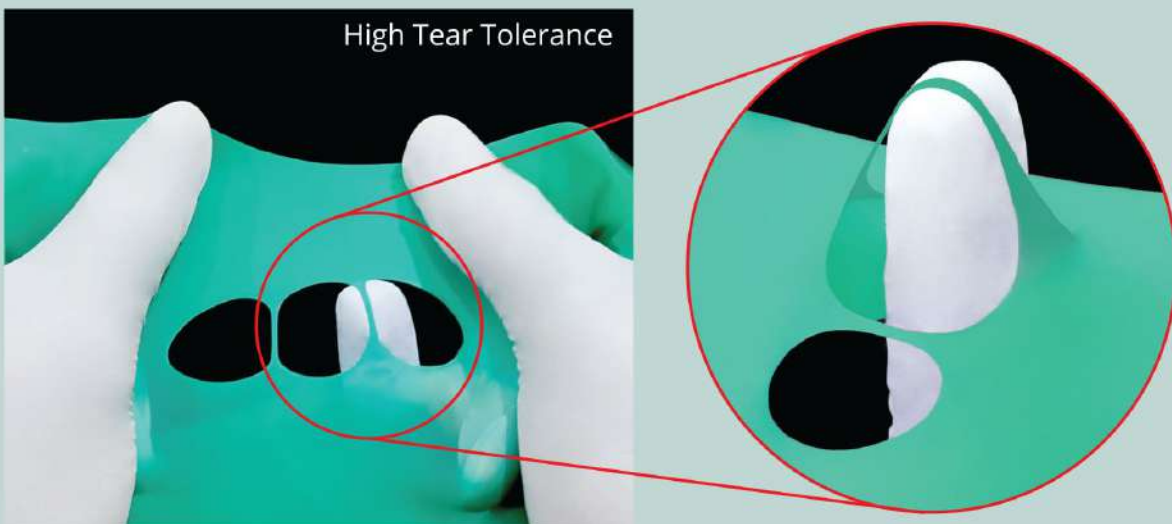
Strength	Thin / Medium / Heavy
Dimension	127 x 127mm (5 inch x 5 inch ) OR 152 x 152 mm (6 inch x 6 inch )
Packing	5" x 5" - 52 Sheets per box 6" x 6" - 36 Sheets per box
Edging	Square cut
Physical Strength	- Tensile Strength: 24.0 MPa (min) - Ultimate Elongation: 700% (min)
Extractable Protein	Less than 50 microgramme / dm <sup>2</sup>
Residual Powder	Less than 2.0 milligramme per dental dam
Scent & Colour	Mint (Green) Plain (Blue)
Packaging	52 pieces / box (for 127 x 127mm) 36 pieces / box (for 152 x 152mm)
Carton & Size	40 Boxes per carton for 127 x 127mm, Carton Size 30.3x30.3x20.8cm (LxWxH) for 152 x 152mm, Carton Size 34.5x34.5x18.0cm (LxWxH)
Certification	CE and FDA registered

## The Significance of Being Powder-free and Low Protein

Plant proteins present in natural rubber is a source of irritation and allergy to some people, causing negative and adverse reactions. These reactions and allergies are significantly and drastically reduced by reducing the protein levels and eliminating powder residues of natural rubber products during production, thus making such products safer and more accessible.


## Powdered Sample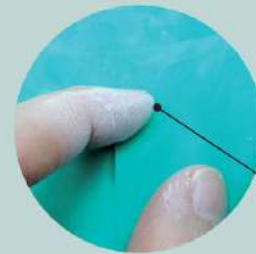

Residue of powder is found

Generally it will be more dull in appearance where it's light powder is sensible. Based on studies, powdered product such as latex gloves where its allergen content is higher in powdered gloves.

## Sanctuary™ Powder Free

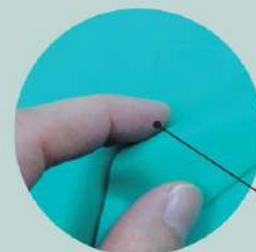No residue of powder


Visually, it looks more shiny and in addition it feels much more fair and smoothly. Sanctuary's Powder Free feature in Dental Dam which helps to address the proteins allergy concern by eliminating the chance of the powder which act as the transporter of protein into the user's skin during contact.

# Powder Free **Non Latex** Dental Dam

## Sanctuary Powder-free Non-Latex Dental Dam

Sanctuary's non-latex dental dam is made from Polyisoprene, a material which very closely resembles rubber. It has a similar polymer structure and physical properties as natural rubber minus the potential allergen of latex protein.

Designed and developed as a superior product, the Sanctuary Non-Latex Dental Dam offers a high tear tolerance (a minimum expansion of 10 times from the original punch hole), great tensile strength (above 20.0MPa) and a good modulus level (1.0MPa), all of which are crucial in the practical and efficient application of the dental dam.

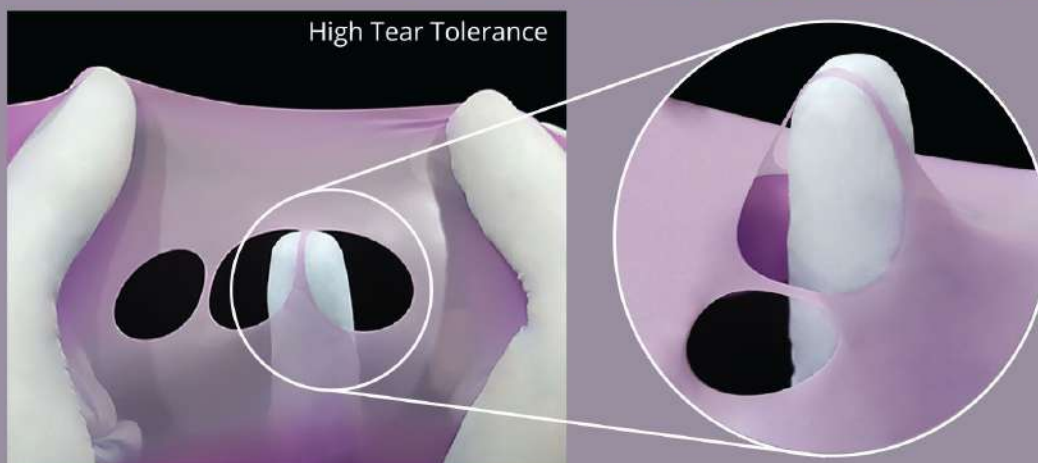DENTAL ADVISOR  
PREFERRED PRODUCT  
**DENTAL DAM:  
NON-LATEX**

## Stretchability

The Sanctuary Non Latex dental dam is made to achieve a very high stretchability consistent with a good tear tolerance.

Before Elongation

After Elongation

Strength	Medium
Dimension	127 x 127mm (5 inch x 5 inch ) or 152 x 152 mm (6 inch x 6 inch )
Packing	5" x 5" - 15 Sheets per box 6" x 6" - 15 Sheets per box
Edging	Square cut
Physical Strength	- Tensile Strength: 20.0 MPa (min) - Ultimate Elongation: 700% (min)
Residual Powder	Less than 2.0 milligramme per dental dam
Scent	Mint
Colour	Purple
Packaging	15 pieces / box
Carton & Size	40 Boxes per carton for 127 x 127mm, Carton Size 30.3x30.3x20.8cm (LxWxH) for 152 x 152mm, Carton Size 34.5x34.5x18.0cm (LxWxH)
Certification	CE and FDA registered

# PHOTOGRAPHY

DENTAL ADVISOR  
TOP AWARD  
DENTAL  
DAM: LATEX

DENTAL ADVISOR  
TOP AWARD  
DENTAL DAM:  
LATEX

# Powder Free Latex Black Dental Dam

## Sanctuary Powder-free Latex Black Dental Dam

Formulated and designed for optimum photographic results, having a clear and distinctive photo outcome is crucial and necessary for dentist to make effective studies of the tooth during procedures.

Sanctuary Latex Black Dental Dam is a great alternative for dental photography compared to conventional coloured dental dams.

### Benefits:

- Powder Free
- Low Protein
- Greater Visual Clarity
- Clear Operating Field
- Effective Barrier for Infection Control
- High Tear Tolerance

Strength	Medium
Dimension	152 x 152 mm (6 inch x 6 inch)
Packing	6" x 6" - 36 Sheets per box
Edging	Square cut
Physical Strength	- Tensile Strength: 24.0 MPa (min) - Ultimate Elongation: 700% (min)
Extractable Protein	Less than 50 microgramme / dm <sup>2</sup>
Residual Powder	Less than 2.0 milligramme per dental dam
Scent	Mint
Colour	Black
Packaging	36 pieces / box (for 152 x 152mm)
Carton & Size	40 Boxes per carton for 152 x 152mm, Carton Size 34.5x34.5x18.0cm (LxWxH)
Certification	CE and FDA registered

## Colour Contrast

Designed to provide the ultimate contrast colour to allow clearer visual of the tooth or teeth during procedures and dental photography.

Developed for

*Better*

*Vision Comfort*

# Powder Free Silk Blue Dental Dam

## Sanctuary Powder-free Latex Silk Blue Dental Dam

Specially developed to help reduce eye fatigue from reflection of procedure lighting and from long hours of concentration.

The soft blue colour provides comfort to the eyes and helps to maintain better vision during procedure.

### Benefits:

- Powder Free
- Low Protein
- Better Vision Comfort
- Clear Operating Field
- High Tear Tolerance

Strength	Thin / Medium
Dimension	127 x 127mm (5 inch x 5 inch ) OR 152 x 152 mm (6 inch x 6 inch )
Packing	5" x 5" - 52 Sheets per box 6" x 6" - 36 Sheets per box
Edging	Square cut
Physical Strength	- Tensile Strength: 24.0 MPa (min) - Ultimate Elongation: 700% (min)
Extractable Protein	Less than 50 microgramme / dm <sup>2</sup>
Residual Powder	Less than 2.0 milligramme per dental dam
Scent	Mint
Colour	Silk Blue
Packaging	52 pieces / box (for 127 x 127mm) 36 pieces / box (for 152 x 152mm)
Carton & Size	40 Boxes per carton for 127 x 127mm, Carton Size 30.3x30.3x20.8cm (LxWxH) for 152 x 152mm, Carton Size 34.5x34.5x18.0cm (LxWxH)
Certification	CE and FDA registered

## Colour Contrast

Sanctuary Silk Blue Dental Dam is also designed for optimum photographic outcome which is a big plus for dentist to have a better and clearer study on the particular tooth.

A close-up photograph of several stainless steel dental dam accessories, including a clamp and a frame component, set against a dark background. The brand name 'Sanctuary' is visible on the metal parts. A red object is partially visible in the top left corner.

**Sanctuary™**

Dental  
Dam  
Accessories

# Dental Dam Premium Kit

## Dental Dam Accessories Product Specifications:

Product Name	Product Code	Product Description
Dental Dam Punch	RDPN1	Dental Dam Punch, 157mm (L)
Dental Dam Clamps	RDC210	Anterior for adult, 30.5mm (W) x 21.5mm (L)
	RDC211	Anterior for adult, 30.5mm (W) x 20.5mm (L)
	RDC207	Premolar for adult, 16.0mm (W) x 17.5mm (L)
	RDC2	Premolar for adult, 17.5mm (W) x 16.5mm (L)
	RDC2A	Premolar for adult, 17.5mm (W) x 17.5mm (L)
	RDC201	Molar for adult, 20.0mm (W) x 22.5mm (L)
	RDC202	Molar for adult, 20.5mm (W) x 24.5mm (L)
	RDC203	Posterior for children, 16.5mm (W) x 19.0mm (L)
	RDC204	Posterior for children, 16.5mm (W) x 19.5mm (L)
	RDSTD2	Dental Dam Clamps with Organizer, 3cm (H) x 10cm (W) x 15cm (L) Included Dental Dam Clamps: 1.) RDC210           6.) RDC201 2.) RDC211           7.) RDC202 3.) RDC207           8.) RDC203 4.) RDC2              9.) RDC204 5.) RDC2A
	Dental Dam Forceps	RDPL1
Dental Dam Frames	RDFR1	Dental Dam Frame, 110mm (W) x 95mm (L)
	RDFR2	Dental Dam Frame, 135.5mm (W) x 100mm (L)

 The Application of  
Dental Dam Video Included

# Dental Dam Punch

Sanctuary Dental Dam Punch is developed specially for the purpose of its durability in obtaining a perfect punch hole on a dental dam.

## Quality Punch Every Time

The key to a perfect punch on a high tear tolerance Sanctuary Dental Dam is a hardened sharp edge on the punch. At Sanctuary, the Stainless Steel 304 is a grade put through a special "work-hardened" process in order to hardened the 304 grade further.

## Long Lasting on Repetitive Application

Sanctuary Dental Dam Punch is engineered to withstand the repetitive stress in application during every punch and the sterilization/cleaning process which involves sterilization substances.

## Safe for Autoclaving

Sanctuary Dental Dam Punch are rust proof, highly corrosion resistant and has very high heat resistant. Therefore, it is safe for autoclaving.

Sanctuary Dental Dam Punch is rated highly among users as a reliable punch.

# Dental Dam Forceps

The Sanctuary Dental Dam Forceps are designed to allow for easy placement of any clamp size with infinite precision.

## Lightweight and Easy Gripping

Sanctuary Dental Dam Forceps are lightweight with easy gripping handle will promotes extra comfort and confidence during usage.

## Long Lasting on Repetitive Application

The Sanctuary Dental Dam Forceps are made from Stainless Steel 304. As such this forceps can withstand the repetitive stress in application and the sterilization / cleaning process involves sterilization substance. With the properties of Stainless Steel 304 material, Sanctuary Dental Dam Forceps can be cleaned easily and used for longer time.

## Safe for Autoclaving

Sanctuary Dental Dam Forceps are rust proof, highly corrosion resistant and has high heat resistant. Therefore, it is safe for autoclaving.

Sanctuary Dental Dam Forceps are rated highly among users.

# Dental Dam Clamps with Organizer

Sanctuary Dental Dam Clamps are developed with a “flexible but strong” concept. Engineered to this concept of application, we use a double vacuum-melted and martensitic age-hardenable alloy - the Stainless Steel Grade 465.

## Strong and Lasting Retention

Sanctuary Dental Dam Clamps allows a good retention of its original shape through repeated usage. The Stainless Steel Grade 465 has achieved tensile strength above 2000 MPa. As such, such material is most suitable for the retention property in the application of a dental dam clamp.

**Suitable for Sterilization** – Dental Dam Clamps are used in contact with human fluid such as blood and saliva over long period of time. As such, it is subject to very intensive sterilization process. Sanctuary Dental Dam Clamps are therefore engineered to accommodate these sterilization processes.

## Resistant to Stress-corrosion

Sanctuary Dental Dam Clamps are highly resistant to stress corrosion making the product performing its application longer.

## Safe for Autoclaving

Sanctuary Dental Dam Clamps are safe for autoclaving and resistant to surface oxidation in steam environment.

A new introduction into the market, Sanctuary's new dental dam clamp offers innovative characteristics and further facilitates the use of Sanctuary Dental Dam.

## Description

Sanctuary Dental Dam Clamp is a tool to hold down dental dam to the cervical region, away from the rest of the exposed teeth.

## Significant Points of Difference

1. It is designed in such a way that it does not press on any peripheral soft tissue during use.
2. The hole and furrow are shaped to allow dental dam to be easily removed by dental instruments.
3. Made of a hard and highly durable metal, it has a superior clamping force.

*Furrow structure for easy removal of dam (applied for a patent)*

Clamp No. 2


Clamp No. 2A


Clamp No. 207


Clamp No. 201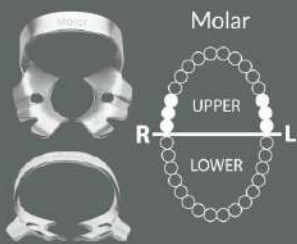

Clamp No. 202

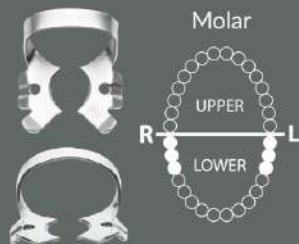
Clamp No. 210

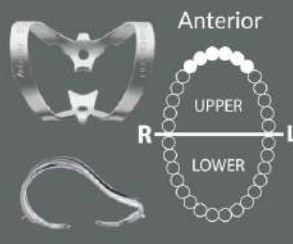
Clamp No. 203

Clamp No. 204

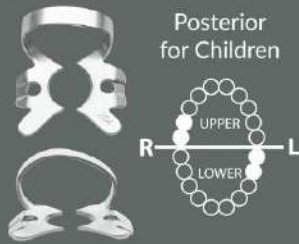Clamp No. 211
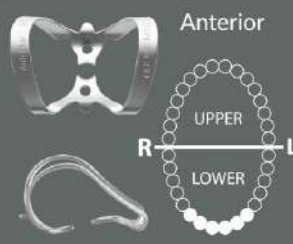
# Dental Dam Frames

Sanctuary Dental Dam Frames are developed to maintain the borders of the Sanctuary Dental Dam in position.

The Sanctuary Dental Dam Frames come with two sizes 105 mm and 129mm.

Sanctuary Dental Dam Frames are made from high grade Stainless Steel 304 which is high corrosion and high heat resistance.

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**Wellkang Ltd**