



## Sterilization Instructions:

### **AFTER EACH PATIENT**

Run the VacuLUX HVE Adapter with a non-corrosive, pH-neutral enzymatic cleaning solution and perform a surface disinfection with a disinfectant wipe.

### **DAILY**

Use a stiff brush and mild detergent to clean internal surfaces.

### **AUTOClave INSTRUCTIONS**

Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 135°C/275°F. 6 minutes holding time, minimum. Avoid contact of sterilizer wall and avoid close proximity to heating elements. Store only in FDA approved sterilization pouch.

Ultrasonic cleaning is not recommended.

**(616) 600-4505**

[www.AscentcareDental.com](http://www.AscentcareDental.com)



SKU: VLUX

## Instructions for use:

1. When the adapter is sterilized and ready to use, spray air and water inside the mouthpiece to lubricate.
2. Slide mouthpiece onto the adapter until the locking post engages.
3. Connect the adapter to your standard HVE valve, hose or the Whisper Light kit.
4. Insert the mouthpiece into the patient's mouth according to the mouthpiece manufacturer's instructions.
5. For optional illumination, magnetically couple the HyperLUX light guide to the adapter and operate the light engine according to the HyperLUX instructions.

## Sterilization Instructions:

### AFTER EACH PATIENT

Run the VacuLUX HVE Adapter with a non-corrosive, pH-neutral enzymatic cleaning solution and perform a surface disinfection with a disinfectant wipe.

### DAILY

Use a stiff brush and mild detergent to clean internal surfaces.

### AUTOClave INSTRUCTIONS

Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 135°C/275°F. 6 minutes holding time, minimum. Avoid contact of sterilizer wall and avoid close proximity to heating elements. Store only in FDA approved sterilization pouch.

Ultrasonic cleaning is not recommended.



(616) 600-4505

[www.AscentcareDental.com](http://www.AscentcareDental.com)