



VacuVUE

HVE DENTAL MIRROR

Instructions For Use

USING A STANDARD HVE VALVE

Simply & securely connect the VacuVUE to your existing HVE valve like a standard suction tip.

USING YOUR OPTIONAL WHISPER LITE KIT

For a lightweight feel, improved maneuverability & reduced fatigue, use the Whisper Lite Kit.

1. Securely insert the curved stainless-steel tube into your HVE valve like a suction tip.
2. Rest your valve in its docking stand.
3. Connect your VacuVUE mirror to the aluminum coupler which allows for 360° rotation and maintains an air-tight seal.



INTENDED USE:

The VacuVUE HVE mirror is intended to provide oral high-volume evacuation, visualization and retraction during dental procedures. The device is intended to be used by trained dental professionals only.

INSTRUCTIONS FOR USE:

Before patient use, the VacuVUE mirror must be cleaned & sterilized. Follow sterilization instructions as directed below.

1. Ensure that the mirror is properly and securely installed by following installation instructions.
2. Ensure that your HVE suction system is turned on.
3. On your HVE valve, switch the lever from off to on.
4. Once on, suction should flow through the VacuVUE mirror evacuation ports and the mirror is ready to use.
5. Operate the on/off lever on your HVE valve as needed throughout your procedures.

To install:

To use:

VacuVUE AUTOCLAVE INSTRUCTIONS

Products should be cleaned between each patient. Clean as soon as possible to prevent soils from hardening.

1. Remove all visible debris and use a stiff brush and mild detergent to clean internal surfaces.
2. Process product through steam autoclave* following the manufacturer's instructions.

3. Do not exceed a temperature of 135°C/275°F. 6 minutes holding time, minimum.

4. Avoid contact of sterilizer wall and avoid close proximity to heating elements.

5. Store only in FDA approved sterilization pouch.

Ultrasonic cleaning is not recommended.

*Do not autoclave the Whisper Lite Kit. Refer to the instructions for use included with your Whisper Lite Kit for separate sterilization instructions.

MIRROR FACE REPLACEMENT

When natural scratches and wear occur, conveniently replace your VacuVUE mirror face.

1. To remove the old mirror, using gloved thumbs, carefully apply even pressure to the mirror face until the back-plate pops free.



① Use both thumbs for even pressure

2. Remove the O-ring with a tool such as a small scaler.

3. Remove and discard the old mirror.



Replace O-Ring for optimal seal ②

4. Replace with a new mirror, **beveled side facing down**.

5. Insert a new O-ring into the track firmly and lubricate.



⑤ O-ring



④ mirror face, **bevel side down**



6. On a hard surface, line up the back plate with the mirror, pegs upward.

7. While applying even pressure with two thumbs, push the mirror head down over the backplate until it pops securely into place.



Use both thumbs for even pressure ⑦

www.AscentcareDental.com/hve-mirror

Visit our website to learn more about your VacuVUE HVE Mirror



2359 Holton Road
Muskegon, MI 49445

(616) 600-4505