

Ascentcare

Dental Products

Whisper Lite Kit

SKU: WLK5





Instructions for use:

1. Connect your VacuLUX HVE Adapter to the aluminum coupler.
2. Connect your HVE Valve to the stainless steel tube.
3. Dock your HVE Valve at your docking station.
4. Attach the mouthpiece to your VacuLUX HVE Adapter

Sterilization Instructions:

Run the Whisper Lite hose with a non-corrosive, pH-neutral enzymatic cleaning solution and perform a surface disinfection with a hospital grade disinfectant wipe between each patient. Gamma & EtO sterilizable. Do not put in autoclave.

www.AscentcareDental.com

Visit our website to learn more about your Whisper Lite Kit
and compatible dental instruments.

2359 Holton Road, Muskegon, MI 49445

(616) 600-4505

support@ascentcaredental.com