





The compact full face mask designed to easily fit everyone

The AirFit F20 features an innovative InfinitySeal™ silicone cushion: tested to perform up to the highest therapy pressures, it's designed to comfortably fit patients of all facial shapes and sizes. It will appeal to patients looking for a less claustrophobic mask that delivers a clear line of sight.



In international studies, AirFit F20 fit 96.5% of all patients.^{1, 2, 3, 4} That means you can set patients up right the first time and send them home with confidence.

Robust seal

This mask features a forgiving cushion that has been tested to prevent blow-outs, accommodate patient movement and provide a robust seal across even the highest pressures.



Comfortable to use

The flexible fabric-lined frame adds further comfort while delivering all-round stability and greater visual freedom.

With the InfinitySeal design, the AirFit F20 adapts to the unique facial contours of each patient, so you can be confident of providing a secure and comfortable mask fit.



Visual freedom

Let patients see and do more in bed with an under-the-eye frame that combines all-round stability and less facial contact.

Soft and gentle

Specifically designed for extra softness and stability, the plush headgear helps ensure patients settle into therapy quickly and comfortably.

Fast fitting time

Less than one minute is needed to fit AirFit F20.¹ Taking the mask on and off is convenient and intuitive, with magnetic clips that guide the headgear to the frame in seconds.

Dependable seal


Set patients up with a reliable and effective seal thanks to the InfinitySeal silicone cushion – designed to prevent blow-outs, tolerate misfitting and seal well under variable pressures.

Soft and flexible frame

Helping support a wider range of patients, the soft, flexible frame adapts to different face shapes while its integrated padding adds comfort.

Flexible connection and movement

The elbow's quick release feature enables easier connection or disconnection at night, while its 360° rotation capability allows for greater freedom of movement.

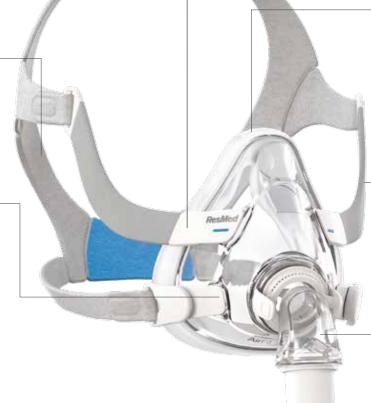




Designed specifically for women, it offers a smaller size to suit female facial contours.



In international studies, AirFit F20 was found to fit 96.5% of patients.^{1, 2, 3, 4}



Minimal parts make the AirFit F20 easy to use and clean

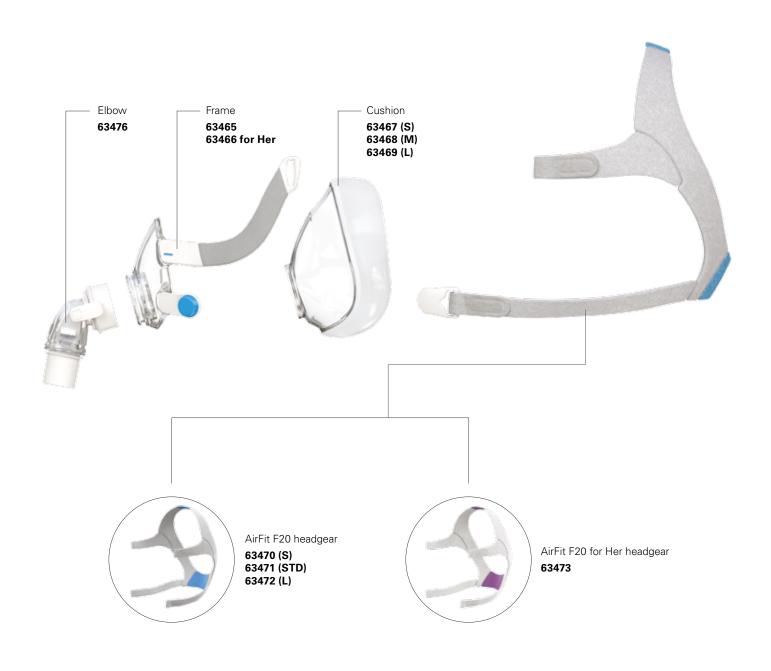


Sizing tips

AirFit F20 has three interchangeable cushion sizes - small, medium and large. The modular design of the AirFit F20 means each size cushion easily clicks into the same mask frame, so you can swap between them with ease for more flexibility.

We recommend that you refer to the mask fitting template to gauge which size is the most suitable for each of your patients.

You can download it on ResMed.com/AirFitF20

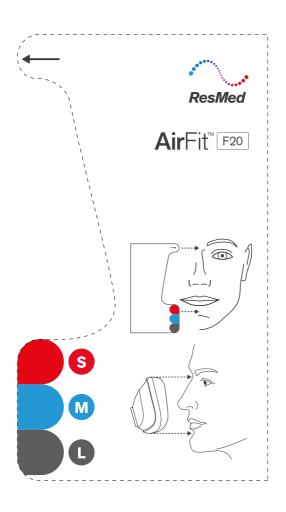

Product codes

	Europe 1	Europe 2	Europe 3	APAC	ANZ	China	Japan
AirFit F20: Small	63405	63410	63415	63420	63430	63435	63425
AirFit F20: Medium	63406	63411	63416	63421	63431	63436	63426
AirFit F20: Large	63407	63412	63417	63422	63432	63437	63427
AirFit F20 for Her: Small	63408	63413	63418	63423	63433	63438	63428
AirFit F20 for Her: Medium	63409	63414	63419	63424	63434	63439	63429

Europe 1: English (UK), Arabic, Dutch (Belgium, Netherlands), French, German and Italian (Switzerland)


Europe 2: English, Danish, Estonian, Finnish, Norwegian, Russian and Swedish

Europe 3: English, Bulgarian, Czech, Greek, Hungarian, Polish, Portuguese, Romanian, Spanish and Turkish


APAC: English, Simplified Chinese, Traditional Chinese



86% of patients preferred AirFit F20's seal to that of the market-leading full-face mask.¹

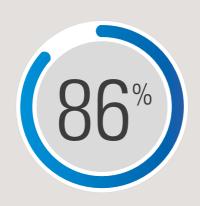

Evolving cushion technology: variable thickness for more comfort and stability

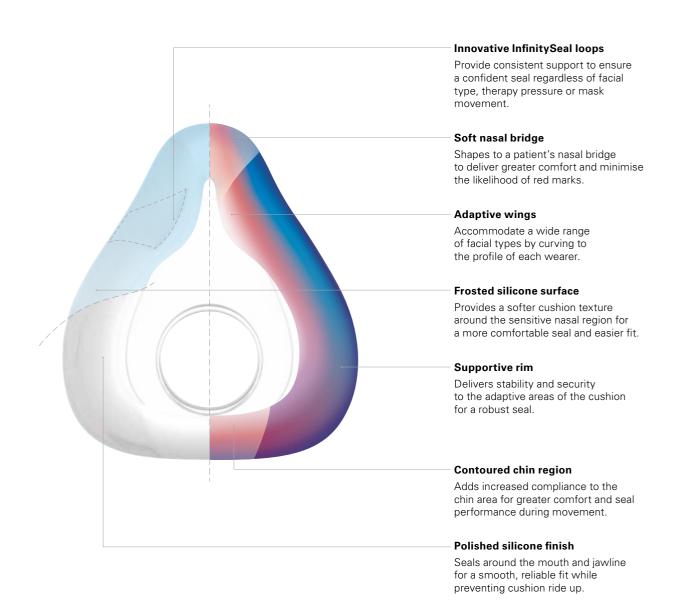



Fitting guide for AirFit F20



- Ensure the magnets on the lower straps are unclipped.
- If they are clipped, detach them by twisting and pulling them away from the frame.



- Hold the mask to the face, ensuring the top of the mask is aligned with the top of the nose bridge.
- Pull the headgear over the head making sure the ResMed logo is facing outwards.



- Position the lower straps so they both sit below the ears.
- Bring each of the magnets up to meet their corresponding clips on the frame.



- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
- Reattach the fastening tabs.



• Repeat Step 4 with the lower straps.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.

Refer to the user guide for further instructions.





Additional options

• A patient can position the lower part of the headgear over or under their hair to suit any hairstyle preference.

- 1 ResMed internal study of existing 22 ResMed patients, conducted between 26/04/2016 27/05/2016 comparing the market leading mask with AirFit F20. Preliminary patient study data on file; ID A3810791. 2 ResMed AirFit F20 internal fitting study of 27 ResMed and non-ResMed patients, conducted between 30/03/2016 04/04/2016. Preliminary patient study data on file; ID A3751086. 3 ResMed AirFit F20 internal international fitting study of 34 ResMed and non-ResMed patients, conducted between 11/04/2016 15/04/2016. Preliminary patient study data on file; ID A3774922.
- 4 ResMed AirFit F20 internal international fitting study of 90 ResMed and non-ResMed patients, conducted between 06/06/2016 22/06/2016. Preliminary patient study data on file; ID A3830701.



1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

Distributed by EC REP ResMed (UK) Ltd 96 Jubilee Avenue Milton Park Abingdon Oxfordshire OX14 4RW UK

AirFit and InfinitySeal are trademarks of ResMed Ltd. Specifications may change without notice. © 2016 ResMed Ltd. 1019492/1 2016-07v

