



EcoStar Info/Auto



T R E A T M E N T

www.sefam-medical.com



Customer Service
 Tel. +33 3 83 44 77 40
 Fax. +33 3 83 44 99 87
customerservice@sefam-medical.com



Technical Service
support@sefam-medical.com



After Sales Service
technicalservice@sefam-medical.com

Factory: SEFAM - 10 allée Pelletier Doisy - 54600 Villers-Lès-Nancy - FRANCE
 Manufacturer: SEFAM - 144 avenue Charles de Gaulle - 92200 Neuilly sur Seine - FRANCE

BRO123-03 / 05-2017



Operating mode	⊙	Standby mode	💡
Pressure symbol	P	Ramp feature	⚡
Compliance symbol	Y	Time symbol	🕒
Software version	Id * *	Adjustable parameter	⬆️
Error message	Er	Information message	In
		Pressure sensor offset	—

EcoStar Info/Auto - Menus and settings



Remark: Exit from the parameter setting menu takes place automatically when you do not push any key for 30 seconds.

EcoStar Info/Auto settings

APAP ↑ ↓	Mode: CPAP. APAP: Auto - CPAP.	CP:HH ↑ ↓	Pneumatic calibration: Configuration 22 (tubing \varnothing 22mm, standard) 15 (tubing \varnothing 15mm), HH (humidifier), SP (specific present) or - (specific absent).
10.0 cmH ₂ O ↑ ↓	Pressure: In CPAP mode.	FL: 1 ↑ ↓	Command on Inspiratory Flow Limitation: In APAP mode only. The command can be enabled (1) or disabled (0).
20.0 cmH ₂ O ↑ ↓	Maximum pressure: In APAP mode.	Sd: F ↑ ↓	Pressure decrease speed: The speed - slow (S) or fast (F).
4.0 cmH ₂ O ↑ ↓	Minimum pressure: In APAP mode.	A 12.0 cmH ₂ O ↑ ↓	Maximum pressure on apnea command: In APAP mode only.
1r ↑ ↓	Ramp: Time ramp (tr). Indexed ramp (ir).	1124 h ↑ ↓	Blower hours: Device operating time in total.
15 min ↑ ↓	Ramp time in CPAP mode. Latency time in APAP mode.	738 h ↑ ↓	Therapy hours: Time during which the patient actually breathed with the mask.
6.0 cmH ₂ O ↑ ↓	Comfort pressure.	2000 ↑ ↓	Pressure sensor offset: Integrated in the device.
Ro: 1 ↑ ↓	Auto ON: The feature can be enabled (1) or disabled (0).	RS: 0 ↑ ↓	Selection of RS232 connection for an external module: The connection can be enabled (1) or disabled (0).
CC: 1 ↑ ↓	Comfort calibration: The feature can be enabled (1) or disabled (0).	Id * * ↑ ↓	Software version: The version can be viewed but no modified.

Specific calibration

- Put the device in standby
- Set up the desired pneumatic configuration (humidifier, patient circuit, etc...) with the patient interface.
- **Keep the patient circuit straight and the air outlet unobstructed.**
- Bring up the screen page of the setting menu related to «Pneumatic Calibration».
- Select the specific patient circuit CP:SP or CP:-:

Specific calibration present

Specific calibration absent

CP:SP
↑ ↓

CP:-
↑ ↓

- Push the hidden key and hold it down until launch of calibration taking care not to touch the pneumatic circuit.

Setting the clock

- Put the device in standby.
- Push \odot for 2 seconds, then keeping it hold down, push \odot .

This screen appears:

8:29
↑ ↓

- Change the time displayed by using the \odot or \ominus .
- Push \odot to change minutes.
- Change minutes by using the \odot or \ominus .
- Push \odot to exit the menu.